Installation manual

MANUEL D'INSTALLATION
INSTALLATIEHANDLEIDING
INSTALLATIONSANLEITUNG
MANUALE DI INSTALLAZIONE
MANUAL DE INSTALACIÓN
INSTRUKCJA INSTALACJI
PŘÍRUČKU PRO INSTALACI

Navix SU

Contents

Preface	4
1. This product	6
2. Scope of delivery	7
3. Assembly and adjustments	8
3.1. Tools	8
3.2. Adjusting the wheelchair to the user	9
3.3. Changing the battery	17
3.4. Changing wheels or tyres	18

Preface

To support you on the installation and repairs of this powered wheelchair, we provide you with this installation manual. Please read it carefully. If you still have questions after reading this manual, do not hesitate to contact Vermeiren.

The information in this manual applies to the following wheelchair(s):

- Navix SU

Important note

Pictures of the product are used to clarify the instructions. Details of the depicted product may deviate from your product.

Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual For user and specialist dealer
	User manuals of applied operating console and battery charger For user and specialist dealer
	Installation instructions For specialist dealer
	Service manual for wheelchairs For specialist dealer
	Drawings of (spare) parts For specialist dealer

EC declaration of conformity

EN

1. This product

1. Backrest
2. Armrests
3. Arm pads
4. Operating console
5. Seat
6. Safety belt
7. Chest belt
8. Leg rest
9. Footplates
10. Driving wheels (front wheels)
11. Steering wheels (rear wheels)
12. Knee protector
13. Support wheels
14. Driving motors
15. Battery housing
16. Anti-tipping
17. Identification plate

2. Scope of delivery

The following components are part of the delivery:

- Framework with armrests, operator control, rear and front wheels, seat and backrest
- Footrests
- 1 footplate incl. knee/leg protector and support wheels
- Chest belt
- Batteries
- Battery charger
- Tools
- User manual of:
 - wheelchair
 - Operating console
 - Battery charger
- Options

Before use, check if everything is included and that no products are damaged.

Please note that the basic configuration may differ in different European countries. Contact your specialist dealer.

3. Assembly and adjustments

CAUTION

Risk of injuries or damage

- Only use the settings described in this manual.
 - Only use parts described in this manual.
 - Only use genuine spare or replacement parts approved by Vermeiren.
 - Prevent bystanders from accessing the working area.
 - Keep the (dis)assembling area clean and tidy, to prevent falling.
 - Collect all small parts in a box or bag.
 - While (dis)assembling and operating the wheelchair, make sure no objects or body parts get caught between moving parts. In particular, pay attention to the position of the cables.
 - Turn the wheelchair off before you perform any mechanical adjustments and make sure that the free-wheel lever is in braking position.
 - Make sure all screws are firmly secured before driving.
 - Software changes are only allowed if executed by skilled personnel, taking into account the user and environment (stability, acceleration, indoor/outdoor, etc.).
-  Read the user manual of this wheelchair. Be aware of the technical details and limits of intended use. For more information, contact Vermeiren.

3.1. Tools


To (dis)assemble and adjust the wheelchair, the following tools are needed:

- Wrench no. 10 to no. 13
- Allen key no. 4, 5, 6
- Screwdriver

3.2. Adjusting the wheelchair to the user

3.2.1. Seat depth

The wheelchair is adjustable in 4 different seat depths over a range of 75 mm (steps of 25 mm).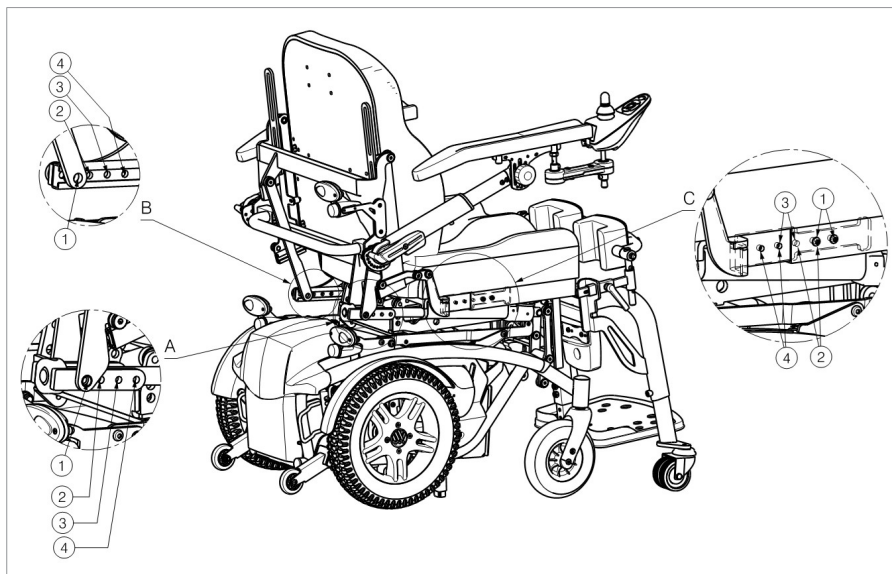

3.2.1.1. Detail A: Adjust the backrest motor.

1. Loosen the screw from the backrest motor.
2. Move the motor more backwards or forwards until the desired position and until the relevant holes lie above one another so that the fixing screws can be inserted.
3. Retighten the screw firmly.

3.2.1.2. Detail B: Adjust the telescopic guide.

1. Loosen the screw from the telescopic guide.
2. Move the telescopic guide more backwards or forwards until the desired position and until the relevant holes lie above one another so that the fixing screws can be inserted.
3. Retighten the screw firmly.

3.2.1.3. Detail C: Moving the seat frame.


1. Loosen the two screws on either side of the frame.
2. Pull the seat more backwards or forwards until the desired position and until the relevant holes lie above one another so that the fixing screws can be inserted through the frame.
3. Retighten the screw firmly.

Hole	Seat depth
1	470 mm
2	445 mm
3	420 mm
4	395 mm

3.2.2. Width between armrests

Adjust the width of the armrests as follows (range 25 mm: stepless / on each side):


1. Loosen the screw (1) on the back of the wheelchair just below the seat frame.
2. Adjust the armrest to a comfortable width.
3. Make sure the armrests are positioned identically on both sides of the seat.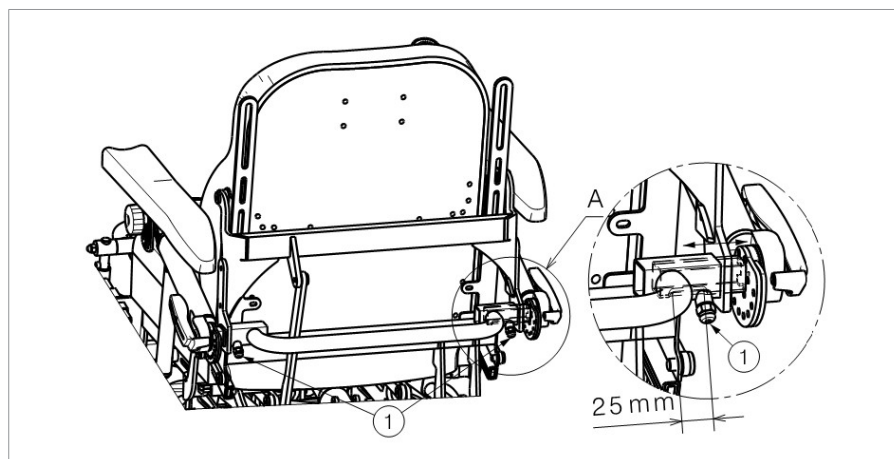

3.2.3. Armrest length

The telescopic armrest can be adjusted in 6 positions as follows:

1. If necessary, cut the cable ties that hold the power cable of the console.
2. Loosen the socket screw (33).
3. Slide the inner tube (34) in/out to the desired length.
4. Insert socket screw (33) and tighten properly.
5. Repeat this for the second armrest.
6. Make sure both armrests have the same length.
7. Make sure that both armrests are firmly secured.
8. Tighten the cable with cable ties.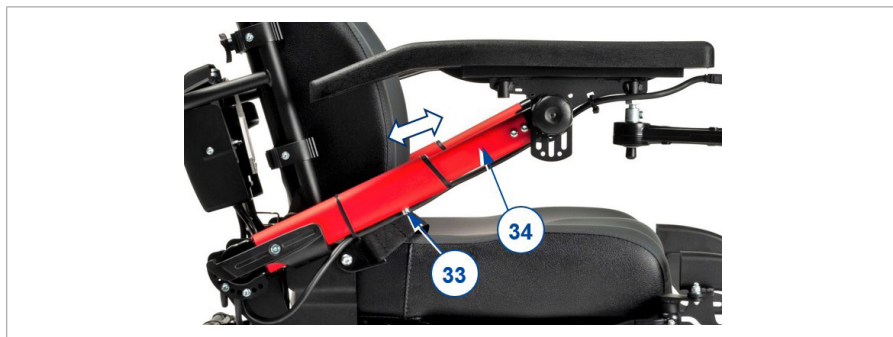

3.2.4. Armrest lowest and highest angular position

⚠ CAUTION

Risk of injuries

- Always use two socket bolts and two adjustment holes to tighten the armrest. Never exceed the maximum angular adjustment (holes).

To change the end position of the turning angle:

1. Hold the armrest and remove the two socket screws (35).
2. Turn the armrest support at the toothed clamp (36).
3. Make sure that the holes of inner and outer clamp part match.
4. Insert the two socket bolts again and tighten.
5. Repeat this for the second armrest.
6. Make sure both armrests have the same end positions.


7. Make sure that both toothed clamps (36) are firmly secured.

3.2.5. Operating console

3.2.5.1. Mounting position left or right

1. Cut the cable ties and unplug the power cable from the operating console.
2. Hold the control with its support (3) while removing M6 socket screw (41).
3. Loosen hex nut (42) and demount shaft (43) from the armrest.
4. Mount shaft (43) in the other armrest and lock it with hex nut (42).
5. Mount the control support (3) with socket screw (41) on the shaft (43).
6. Make sure all connections are well secured.
7. Guide the cable and plug the cable in the operating console.
8. Tie the cable to the back of the wheelchair subframe and on the armrest with cable ties. Make sure that the cable is long enough to make the adjustment of armrest and console possible.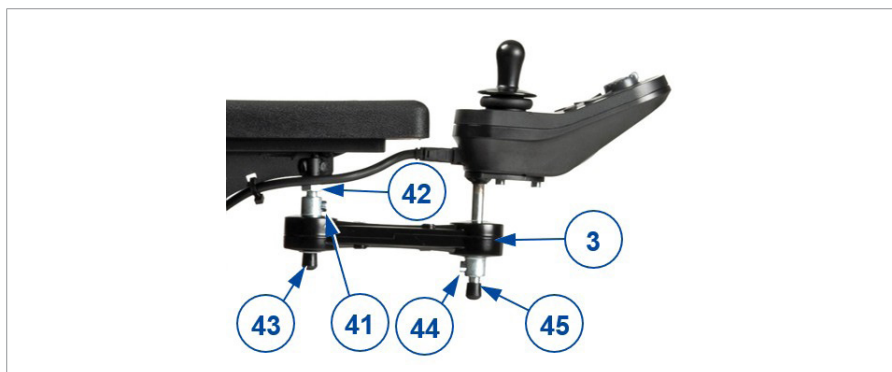

3.2.5.2. Operating console height

The height of the operating console can be adjusted in 2+4 positions, or 6+4 positions depending on the type of shaft:

1. Hold the control while removing M6 socket screw (41).
2. Move the control support (3) up/down on shaft (43) and mount the socket screw (41).
3. If necessary adjust the height of the console on shaft (45) in the same way and lock it with socket screw (44).
4. Make sure all connections are well secured.
5. Guide the cable and plug the cable in the operating console.

3.2.6. Footplate height


WARNING

Risk of injuries or damage

- Make sure that the footplate is properly tightened before using the wheelchair.
- Make sure that the footrests won't touch the ground. Keep a minimum safe distance of 60 mm above the ground.

Adjust the height of the footplate as follows (7 different positions: range 156 mm: steps 26 mm):

1. Loosen and remove the screws (1) and (2).
2. Move the footplate to the desired height.
3. Retighten the screws.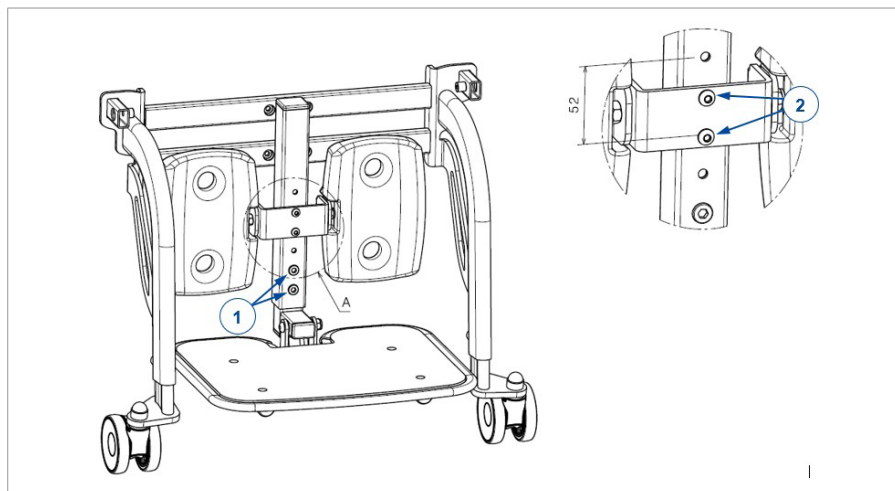

3.2.7. Calf rest height


Adjust the height of the calf rests as follows (3 different positions: range 52 mm, steps 26 mm):

1. Incline the footplate with the operator control.
2. Loosen and remove the screws (2).
3. Adjust the calf rests to a comfortable height.
4. Retighten the screws.

3.2.8. Calf rest width

Adjust the width of the calf rests as follows (2 different positions over a range of 15 mm):


1. Incline the footplate with the operator control.
2. Loosen the screws (1) just below the calf rests.
3. Adjust the calf rests to a comfortable width.
4. Retighten the screws.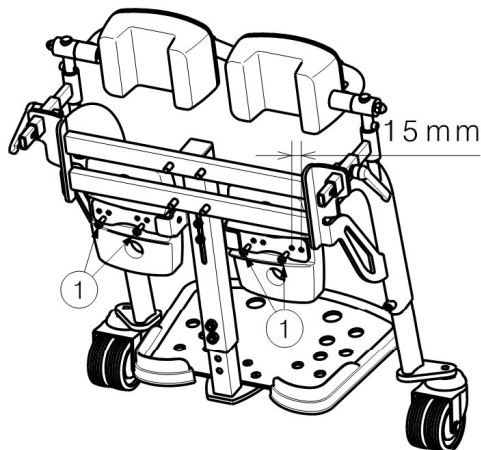

3.2.9. Knee protector height

Adjust the height of the knee protector as follows (range 60 mm: steps 20 mm):

1. Loosen the two screws (1) on both sides.
2. Adjust the knee protector to a comfortable height by placing the two screws in another hole. See that the screws on each side are placed in a hole at the same height.
3. Retighten the screws.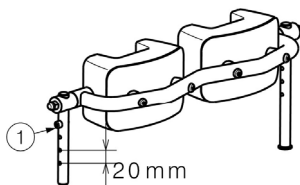

3.2.10.Support wheels


⚠ WARNING

Risk of injuries

- Make sure that the supporting wheels are properly tightened before using the wheelchair.
- Make sure that the supporting wheels rest on the ground and are positioned at the same height.

The supporting wheels are adjustable in 6 different positions (range: 100 mm steps 20 mm):

1. Loosen the hexagon socket screw (1).
2. Move the supporting wheel (2) to the desired height.
3. Retighten the hexagon socket screw (1).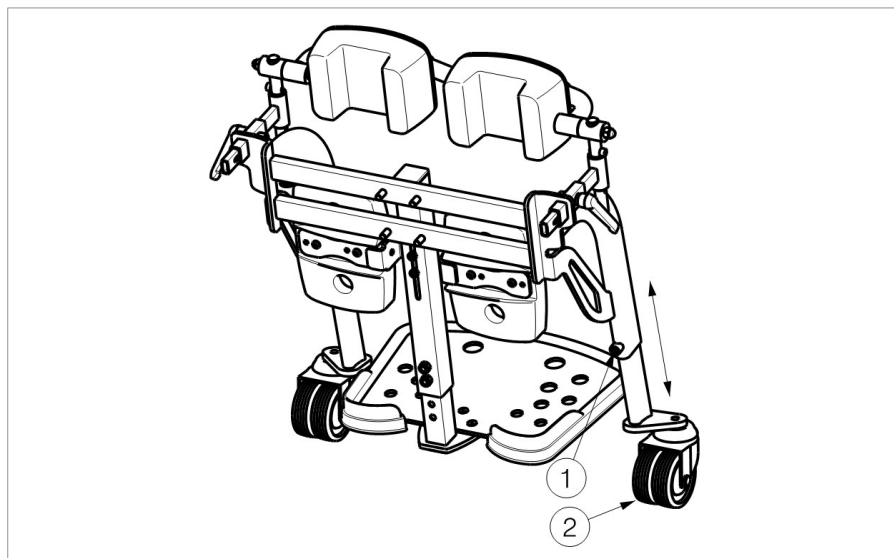

3.3. Changing the battery


CAUTION

Risk of injuries or damage

- The batteries must only be replaced by trained personnel.
- Do not come into contact with the acid from the batteries.
- Replace the batteries in a well-ventilated room.
- Make sure that the ventilation holes of the battery housing are free from any contaminations.
- First read the charging instructions in the user manual before charging the batteries.
- Do not place fingers between the batteries and the battery housing.

i Replace both batteries at the same time and with the same type as the ones used in the wheelchair. If this is not possible, contact the nearest Vermeiren facility.

1. Switch OFF the wheelchair.
2. Remove the seat frame (see user manual).
3. Disconnect all the wires on the batteries.
4. Lift the first battery (1) out the battery housing (2).
5. Slide the second battery (3) forward and take it out the battery housing (2).
6. Take the new batteries.
7. Place the first battery (3) in the battery housing (2) and push it backwards.
8. Place the second battery (1) in the battery housing (2).
9. Connect all the wires to the batteries.
10. Replace the seat frame (see user manual).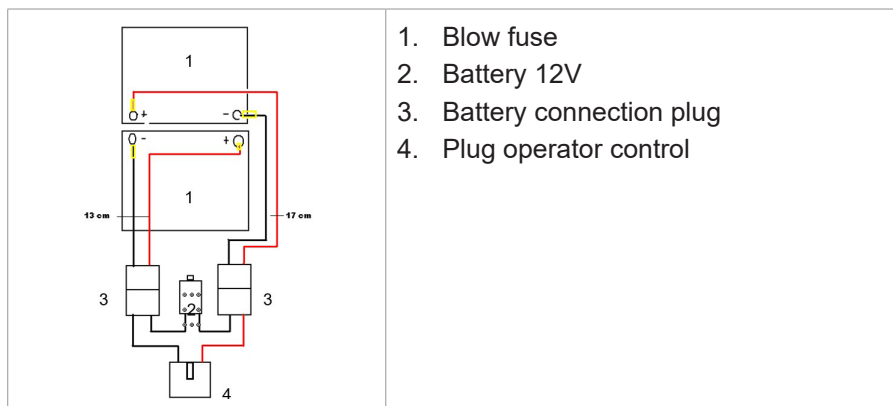

3.4. Changing wheels or tyres

⚠ CAUTION


Risk of injuries or damage

- Make sure that all air is released from the tyre, before removing it.
 - Use a wooden block and jack to lift the wheelchair. Always place the jack under the lower frame, never under any other parts.
 - Make sure that no body parts, the inner tube nor any other objects are pinched between tyre and rim.
 - Handle carefully to prevent damage to the rim.
1. Replace the inner tube, and if necessary also the tyre.
 2. Inflate the tyre using inflating equipment which complies to regulations and with a pressure indication in “bar”.
 3. Make sure that the pressure is correct, see the pressure indication on the tyres.

3.4.1. Driving wheels

i If a tyre needs to be replaced, first check the type of tyre that has been fitted. In case of a tyre with line profile, do not switch left and right tyre.

1. Lift the driving wheel from the ground using a jack.
2. Remove the socket screws (24) and take the wheel from the flange.
3. Open tyre valve and push it gently to release the air.
4. Loosen the 5 screws that connect both rim halves (on the inner rim half).
5. Separate the rim halves and remove the inner tube from the tyre.
6. Inflate the new or repaired inner tube partly.
7. Insert the inner tube into the tyre.
8. Place the two rim halves back on the tyre. Make sure that the tube valve pops out of the opening in the rim.
9. Connect the rim halves with the socket screws. Tighten by hand.
10. Inflate the tyre at the correct pressure, see the indication on the tyre.
11. Remove grease and other contamination from the thread of the 5 flange socket screws. Apply screw adhesive (ex. Loctite) to these screws.
12. Mount the wheel on the flange.
13. Lower the jack.
14. Make sure the socket bolts are firmly secured.

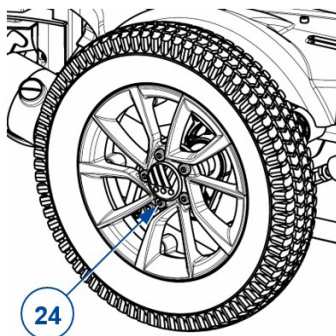3.4.2. Steering wheels

3.4.2.1. Dismantling

1. Loosen the screw on the steering wheel axle (25) and remove it from the steering wheel fork.
2. Let the air out of the steering wheel by lightly pressing the pressure pin on the valve.
3. Loosen the 5 screws that hold the split rim together (26).
4. Now separate the rim sides.

3.4.2.2. Assembly

1. Insert the partly-filled inner tube into the tyre.
2. Connect the two sides of the rim through the tyres and use the 5 connecting screws to screw the rim together (26).
3. Make sure that the valve juts out of the valve opening in the rim.
4. Put the wheel back into the front wheel fork (25) and inflate the wheel.

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All rights reserved, including translation.