User manual

MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

Gemini 2

Contents

Preface	4
1. Your product	6
1.1. Options	6
2. Before use	7
2.1. Intended use and indications	7
2.2. Safety instructions and applicable risks	7
2.3. Symbols on the wheelchair	8
2.4. Transport	9
2.5. Assembly/Disassembly	10
2.6. Placing the reflective stickers (optional)	12
2.7. Storage	13
3. Using your wheelchair	13
3.1. Operating the brakes	13
3.2. Moving to/from the wheelchair	14
3.3. Comfort adjustments	15
3.4. Riding the wheelchair	20
4. Maintenance	21
4.1. Points of maintenance	21
4.2. Maintenance instructions	22
4.3. Troubleshooting	23
4.4. Expected lifespan	24
4.5. Reuse	24
4.6. End of use	24
4.7. Warranty	24
5. Technical specifications	25

Preface

Congratulations! You are now owner of a Vermeiren wheelchair!

This wheelchair is made by qualified and committed personnel. It is designed and produced according to high quality standards, guarded by Vermeiren.

Thank you for your trust in the products of Vermeiren. To support you on the use of this wheelchair and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your wheelchair.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

Important note

To ensure your safety, and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual For user and specialist dealer
	Installation instructions For specialist dealer
	Service manual for wheelchairs For specialist dealer
	EC declaration of conformity

1. Your product

1. Push bar
2. Backrest
3. Safety belt
4. Seat
5. Leg rest
6. Footplate
7. Steering wheels (front wheels)
8. Driving wheels (rear wheels)
9. Brakes
10. Frame
11. Identification plate

1.1. Options

Contact your specialist dealer about options. He will gladly advise you.

2. Before use

2.1. Intended use and indications

- This product is a medical device.
- Indications and contraindications: The wheelchair can be pushed by an attendant or parent. The wheelchair is intended for children with a disability or movement dysfunction because of brain damage. The modular construction allows you to choose the optimal design for your child's specific needs.
- This wheelchair is suitable for indoor and outdoor use.
- This wheelchair is designed and produced solely to transport/transfer one (1) person with a maximum weight as specified in §5.. It is not designed for transportation of goods or objects, nor for any use other than previously described.
- Only use accessories and spare parts approved by Vermeiren.
- Please read all technical details and limits of your wheelchair in chapter 5..
- The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

2.2. Safety instructions and applicable risks

CAUTION

Risk of injuries and/or damage

- Carefully read and follow the instructions in this manual. Otherwise you may get injured or your wheelchair may get damaged.

Keep the following general warnings in mind during use:

- As the attendant, do not operate the wheelchair if you are under influence of alcohol, medicines or other substances that may influence your riding abilities.

- Be aware that some parts of this wheelchair may get very hot or cold due to ambient temperature, solar radiation, heating devices. Be careful when touching. Use protective clothing if the weather is cold.
- Do not modify your product in any way.
- If the packaging of your product has been damaged, (unintentionally) opened, or affected by environmental conditions (moisture, heat,...) upon delivery, check the device integrity of your product. When in doubt, contact your specialist dealer.

Be aware that your wheelchair may interfere with some types of anti-theft systems, depending on the settings used. This may cause the shop's alarm to go off.

Any serious incident [MDR (EU) 2017/745 §2 (65)] that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

2.3. Symbols on the wheelchair

	Type designation
	Catalogue number
	Serial number
	Medical device
	Manufacturer
	Date of manufacture
	Declaration of conformity

	Attention: important information
	It is advised to read the manual
	Risk of entrapment
	Maximum weight of the user in kg
	Maximum safe slope in ° (degrees).
	Not intended to be used as a seat in a motor vehicle

2.4. Transport

2.4.1. Moving out of the way

Move the wheelchair by using its wheels and rolling it to the destination.

If this is not possible, lift the wheelchair by the solid parts of the frame. Do not grasp the footrests, armrests or wheels.

2.4.2. Transport by vehicle, as luggage

WARNING

Risk of injuries

- Do NOT use your wheelchair as a seat in a vehicle, see the following symbol.

- Make sure that the wheelchair is attached properly to avoid injury from the passengers during collision or sudden braking.
- NEVER use the same seatbelt to secure the wheelchair and passenger.

1. Fold the wheelchair to its most compact size according to the instructions in **§2.5.1.**
2. Place the wheelchair in the luggage compartment.
3. If the luggage and passenger compartments are NOT separated, attach the frame of the wheelchair securely to the vehicle. You can use the available safety belts in the vehicle.

2.5. Assembly/Disassembly


⚠ CAUTION

Risk of injuries

- Keep fingers away from moving parts of the wheelchair.

2.5.1. Folding the wheelchair

1. Use the central brake to stop the wheelchair from moving, see **§3.1.**
2. Pull the red lever under the front of the seat (1). The lever can be reached through the opening in the leg rest upholstery, or by folding up the leg rest upholstery.
3. Lift the front of the seat up and pull the seat as a whole forward. You can now remove the seat from the frame.
4. Pull parts (2) upwards on both sides until the upper frame is unlocked.
5. Push the push bar (3) to the front to fold the frame.
6. Let the frame rest on the rear wheels and frame tube. Now the frame can be folded completely.
7. Turn the push bar downwards, see **§3.3.9.**
8. If necessary, take off accessories from the seat and fold the backrest down (4).

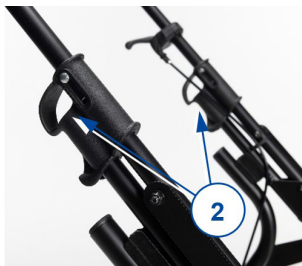2.5.2. Unfolding the wheelchair

1. Grasp the push bar (3) and move it upwards to unfold the frame.
2. Turn the frame so the front and rear wheels can rest on the ground.
3. Move the push bar backwards to unfold the frame completely.
4. Check that parts (2) click over the lower frame tube and the frame is firmly secured.
5. Turn the push bar in the desired position, see §3.3.9..
6. Take the seat and unfold the backrest if necessary.
7. Install the rear of the seat first *: the two metal hooks must be placed over the lower frame rod that is furthest away from the front of the seat.
8. When the rear of the seat has been placed correctly, the front can be easily locked by pushing down. A click will be heard.
9. Make sure that the seat has been attached properly to the lower frame.
10. Adjust the seat for the comfort of your child and install any accessories that had been removed for transport (see §3.3.).

* The seat of Gemini 2 can be installed with your child facing forwards or facing towards you as you push. The same instructions apply for both options.

It is also possible not to remove the seat and fold the Gemini 2. Be aware that you should first remove all the accessories and supports before completely folding the backrest to the front.

2.6. Placing the reflective stickers (optional)

	White
	Red

2.7. Storage

CAUTION

Risk of damage

- Make sure that your wheelchair is stored in a dry environment to prevent mould from growing and the upholstery from being damaged, see also chapter 5..

3. Using your wheelchair

WARNING

Risk of injuries

- First read previous chapters and inform yourself about the intended use. Do NOT use your wheelchair unless you have read and fully understood all instructions.
- In case of doubts or questions, do not hesitate to contact your local specialist dealer, your care provider or technical adviser to help you with this.

3.1. Operating the brakes


CAUTION

Risk of injuries

- The brakes are not used to slow down the wheelchair during movements. Use the brake only to prevent the wheelchair from unintended movements.
- Good operation of the brakes is influenced by wear and contamination of the tires (water, oil, mud, ...). Check the condition of the tires before each use.
- The brakes are adjustable and can wear. Check the operation of the brakes before each use.
- Make sure the wheelchair is on a flat horizontal surface before releasing the brakes. Never release both brakes simultaneously.

Gemini 2 brakes are operated by one central brake lever:

- Push the central brake in direction (1) to activate the brake.
- By pushing the central brake in direction (2), the brake will be released.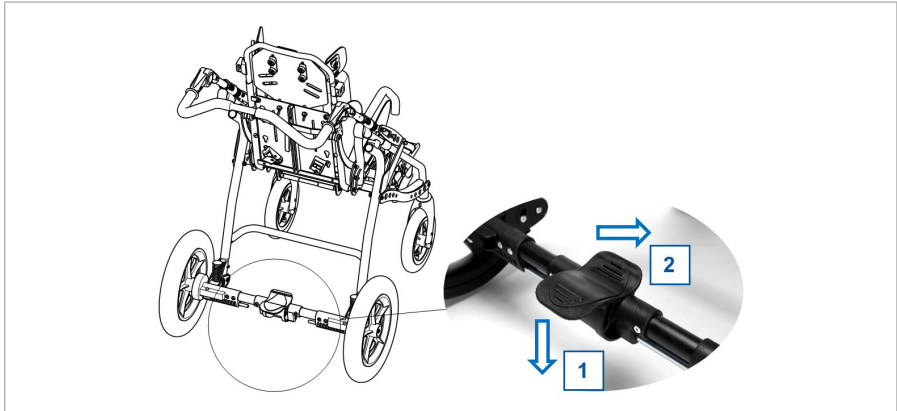

3.2. Moving to/from the wheelchair

⚠ CAUTION

Risk of injuries or damage

- Do not stand on the footplates.

3.2.1. Transfer

1. Position the wheelchair as close as possible to the chair, couch or bed to/from which you wish to transfer.
2. Check that the central brake is activated.
3. Transfer the child to/from the wheelchair.
4. Secure the safety belt.

3.2.2. Sitting in the wheelchair

Some recommendations for a comfortable use of the wheelchair:

- Sit down on the seat with your lower back against the backrest.
- Make sure that your upper legs are horizontal and that your feet are in a comfortable position. Adjust if necessary.


3.3. Comfort adjustments

⚠ CAUTION

Risk of injuries or damage

- The following comfort adjustments can be done by the attendant or caretaker. All other adjustments are done by your specialist dealer according to the installation instructions, see preface.

3.3.1. Seat inclination

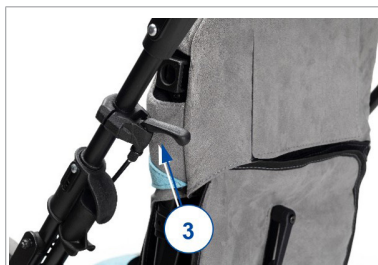1. Press the lever (3) on the side of the push bar.
2. Press down on/lift the front of the seat until the desired inclination.
3. Release the lever.

3.3.2. Backrest inclination

Seat width 32

1. Loosen the lever (4) on the rear side of the backrest.
2. Push or pull the backrest tubes in the desired position (from folded position for easy transport, to lying down position: 0° - 45°). The backrest automatically stops when it reaches the maximum inclination.
3. Retighten the lever (4).
4. Check that the backrest is well fixated.


Seat width 40

1. Pull the pin (5).
2. Pull or push the backrest until the desired angle (6 different positions in steps of 8°). The backrest automatically stops when it reaches the maximum inclination.
3. Check that the backrest is well fixated.
4. Release the pin (5).

There is an extra hole that prevents that the backrest will be lift back in the vertical position when the backrest is folded to the seat (for transportation purposes).

3.3.3. Backrest height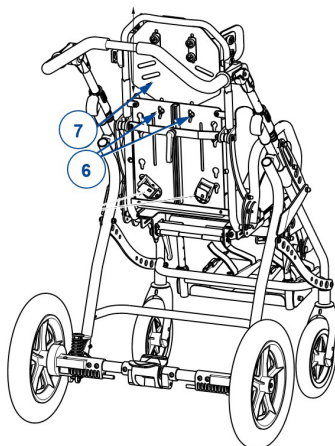

Remove the upholstery to alter the backrest height.

1. Loosen the two wing nuts (6) on the back of the backrest.
2. Move the upper part (headrest) (7) of the backrest upwards or downwards until the desired backrest height.
3. Retighten the two screws and wing nuts (6) firmly by hand.

3.3.4. Leg rest inclination

1. Loosen the levers (8) on both sides.
2. Adjust the leg rest to the desired position (stepless; range 83° for seat width 32, range 85° for seat width 40).
3. Retighten both levers (8).
4. Make sure that the leg rest is firmly fixated.

3.3.5. Leg rest length (only for seat width 32)


The leg rest can be set into 4 different lengths (steps of 30 mm):

1. Retract both blocking pins (9).
2. Slide the footplate up/down to the desired position.
3. Block the undercarriage again with the blocking pins (9).
4. Make sure the footplate is firmly fixated.

3.3.6. Foot supports (optional)


The angle of the foot supports (10) can be adjusted in several positions (Seat width 32: 5 positions, steps of 10° – Seat width 40: 4 positions, steps of 8,5°)

1. Loosen the wing nuts (11) on the bottom of the footplate.
2. Turn the foot supports (10) in the desired position.
3. Retighten the wing nuts (11).
4. Make sure that the foot supports are firmly fixated.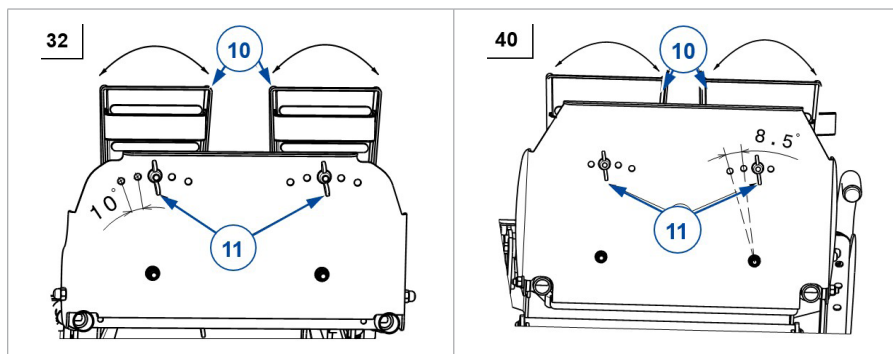

3.3.7. Side supports (optional)

3.3.7.1. Head pelots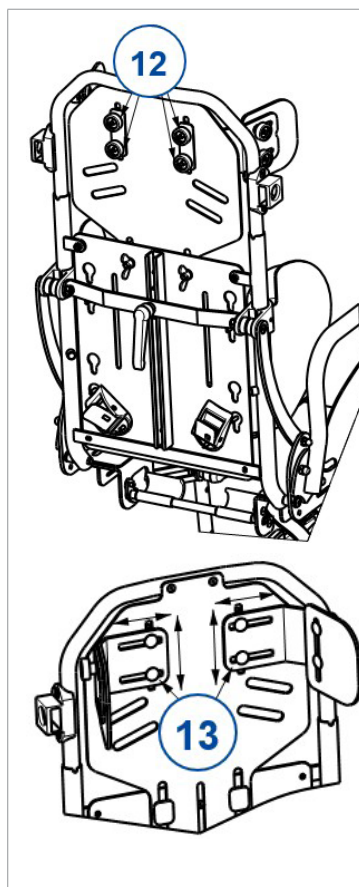


Adjusting width

1. Loosen the two knurled knobs (12) on the rear side of the backrest.
2. Move the head pelot (13) inwards or outwards until the desired position (stepless, range 25 mm each side).
3. Retighten the two knurled knobs (12).
4. Check that the pelot (13) is well fixated.
5. Repeat these instructions for the other pelot (13).

Adjusting height

1. Loosen the two knurled knobs (12) on the rear side of the backrest.
2. Move the head pelot (13) up or down until the desired height (stepless, range 30 mm).
3. Retighten the two knurled knobs (12).
4. Check that the pelot (13) is well fixated.
5. Repeat these instructions for the other pelot (13).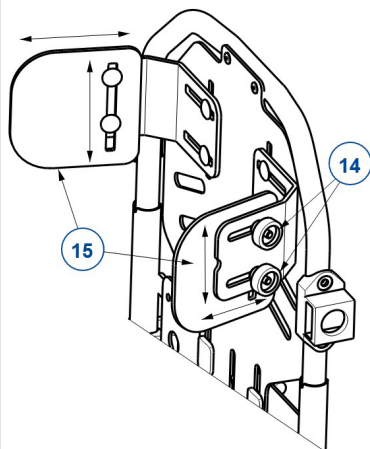


Adjusting height

1. Loosen the two knurled knobs (14) on the side of the pelot.
2. Move the head pelot (15) up or down until the desired height (stepless, range 25 mm).
3. Retighten the knurled knobs (14).
4. Check that the pelot (15) is well fixated.
5. Repeat these instructions for the other pelot.

Adjusting depth

1. Loosen the two knurled knobs (14) on the side of the pelot.
2. Move the head pelot (15) to the front or back until the desired depth (stepless, range 22,5 mm).
3. Retighten the knurled knobs (14).
4. Check that the pelot (15) is well fixated.
5. Repeat these instructions for the other pelot.


3.3.7.2. Backrest pelots height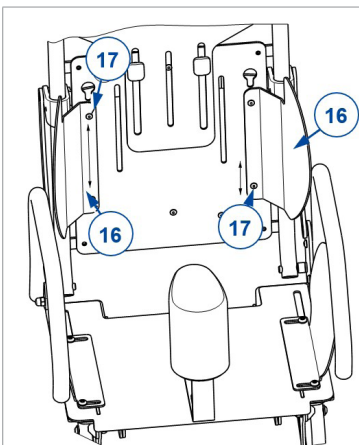

The backrest pelots (16) can be placed in 2 positions (40 mm distance).

1. Slightly lift the pelot upwards to detach it from the backrest.
2. Place it in the desired position by placing the screws (17) through the holes in the backrest.
3. Repeat for the other pelot.

3.3.8. Abduction wedge (optional)

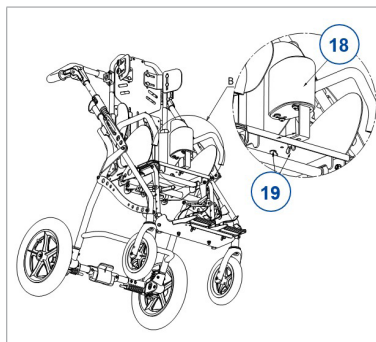The abduction wedge (18) is mounted with two wing nuts (19). It can easily be removed by loosening/tightening the wing nuts (19).

The abduction wedge can be moved forward/backward over a distance of 115 mm.

3.3.9. Push bar position

Simultaneously push both buttons (20) at the clamp of the pushbar and move to the desired position. Releasing the buttons (20) on the clamp will automatically lock the pushbar in the desired position.

3.4. Riding the wheelchair

CAUTION

Risk of injuries or damage

- Do NOT drive on slopes, obstacles, steps or kerbs larger than specified in § 5..
- Pay attention if the road has holes or gaps that may cause entrapment of the wheels.
- Avoid stones and other objects that may block the wheels.
- Always keep the swing range of the footplate in mind to prevent bystanders from being injured or objects from being damaged.
- Before every usage, make sure that:
 - all adjustments are firmly secured, see §3.3..
 - the parking brakes are working properly.
 - the tires are in good condition, see §4.2.1..

3.4.1. Pushing the wheelchair

The wheelchair can only be operated by an attendant:

1. Take hold of the push bar.
2. Release the brakes.
3. Push the wheelchair.
4. For seat width 40: Use the tipping aid on the lower frame to overcome obstacles.

4. Maintenance

- i** Regular care ensures that your wheelchair is preserved in a perfectly functional condition. For the maintenance manual, refer to the Vermeiren website: www.vermeiren.com.

4.1. Points of maintenance

CAUTION

Risk of injuries and damage

- Repairs and replacements may only be undertaken by trained persons and only genuine replacement parts of Vermeiren should be used.
- i** The last page of this manual contains a registration form for the specialist dealer to record each service.
The service frequency depends on the frequency and intensity of use.
Contact your dealer to agree to a common timetable for inspection/maintenance/repair.

4.1.1. Before each use

Inspect the following points:

- All parts: Present and undamaged or unworn.
- All parts: Clean, see § 4.2.2..
- Wheels, backrest, seat, calf rests, arm rests, footplates: Well secured.
- Condition of wheels/tyres, see § 4.2.1..
- Condition of frame parts: No deformation, instability, weakness or loose connections.
- Seat, backrest, arm pads, calf pads and head rest (if applied): No excessive wear (like dented spots, damage or tears).
- Brakes: Undamaged and functioning.

Contact your specialist dealer for possible repairs or part replacements.

4.1.2. Yearly or more often

Have your wheelchair inspected and serviced by your specialist dealer, at least once a year, or more often. The minimum maintenance frequency depends on use and should therefore be commonly agreed upon with your specialist dealer.

4.1.3. If stored

Make sure that your wheelchair is stored in a dry environment to prevent mould from growing and the upholstery from being damaged, see also chapter 5..

4.2. Maintenance instructions

4.2.1. Wheels and tyres

- ❗ Proper working of the brakes depends on the state of the tyres, and can change due to wear and contamination (water, oil, mud, ...).

Keep your wheels free of wires, hair, sand and fibres.

Check the profile of the tyres. If the tread depth is less than 1 mm, the tyres need to be replaced. Contact your specialist dealer for this matter.

Inflate each pneumatic tyre to the correct pressure (see pressure indication on the tyres).

4.2.2. Cleaning

CAUTION

Risk of damage by moisture

- Never use a hose or high-pressure cleaner to clean the wheelchair.

Wipe all rigid parts with a damp cloth (not drenched). If necessary, use a mild soap, suitable for varnishes and synthetics.

The upholstery can be cleaned with lukewarm water and a mild soap. Do not use abrasive cleaning agents to clean.

4.2.3. Disinfection

CAUTION

Risk of damage

- Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

4.3. Troubleshooting

Even if you use your wheelchair properly, a technical problem may occur. In this case, contact your local specialist dealer.

WARNING

Risk of injuries and damage

- NEVER attempt to repair your wheelchair yourself.

The following symptoms may indicate a serious problem. Therefore always contact your specialist dealer if you spot any of the following deviations:

- Strange sound;
- Uneven tread wear on one of the tyres;
- Jerky movements;
- Wheelchair deflects to one side;
- Damaged or broken wheel assemblies.

4.4. Expected lifespan

The wheelchair is designed to have an average lifespan of 5 years. Depending on the frequency of use, driving circumstances and maintenance, the lifespan of your wheelchair will increase or decrease.

4.5. Reuse

Before each reuse, have the wheelchair disinfected, inspected and serviced according to the instructions in § 4.1. and § 4.2..

4.6. End of use

At end of life, you need to dispose your wheelchair according to the local environmental legislation. The best way to do so, is to disassemble the wheelchair to facilitate the transport of recyclable parts.

4.7. Warranty

The warranty on this product is subject to the general terms and conditions of each country.

5. Technical specifications

The technical details below are only valid for this wheelchair, at standard settings and optimal ambient conditions. Take these details into account during use. The values are no longer applicable if your chair has been modified, damaged, or is severely worn.

Brand	Vermeiren	
Product group	Manual wheelchair	
Type	Gemini 2	
Description	Dimensions	
	Seat width 32	Seat width 40
Maximum user weight	35 kg	50 kg
Overall length	950 mm	1060 mm
Overall width	620 mm	690 mm
Overall height	1200 mm	1275 mm
Total mass	18 kg	27 kg
Mass of removable parts		
Seat	7,2 kg	13,55 kg
Lower frame	10,8 kg	13,45 kg
Static stability downhill	10°	
Static stability uphill	10°	
Static stability sideways	10°	
Seat plane angle	-25° - +30°	
Effective seat depth	300 - 360 mm	360 - 420 mm
Effective seat width	320 mm	400 mm
Seat surface height at front edge	510 mm	
Backrest angle	0° - 45°	0° - 40°
Backrest height	520 - 640 mm	650 - 770 mm
Footplate to seat distance	250 - 340 mm	315 - 400 mm
Leg to seat surface angle	2,5° - 85,5°	3° - 88°
Distance between arm pad and seat	200 mm	
Front location of arm pad structure	235 mm	310 mm
Horizontal location of axle (deflection)	230 mm	150 mm
We reserve the right to introduce technical changes. Measurement tolerance +- 15 mm / 1,5 kg / 1,5°		

Brand	Vermeiren	
Product group	Manual wheelchair	
Type	Gemini 2	
Description	Dimensions	
	Seat width 32	Seat width 40
Minimum turning radius	Footplate folded up: 1200 mm Footplate folded down: 1480 mm	Footplate folded up: 1620 mm Footplate folded down: 1850 mm
Diameter of rear wheels	295 x 50 mm	
Tyre pressure rear wheels	Max. 2,5 bar	
Diameter of front wheels	200 x 40 mm	
Tyre pressure front wheels	Max. 2,5 bar	
Use temperature	+5°C - +41°C	
Storage and use humidity	30% - 70%	
We reserve the right to introduce technical changes. Measurement tolerance +- 15 mm / 1,5 kg / 1,5°		

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**Service registration form**

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:

VERMEIREN GROUP

Vermeirenplein 1/15

2920 Kalmthout

Belgium

www.vermeiren.com

Version: C, 2025-06

Basic UDI-DI: 5415174122103Gemini29D

All rights reserved, including translation.

VERMEIREN
we care **for** you