User manual

MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

385

Contents

Preface	4
1. Your product	5
2. Before use	5
2.1. Upon delivery	5
2.2. Intended use	6
2.3. General safety instructions	6
2.4. Symbols on the product	7
2.5. Assembly	7
2.6. Shipping and storage	8
3. Using your table	8
3.1. Operating the brakes (only for cupboard with wheels)	8
3.2. Using the expandable tray (if available)	9
4. Maintenance	10
4.1. Cleaning	10
4.2. Disinfection	10
4.3. Inspection / Service / Repairs	10
4.4. Reuse	11
4.5. Disposal at end of life	11
5. Technical specifications	11

Preface

Thank you for your trust in the products of Vermeiren. To support you on the use of this product and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your product.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

Important note

To ensure your safety, and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures of the product are used to clarify the instructions. Details of the depicted product may deviate from your product.

Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the following information. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual For user and specialist dealer
	EC declaration of conformity

1. Your product

1. Expandable tray (if available)
2. Door
3. Drawer
4. Legs or wheels
5. Identification plate

2. Before use

2.1. Upon delivery

Unpack your product and verify if the delivery is complete. The following items should be included:

- Cupboard (including 4 legs or 2 castors with brake and 2 without brake; with or without expandable tray)
- User manual

2.2. Intended use

- This product is a medical device.
- This cabinet, with or without table, is intended to be used near a chair or bed, allowing the user to remain in their wheelchair or bed.
- It is not intended to support or transport goods or persons.
- The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

2.3. General safety instructions

CAUTION

Risk of injuries and/or damage

- Carefully read and follow the instructions in this manual. Otherwise you may get injured or your wheelchair may get damaged.

Keep the following general warnings in mind during use:

- Do not lean against or on the expandable tray, since the cupboard could tip over.
- Do not lift the cupboard by means of the tray.
- Do not place any objects larger than the cupboard on them.
- Do not exceed the bearing capacity of the expandable tray (10 kg).
- Do not convey persons or objects.
- The cupboard must only be set up indoors on level surfaces. When in use, check that all 4 castors / legs press evenly on the floor.
- Repairs and replacements may only be undertaken by trained persons and only genuine replacement parts of Vermeiren should be used.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing / care.

Any serious incident [MDR (EU) 2017/745 §2 (65)] that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

2.4. Symbols on the product

	Type designation
	Catalogue number
	Serial number
	Medical device
	Manufacturer
	Date of manufacture
	Declaration of conformity
	Attention: important information
	It is advised to read the manual
	Risk of entrapment

2.5. Assembly

The cupboard is delivered fully assembled and the handgrips of the drawer and door are mounted on the inside. You only need to mount the handgrips to the outside with a screwdriver.

2.6. Shipping and storage

The shipping and storage of the table should happen according to the technical specifications in §5.. Make sure that the table is stored dry.

Provide sufficient covering or packaging to protect the table from rust and foreign bodies (e.g. salt water, sea air, sand, dust).


3. Using your table

3.1. Operating the brakes (only for cupboard with wheels)

CAUTION

Risk of injuries and/or damage

- The brakes can wear. Check the working of the brakes before each use.

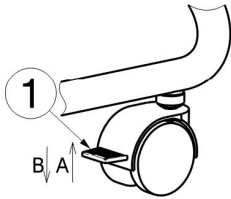3.1.1. Releasing the brakes (A)

With your foot, press the brake plate of the wheel downwards until it locks into place. The wheels cannot roll.

3.1.2. Activating the brakes (B)

With your foot, press the brake plate of the wheel in the direction of the frame. The wheels have now been released and can both roll and swivel.

To put the cupboard in a parking position, the two wheels must be locked in place.

3.2. Using the expandable tray (if available)

CAUTION

Risk of injuries and/or damage

- The stability of the product is affected when the expandable tray is fold out (danger of tipping over).

3.2.1. Sliding tray

3.2.1.1. Slide out

Grasp the expandable tray by the handgrip. Pull the expandable tray backwards and afterwards pull it upwards.

3.2.1.2. Slide in

Lift the tray a little bit upwards and slide the tray in by pushing it down in the cupboard rails.

3.2.2. Folding tray

CAUTION

Risk of injuries

- Keep fingers, buckles, jewellery and clothes away from the folding mechanism.

3.2.2.1. Unfolding

Grasp the tray at the bottom and pull it upwards until it clicks into place.

3.2.2.2. Folding

Press the folding mechanism levers on the bottom of either side of the tray. Carefully lower the tray towards the cabinet.

4. Maintenance

4.1. Cleaning

Use a soft, damp cloth and lukewarm water with a mild soap to clean the table. Avoid soaking.
Do not use abrasive cleaning agents to clean.

4.2. Disinfection

CAUTION

Risk of damage

- Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

Use a soft, damp cloth and lukewarm water with a mild household disinfectant to clean the table.

4.3. Inspection / Service / Repairs

The expected service life of your table is 5 years. The lifespan is influenced by use, storage, maintenance, service and cleaning.

Before you resume use of the table, it should be serviced according to the following criteria:

- Completeness.
- Castors (stability, tracking, fixing screws).
- Fastenings: well secured (Condition, thread, holding security).
- Cupboard (deformation, stability).
- Tray (closed surfaces, splinters, tears).

Repairs and replacements may only be undertaken by trained persons and only genuine replacement parts of Vermeiren should be used.

4.4. Reuse

Before each reuse, have the table disinfected, inspected and serviced according to the instructions in §4..

4.5. Disposal at end of life

At end of life, you need to dispose of your table according to the local environmental legislation. The best way to do so, is to disassemble the table to facilitate the transport of recyclable parts.

5. Technical specifications

Brand	Vermeiren
Product group	Tables
Type	385
Description	Dimensions
Overall length	590 mm
Overall width	460 mm
Overall height	910 mm
Height tray (if available)	885 mm
Length tray unfolded	1250 mm
Total mass	35 kg
Dimensions drawer (inside)	95 mm x 425 mm x 375 mm (HxWxD)
Dimensions door	645 mm x 460 mm (HxW)
Tray area (if available)	675 mm x 360 mm (L x W)
Max. load expandable tray (if available)	10 kg
Ground clearance	70 mm
Wheel diameter	50 mm
Storage and use temperature	+5°C - +41°C
Storage and use humidity	20% - 75%
We reserve the right to introduce technical changes. Measurement tolerance +- 15 mm / 1,5 kg / 1,5°	

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This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp):	By (stamp):	By (stamp):
Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:
By (stamp):	By (stamp):	By (stamp):
Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:
By (stamp):	By (stamp):	By (stamp):
Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:
By (stamp):	By (stamp):	By (stamp):
Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:
By (stamp):	By (stamp):	By (stamp):
Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:	Kind of work: I / S / R / D Date:

VERMEIREN GROUP
Vermeirenplein 1/15
2920 Kalmthout
Belgium
www.vermeiren.com

Version: C, 2024-10

Basic UDI-DI: 54151743856H

All rights reserved, including translation.

VERMEIREN
we care **for you**