VERMEIREN

Flora 1 - 2 - 3

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K POUŽITÍ

EN

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Verze: A, 2025-01

All rights reserved, including translation.

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EN

Contents

1.	Technical details	2
2.	Components	2
3.	Explanation of symbols	2
4.	Check after delivery	2
5.	Intended use / environment	2
6.	Height adjustment walking stick	3
7.	Walking	3
8.	Walking on stairs	3
9.	Safety instructions	3
10.	Maintenance	3

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Walking stick Flora

1. Technical details

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

Make	Vermeiren
Address	Vermeirenplein 1/15, B-2920 Kalmthout
Type	Walking stick
Model	Flora
Width	120 mm
Total height	840 mm – 940 mm
Depth	29 mm
Handgrip length	120 mm
Weight	0,30 kg
Maximum user weight*	max. 100 kg
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg.	

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the walking stick, read the instruction manual carefully: it will familiarise you with this product.

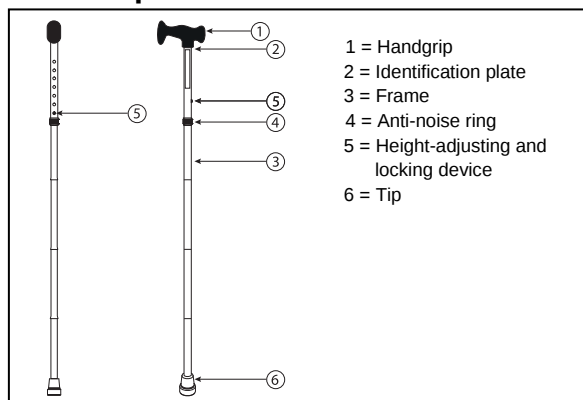
Refer to the Vermeiren website for the instructions for care and safety of the walking stick. Please remember that observance of our hints would keep your walking stick in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

Visually impaired people can contact the dealer for the instructions for use.

2. Components

3. Explanation of symbols

Maximum mass

CE conformity

4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- 1 walking stick
- Tip
- Height adjustment system
- Manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- Contact the transporter
- Have a list made of any problems
- Contact your supplier

5. Intended use / environment

⚠ WARNING: Risk of falling - Do not use the walking stick if you have weak arm, wrist joint function or poor equilibrium.

A walking stick give the user support during walking and makes it easier to walk. These support can be useful for balance problems or pain when walking.

The walking stick can be used indoor and outdoor.

Be extra careful that you do not slip when using on wet surfaces.

However, it cannot be used by persons with a visible weakened hand-/arm function, wrist joints.

The walking stick is not suited to use in high mountains, or on rough underground (big stones, ...).

6. Height adjustment walking stick

⚠ CAUTION: Risk of falling -
Check that the spring pins ② are good
clicked back.

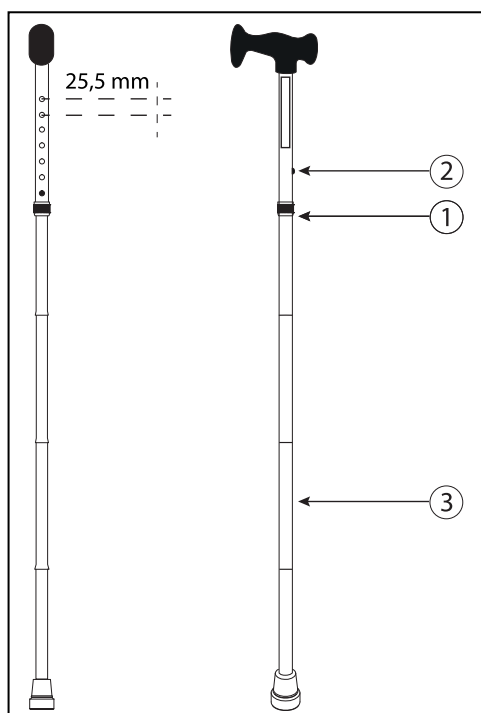
Before using the walking stick you must adjust these in the correct height.

Adjust the height of the walking stick according following instructions:

1. Unscrew the anti-noise ring ①.
2. Press on the spring pin ②.
3. Pull / push the lower tube ③ until the desired height (5 different positions: increments of 25,5 mm).

Ensure that the height of your wrist is at the same height of the handgrip when you leave your arms loose next your body. Ensure that your body is in an upright position.

4. Check that the spring pin ② is good clicked back.
5. Tighten the anti-noise ring ① again firmly.

7. Walking

⚠ CAUTION: Risk of falling - Do not
place the walking stick too far to the
front.

Follow the advice of your doctor or physiotherapist how you must walk with the walking stick. When you have no advice, you can use the instructions below:

- Always walk upright with the walking stick right next to your body.
- Put the walking stick while walking never further than your front leg.

- Be careful with wet/slippy ground.
- Do not place the walking stick in a curve while walking otherwise you can slip out.
- Place the walking stick on the side of your unaffected leg or the side were you need support. Place your affected leg forward simultaneously with the walking stick and then your unaffected leg.

8. Walking on stairs

When there is a handrail available, you can use these as support and on the other side you can use the walking stick. Ensure that the handrail is on the healthy side of your body, if possible.

Lean on the handrail and place the walking stick on the first step. Place your unaffected leg on the first step and put your affected leg beside them.

9. Safety instructions

- Let someone assist you if this is the first time you are using the walking stick.
- Note that you use your walking stick on a good surface. (Not with too many gravel).
- Be extra careful on a wet / slippery ground.
- Before every usage, check if the walking stick is adjusted in the correct height.
- Check that the spring pins for the height adjustment are good clicked back and the anti noise-ring is firmly secured.
- Do not place the walking stick in a curve or too far from your body to avoid risk of falling.
- Do not use the walking stick as a toy.
- Do not hold the handgrip with wet hands. If you do so, you can lose your grip and lose your balance.
- Do not take to big steps with the walking stick.
- **The max. user weight of the product is 100 kg.**
- Risk of burns – Be careful when using in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

10. Maintenance

For the maintenance manual of the walking sticks refer to the Vermeiren website: www.vermeiren.com.

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