# Installation manual

MANUEL D'INSTALLATION  
INSTALLATIEHANDLEIDING  
INSTALLATIONSANLEITUNG  
MANUALE DI INSTALLAZIONE  
MANUAL DE INSTALACIÓN  
INSTRUKCJA INSTALACJI  
PŘÍRUČKU PRO INSTALACI

## Eon

CE

|                                    |          |
|------------------------------------|----------|
| <b>Preface</b>                     | <b>4</b> |
| <b>1. This product</b>             | <b>5</b> |
| <b>2. Scope of delivery</b>        | <b>6</b> |
| <b>3. Assembly and adjustments</b> | <b>6</b> |
| 3.1. Tools                         | 7        |
| 3.2. Seat height                   | 7        |
| 3.3. Changing the battery          | 8        |
| 3.4. Changing wheels or tyres      | 9        |

# Preface

## Important note

To support you on the installation and repairs of this scooter, we provide you with this installation manual. Please read it carefully. If you still have questions after reading this manual, do not hesitate to contact Vermeiren.

Pictures of the product are used to clarify the instructions. Details of the depicted product may deviate from your product.

## Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

|   |  |
|---|--|
|    | User manual<br>For user and specialist dealer      |
|    | Installation instructions<br>For specialist dealer |
|  | Drawings of (spare) parts<br>For specialist dealer |
|  | EC declaration of conformity                       |

# 1. This product

1. Backrest
2. Armrest / Arm pad
3. Seat
4. Rear wheels
5. Footplate
6. Front wheels
7. Front light
8. Indicator lights
9. Speed lever
10. Handgrips
11. Free-wheel lever
12. Rear light
13. Anti-tipping
14. Identification plate

## 2. Scope of delivery

The following components are part of the delivery:

- Frame with steering unit, rear and front wheels, footplate
- Seat with backrest
- Options
- Manual
- Batteries

Before use, check if everything is included and that no products are damaged.

Please note that the basic configuration may differ in different European countries. Contact your specialist dealer.

## 3. Assembly and adjustments

### CAUTION

#### **Risk of injuries or damage**

- Only use the settings described in this manual.
- Only use parts described in this manual.
- Only use genuine spare or replacement parts approved by Vermeiren.
- Prevent bystanders from accessing the working area.
- Keep the (dis)assembling area clean and tidy, to prevent falling.
- Collect all small parts in a box or bag.
- While (dis)assembling and operating the scooter, make sure no objects or body parts get caught between moving parts. In particular, pay attention to the position of the cables.
- Turn the scooter off before you perform any mechanical adjustments and make sure that the free-wheel lever is in braking position.
- Make sure all screws are firmly secured before driving.
- Software changes are only allowed if executed by skilled personnel, taking into account the user and environment (stability, acceleration, indoor/outdoor, etc.).

- i** Read the user manual of this scooter. Be aware of the technical details and limits of intended use. For more information, contact Vermeiren.

### 3.1. Tools

To (dis)assemble and adjust the scooter, the following tools are needed:

- Wrench no. 13
- Allen key no. 5

### 3.2. Seat height

1. Remove the seat.
2. For easier access, it is also possible to remove the plastic frame cover.
3. Remove the locking pin (2) and loosen the bolt (4).
4. Raise/Lower the seat support to the desired height.
5. Replace the locking pin and tighten the bolt.
6. Replace the seat.

### 3.3. Changing the battery

#### CAUTION

##### **Risk of injuries or damage**

- The batteries must only be replaced by trained personnel.
  - Do not come into contact with the acid from the batteries.
  - Replace the batteries in a well-ventilated room.
  - Make sure that the ventilation holes of the battery housing are free from any contaminations.
  - First read the charging instructions in the user manual before charging the batteries.
  - Do not place fingers between the batteries and the battery housing.
-  Replace both batteries at the same time and with the same type as the ones used in the scooter. If this is not possible, contact the nearest Vermeiren facility.

The wiring and charger are well-situated in the scooter. Do not attempt to relocate the wiring by yourself. The improper installation of the wiring may result in pinched wires between the battery box, which cause failure to the electronic system of your scooter.

Make sure that the battery cables are connected to the right battery.

1. Switch off the scooter.
2. Remove the seat, see user manual.
3. Remove the screws of the rear cover, near the footplate.
4. Lift off the rear plastic cover to reach the batteries.
5. Loosen the screw with lever, and remove the bracket used to keep the batteries in place.
6. Remove the pole connectors.
7. Lift the batteries out of the battery housing.
8. Replace the batteries and reconnect the poles.
9. Make sure that the cables are placed properly.
10. Replace the bracket and tighten the screw with lever to tighten the batteries into place. The batteries should not move, even while driving.

11. Mount the rear plastic cover and tighten the screws.
12. Replace the seat.

### 3.4. Changing wheels or tyres

#### CAUTION

##### **Risk of injuries or damage**


- Make sure that all air is released from the tyre, before removing it.
- Make sure that no body parts, the inner tube nor any other objects are pinched between tyre and rim.
- Handle carefully to prevent damage to the rim.

Replace the inner tube, and if necessary also the tyre.

Inflate the tyre using inflating equipment which complies to regulations and with a pressure indication in “bar”.

Make sure that the pressure is correct, see the pressure indication on the tyres.

1. Lift the driving wheel from the ground using a jack.
2. Click off the hubcap from the wheel using a screwdriver.
3. Remove the bolt (23) to take off the wheel.
4. Remove the socket screws (24) and take the wheel from the flange.
5. Open tyre valve and push it gently to release the air.
6. Separate the rim halves and remove the inner tube from the tyre.
7. Inflate the new or repaired inner tube partly.
8. Insert the inner tube into the tyre.
9. Place the two rim halves back on the tyre. Make sure that the tube valve pops out of the opening in the rim.
10. Connect the rim halves with the socket screws. Tighten by hand.
11. Inflate the tyre at the correct pressure, see the indication on the tyre.
12. Mount the wheel and tighten the bolt (23).
13. Lower the jack.
14. Make sure the socket bolts are firmly secured.
15. Click the hubcaps back in place.

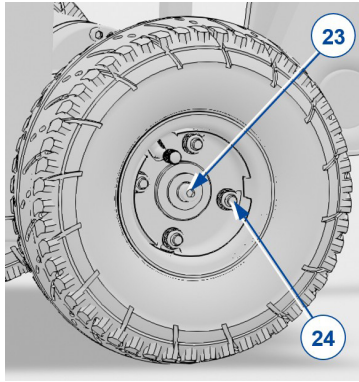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All rights reserved, including translation.