Installation manual

MANUEL D'INSTALLATION
INSTALLATIEHANDLEIDING
INSTALLATIONSANLEITUNG
MANUALE DI INSTALLAZIONE
MANUAL DE INSTALACIÓN
INSTRUKCJA INSTALACJI
PŘÍRUČKA PRO INSTALACI

Alois, Alzeis

Contents

Preface	4
1. This product	5
2. Scope of delivery	6
3. Transport	6
3.1. Loading the transport trolley	7
3.2. Assembling the transport set	7
4. Assembling the bed	8
4.1. Tools	8
4.2. Assembly	8
4.3. Emergency lowering of the headrest	15

Preface

To support you on the installation and repairs of this nursing bed, we provide you with this installation manual. Please read it carefully. If you still have questions after reading this manual, do not hesitate to contact Vermeiren.

The information in this manual applies to the following bed(s):

- Alois / Alzeis

Important note

Pictures of the product are used to clarify the instructions. Details of the depicted product may deviate from your product.

Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual For user and specialist dealer
	Installation instructions For specialist dealer
	Service manual for beds For specialist dealer
	Drawings of (spare) parts For specialist dealer
	EC declaration of conformity

1. This product

1. Head panel
2. Lifting pole / Trapeze handle
3. Mattress retainer
4. Foot panel
5. Side rail
6. Castor wheel with brake
7. Foot section
8. Head section
9. Undercarriage
10. Handset
11. Identification plate

2. Scope of delivery

The following components are part of the delivery:

- Undercarriage, incl. motor and transformer
- Slatted frame, foot-end incl. motor
- Slatted frame, head-end incl. motor
- Headboard
- Footboard
- Lifting pole / Trapeze handle
- Foldable side rail (L&R)
- Transport trolley
- Handset
- User manual
- Tools

Before use, check if everything is included and that no products are damaged.

3. Transport

CAUTION

Risk of injuries or damage

- Attach all bed parts and transport trolley firmly to the vehicle, to prevent them from moving during transport.
- Only transport the bed without the patient in it.

To avoid damage when transported, the lifting pole should be detached. Unplug the power cable and check that no cables hang near the floor.

If you are transporting the bed over a long distance, you can use the transport set.

3.1. Loading the transport trolley


CAUTION

Risk of injuries or damage

- Check if all parts are well secured.

3.2. Assembling the transport set

1. Lower the bed into its lowest position and then remove the power cable from the power socket.
2. Move the head and feet panels from the mattress slatted frame.
3. Insert one transport kit **(1)** into the mattress slatted frame **(2)** (e.g. head end). Secure the transport kit with the star handle **(3)**.
4. Insert the other transport kit into the mattress slatted frame on the other side (e.g. feet end). Secure the transport kit with the star handle.
5. Turn the two remaining star handles back into the intended positions on the mattress base.
6. Then turn the mattress slatted frame on the side where the transport kit is mounted thus moving the transport set's castors downward. The bed can now be pushed easily on the rolling castors.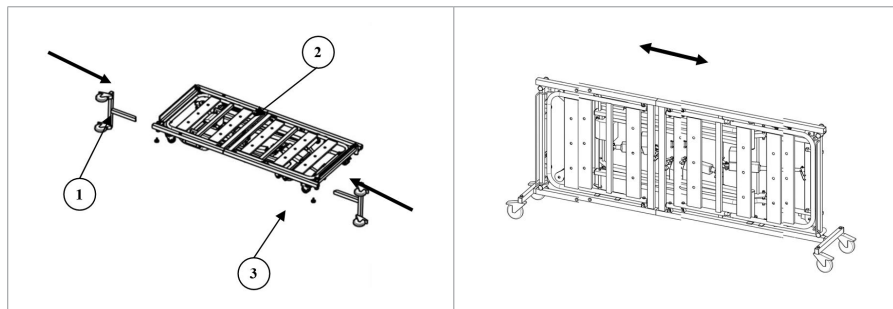

4. Assembling the bed

CAUTION

Risk of injuries or damage

- The bed needs to be (dis)assembled by a specialist dealer according to the instructions in this chapter.
- Only use the settings described in this manual.
- Only use parts described in this manual.
- Only use genuine spare or replacement parts approved by Vermeiren.
- Prevent bystanders from accessing the working area.
- Keep the (dis)assembling area clean and tidy, to prevent falling.
- Collect all small parts in a box or bag.
- While (dis)assembling and operating the bed, make sure no objects or body parts get caught between parts. In particular, pay attention to the position of the cables.

 Read the user manual of this bed. Be aware of the technical details and limits of intended use. For more information, contact Vermeiren.

4.1. Tools


To set up the bed, the following tools are needed:

- Screwdriver
- Allen key set

4.2. Assembly


4.2.1. Undercarriage

1. Place the frame with the wheels on a level underground. Activate the brakes on all four wheels.
2. Place the bracket with the bearings in the C-profile and screw in the other side.
3. Secure the motor with the locking pins **(1)**.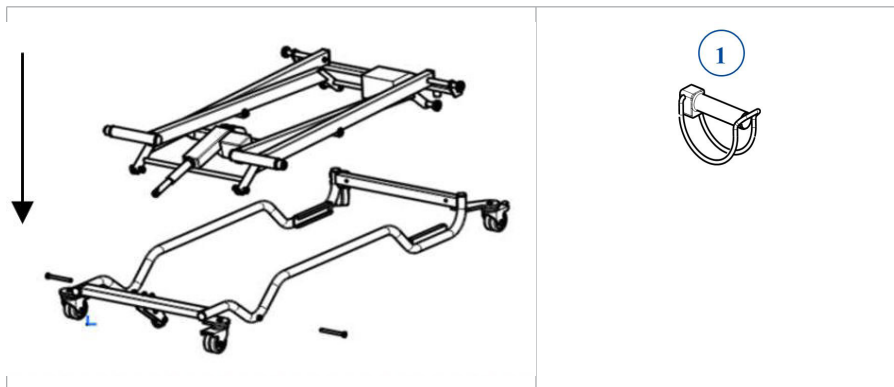


4.2.2. Slatted frame

1. Place the C-profile of the head section over the bearing holders on the undercarriage. The bolt in the C-profile serves as a stop.
2. Push the leg section over the joining tubes of the head end. Join the leg section to the subframe using locking pins.
3. Firmly screw the mattress slatted frame into the joining tubes using a screwdriver.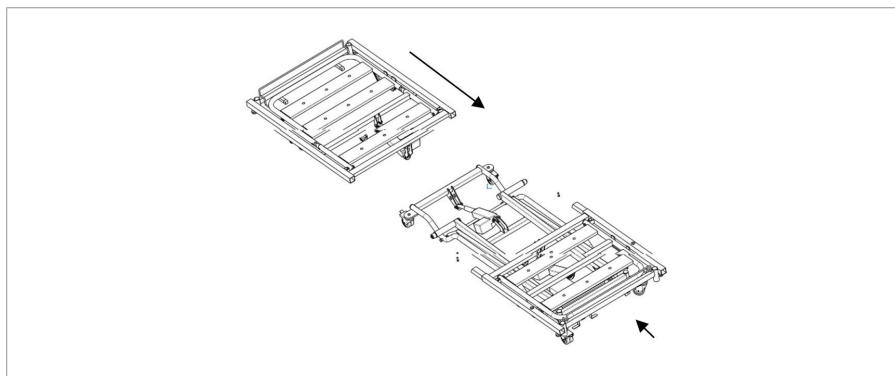

4.2.3. Head panel / foot panel


1. Slide the head panel (2) into the mattress slatted frame (1) at the head end of the bed. The fixing screws (3) must first be unscrewed.
2. Firmly tighten both fixing screws.
3. Repeat the steps above for the foot panel.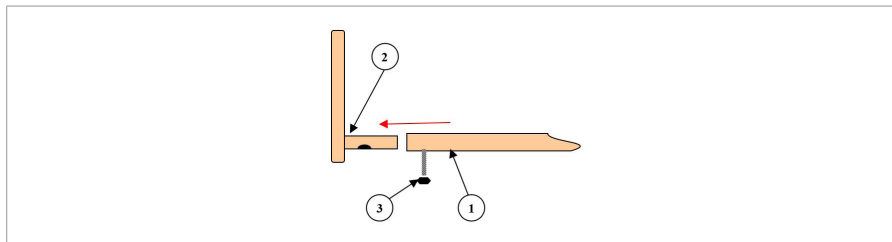

4.2.4. Side rails

4.2.4.1. Wooden side rails


Adjust the bed to medium height. If the bed is adjusted too low, it is not possible to mount the side rails; if it is adjusted too high, you have to pull the side rails too far upwards.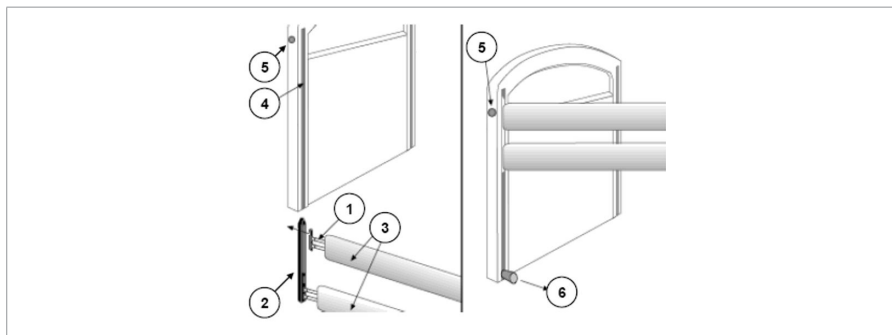

1. Insert the short bolt rails **(1)** on the right and left in the upper wood slat. Insert the long bolt rails **(2)** right and left in the lower wood slat.
2. Insert the short bolt rails **(1)** + the upper wood slat on one side (left or right) in the long bolt rails **(2)**. Slide the entire bolt rail system **(1)** + **(2)** in the relevant sliding rail **(4)** from the head and feet end. Grasp the upper wooden slat upwards until the wooden side rail locks in position.
3. Insert the locknuts **(6)** and tighten them manually.
4. Repeat these steps for the other side rail.

4.2.4.2. Metal side rails

1. Place the hooks **(1)** over the frame tube **(2)**.
2. Tighten the star knob **(3)**. Make sure that both sides have been tightened securely.
3. Repeat these steps for the other side rail.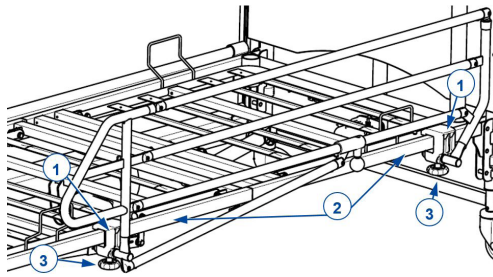


- i** The horizontal distance between headboard and side rail must be smaller or equal to 60 mm.
- The horizontal distance between footboard and side rail must be greater or equal to 250 mm.

4.2.5. Transformer connections

⚠ CAUTION

Risk of injuries

- Do not place any objects on the connection and the cable, and protect them against mechanical loads.
- Connection requires a 220V/230V, 50Hz power socket conform to VDE specifications.
 - The diagram for the control box connections is given in the figure below.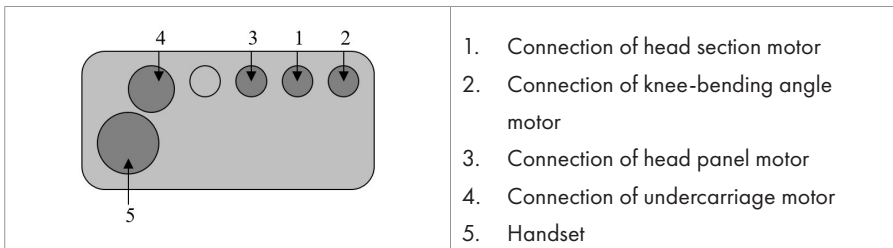


- i** The motor connection plugs are labelled with the same number as the connections on the transformer.

Take hold of the plug only to remove it from the power socket.

The power connection must not have any loops or kinks and should not pass around sharp edges. Protect the power connection and the cable leading to the handset from being crushed.

There is an equipotential connection (1) under the mattress slatted frame. This serves to bring conductive parts to the same potential. This connection is connected to an earth cable.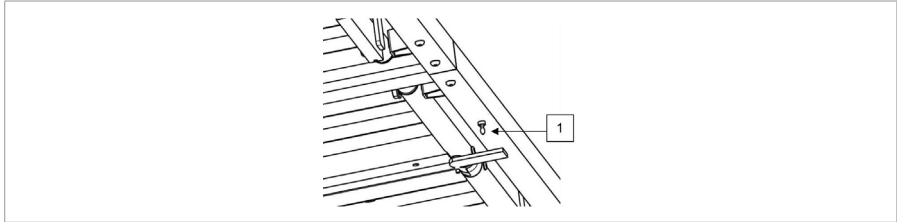

4.2.6. Strain relief and power cable



CAUTION

Risk of damage

- The power cable must be carefully positioned without kinks, without pulling, and without touching the floor.


Below the cross bar of the head slatted frame, there is a strain relief hook for the power cable.

1. Insert the relief unit of the power cable (4) into the suspension (3).
2. Pull the desired length of the power cable (1) in such a way that the power cable (2) can be laid under the mattress support platform without sagging. Be careful to avoid that the cable (2) come under the traction in any adjustment position.
3. Fasten the power cable by properly tightening the relief unit of the power cable (4).
4. Fasten the position of the relief unit by properly tightening the fixing screws (5).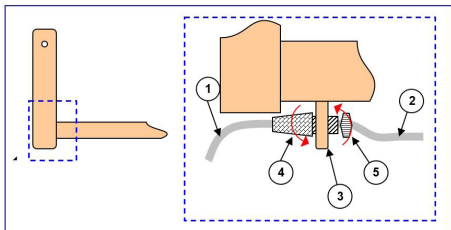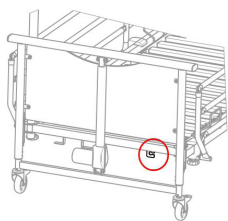

1. Power cable (power connection)
2. Power cable (transformer connection)
3. Suspension
4. Strain relief - Power cable
5. Strain relief - Fixing screw


4.2.7. Cable placement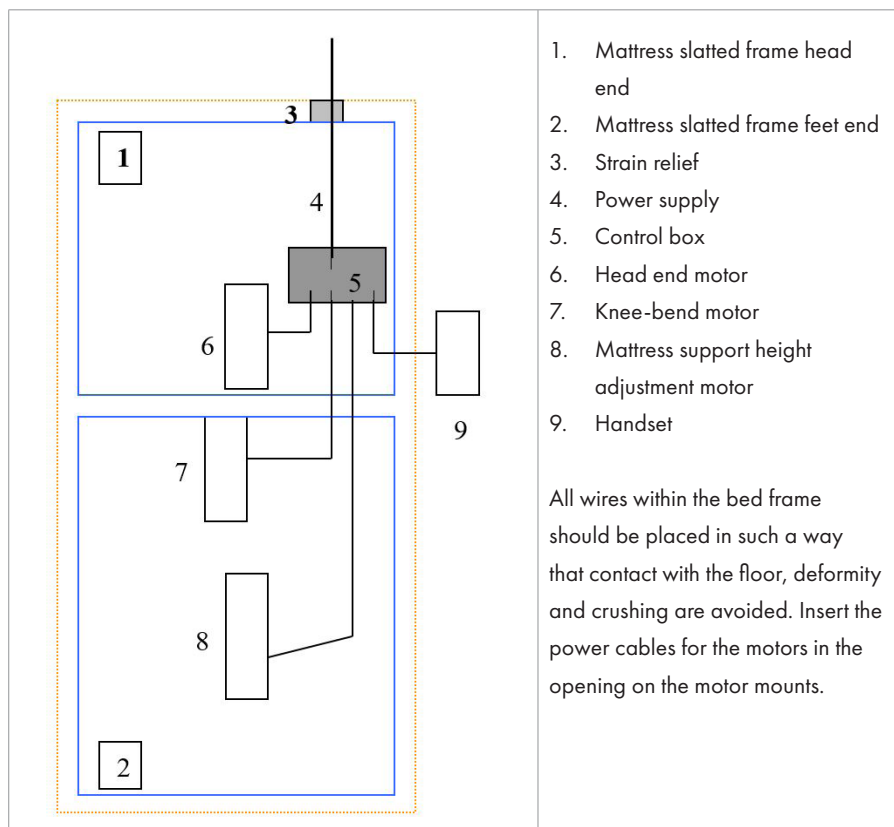

4.2.8. Lifting pole

The lifting pole can be mounted on either side of the head end.

1. Insert the pole (1) as far as it will go into the relevant socket (2).
2. Let the arresting bolt (3) slide into the groove (4).
3. Pull the metal ring (7) over the end of the upper frame (5) until the bolts engage one of the socket holes (6).
4. Also pull strap (8) into the metal loop and through the triangle (9); then fix its length.
5. Secure the strap.

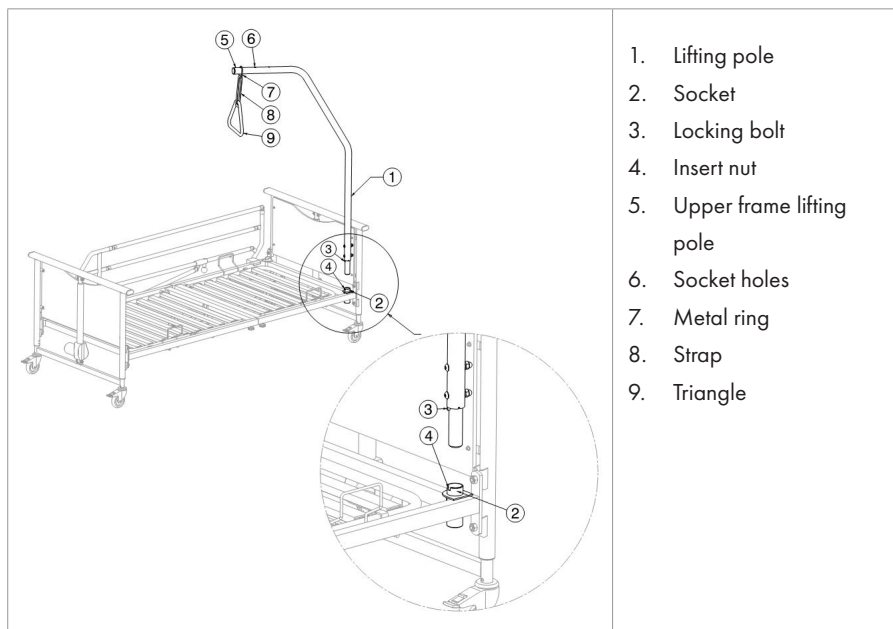4.3. Emergency lowering of the headrest

⚠ CAUTION

Risk of injuries

- Make sure that there are no objects or body parts between the headrest and the frame.
- Emergency lowering is only permitted if at least two persons are available.

The headrest can be lowered manually (emergency lowering in the case of motor failure):

1. Firmly grasp the headrest to prevent it from falling.
2. A second person can release the suspension pin by folding the securing batch and pulling the pin from the motor suspension.
3. Lower the headrest gently.

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All rights reserved, including translation.