## **B91 - Anti theft**

For some wheelchairs there is an option anti theft (B91).

This option is provided so that the risk of theft of your wheelchair significantly is reduced.

### **1. Technical specifications**

In the technical specifications of your standard wheelchair, only the folded width of 300 mm of your wheelchair is no longer possible. Your wheelchair is no longer foldable after mounting the anti theft system B91.

The anti theft system is available in different lengths associated with the various seat widths of your wheelchair. Ensure that you give the correct length when ordering.

### **2. Safety rules**

- ⚠ **Attention: Your wheelchair is no longer foldable after mounting the anti theft system B91.**
- ⚠ **Check that the anti theft system is properly fixated to the frame of your wheelchair.**

### **3. Use**


Some wheelchairs can be foreseen with an anti theft system. This system can be used in supermarkets or rest homes where the risk of theft of your wheelchair is quite large.

If your wheelchair has to be folded for transport in a car several times, it is not advisable to use this option B91. Some wheelchairs can be used as a seat in a motor vehicle, for persons that use their wheelchair as a seat in a motor vehicle the option B91 can be used.

### **4. Mounting option B91**

To mount the option B91 you can use following instructions:

1. Unscrew the cap nut ① on both sides of the front side of the wheelchair frame.
2. Remove the washers ②. Do not mount these washers again, when attaching the anti theft system (B91).
3. Mount the anti theft system (B91) ③.
4. Remount the cap nut ①.
5. Check that the anti theft system is properly fixated to the frame of your wheelchair.

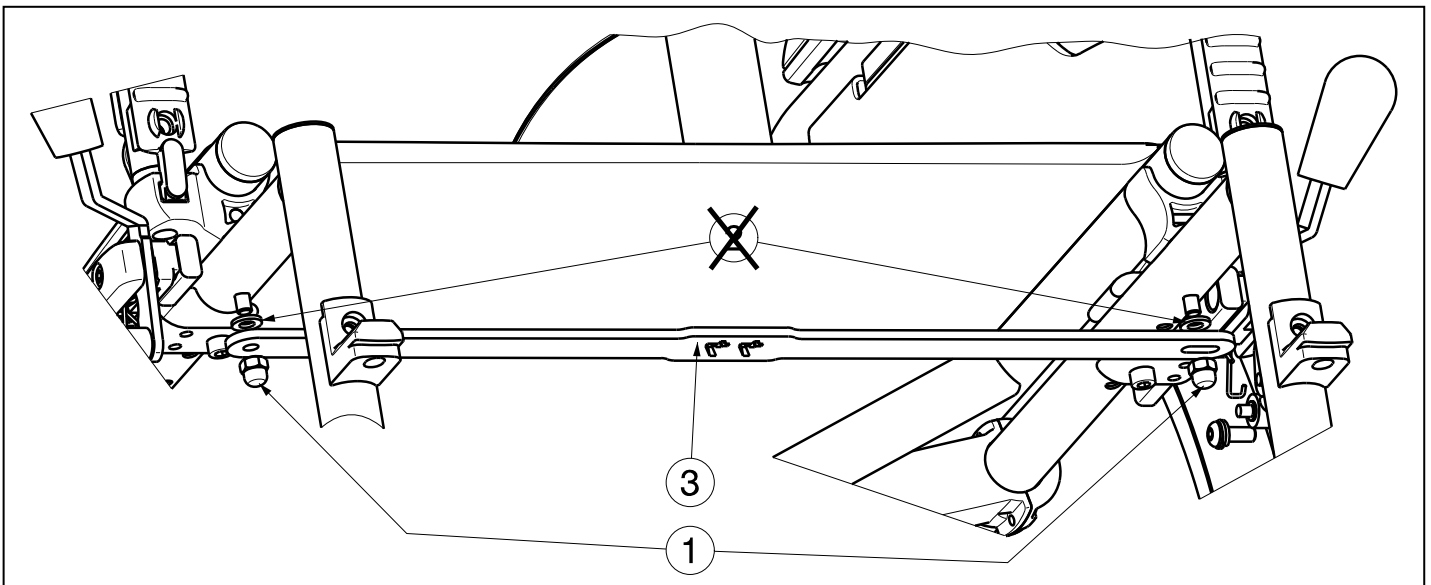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### Vermeiren Suisse S.A.