User manual

MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

276

Contents

Preface	4
1. Your product	5
2. Before use	6
2.1. Intended use and indications	6
2.2. Safety instructions and applicable risks	7
2.3. Symbols on the rollator	8
2.4. Upon delivery	9
2.5. Shipping and storage	9
3. Assembly/Disassembly and adjustments	10
3.1. Assembly	10
3.2. Height adjustments	10
3.3. Adjusting the brakes	12
4. Using your rollator	12
4.1. Brake system	12
4.2. Folding/Unfolding	13
4.3. Sitting down, standing up and walking	14
5. Maintenance	15
5.1. Cleaning	15
5.2. Disinfection	15
5.3. Inspection / Service / Repairs	15
5.4. Reuse	16
5.5. Disposal at end of life	16
6. Technical specifications	17

Preface

Congratulations! You are now owner of a Vermeiren rollator!

Thank you for your trust in the products of Vermeiren. To support you on the use of this rollator and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your rollator.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

Important note

To ensure your safety, and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures of the product are used to clarify the instructions. Details of the depicted product may deviate from your product.

Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of the following information. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual For user and specialist dealer
	Service manual for rollators For specialist dealer

1. Your product

1. Handgrips
2. Brake lever
3. Underarm support
4. Backrest
5. Seat
6. Plastic tray
7. Basket
8. Front wheels
9. Rear wheels
10. Brake cable
11. Identification plate

2. Before use

2.1. Intended use and indications

- This product is a medical device.
- Indications and contraindications: The rollator is meant as support of the leg function during walking and daily activities. With the underarm supports, it is specifically suited for users suffering from mobility impairments involving reduced movement/balance but with sufficiently preserved walking ability and coordination, when the use of the standard rollator is not possible due to impaired hand function and the use of other walking aids is insufficient. Do not use the rollator if you have other physical limitations that may cause unsafe use of your rollator, like a weak arm, balance disorders, impaired vision, limb loss, ...
- The rollator is designed and produced solely to be used as walking aid for one person with a maximum weight of 120 kg.
- The rollator will compensate the lack of adequate support by the legs by leaning on, or moving with, the rollator with usage of the upper limbs or chest. The rollator will provide better stability, support and safety during walking.
- The rollator can be used indoors and outdoors.
- It is not intended for transport of goods and/or more people than the user. It is also not intended to be used as a transport aid while sitting down.
- It should only be used on surfaces where all four wheels are touching the ground. Do not use the rollator on rocky, bumpy or sandy terrain, or places with a steep inclination. The rollator may become unstable. Look for flat, stable areas, and bevelled kerbs to walk with your rollator.
- The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.
- Please check all technical details and limits of your rollator in §6..

2.2. Safety instructions and applicable risks

CAUTION

Risk of injuries

- Please read and follow the instructions in this manual. Otherwise you may get injured, or your product may get damaged.

Keep the following general warnings in mind during use:

- Check that no stones or other objects block the wheels of the rollator.
- Make sure that both brake levers are in parking mode before you use the rollator as aid to stand up. When you are standing comfortable and stable, you can unlock the parking brakes.
- While walking, keep both hands on the handgrips.
- Be aware that some parts of your rollator may get very hot or cold due to ambient temperature, solar radiation, heating devices, etc. Be careful when touching.
- Before every use, check that the height adjustments are firmly secured, see **§3.2.**
- Do not hold the handgrips with wet hands. If you do so, you can lose your grip and lose your balance.
- Do not proceed with your rollator into traffic. Always stay on the pavement.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing / care.

Any serious incident [MDR (EU) 2017/745 §2 (65)] that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Be aware that your rollator may interfere with some types of anti-theft systems, depending on the settings used. This may cause the shop's alarm to go off.

2.3. Symbols on the rollator

	Maximum weight of the user in kg
	Type designation
	Catalogue number
	Serial number
	Medical device
	Manufacturer
	Date of manufacture
	Declaration of conformity
	Attention: important information
	It is advised to read the manual

2.4. Upon delivery

Unpack your product and verify if the delivery is complete. The following items should be included:

- Base frame with seat, 4 wheels, backrest;
- Handgrips with brakes
- Basket
- Plastic tray
- Manual

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

2.5. Shipping and storage

The shipping and storage of the rollator should happen according to the technical specifications in §6.. Make sure that the rollator is stored dry.

Provide sufficient covering or packaging to protect the rollator from rust and foreign bodies (e.g. salt water, sea air, sand, dust).

Make sure no objects are stored on or against the rollator to prevent damage.

3. Assembly/Disassembly and adjustments

3.1. Assembly

1. Slide the handgrips in the front frame.
2. Adjust the handgrips to the desired height (see §3.2.).

CAUTION

Risk of falling or tipping over

- Do not overload the basket.

The maximum carrying weight of the shopping basket can be found in §6..

The plastic tray can be inserted from the front below the seat. If you want to use the tray, it can be placed on top of the basket.

The shopping basket is mounted to the frame below the seat with 2 hooks. The bottom of the basket rests on the front transversal tube (between the front wheels).

3.2. Height adjustments

CAUTION

Risk of falling or tipping over

- Firmly fix the handgrips before using the rollator.
- Do not exceed the minimum or maximum adjustment as indicated on the handgrips. Your rollator will not be stable anymore.

CAUTION

Risk of damage

- Don't pull the handgrips too high to prevent damage to the brake cable.

3.2.1. Handgrip height

The rollator can be altered in height by adjusting the handgrips:

1. Put your rollator in parking mode, see §4.1..
2. Loosen and remove the star button (1).
3. Remove the bolt from the tube.
4. Adjust the handle to the desired height (7 positions, steps of 25 mm).
5. Place the bolt back in the hole of the tube.
6. Place the star button (1) on the bolt. Retighten firmly by hand.
7. Repeat for the second handgrip. Make sure that both handgrips are well secured, and that they are set to the same height.

3.3. Adjusting the brakes

The force of the brakes can be adjusted by loosening/tightening the screw (14) on the back of the brake lever.

4. Using your rollator

4.1. Brake system

CAUTION

Risk of falling

- Check the working of the brakes to ensure your safety. The condition of the brakes is influenced by wear, oil and other contamination of the rear wheels and braking mechanism.

The rollator is equipped with brakes, operated by two brake levers (15). The brake levers operate in two modes: running brakes and parking brakes.


CAUTION

Risk of damage

- Do not push the rollator too hard while using the running brakes. This may cause damage to the tyres or brakes.

To use the running brakes, pull both brake levers (15) simultaneously upwards [A] in a squeezing motion. To stop braking, release the brake levers.

To use the parking brakes, push both brake levers **(15)** downwards in locked position [B]. Always use both parking brakes simultaneously. To end the parking mode, pull both brake levers simultaneously upwards.

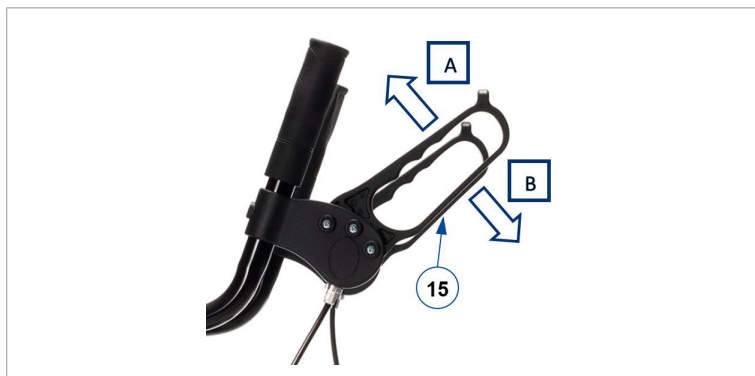4.2. Folding/Unfolding

⚠ CAUTION

Risk of entrapment

- Keep fingers, buckles, jewellery and clothes away from the folding mechanism.

⚠ CAUTION

Risk of falling

- Check that the rollator is completely open and that the frame is locked.

4.2.1. Unfolding the rollator

1. Push both brake levers down to parking position, see §4.1..
2. Pull the rear frame tubes (3) backwards to open the frame. Push down on the backrest (4) to make sure that the frame is fully opened.
3. Place the shopping basket and plastic tray.
4. Adjust the rollator to the desired height (see §3.2.).

4.2.2. Folding the rollator

1. Stand behind the rollator.
2. Remove the shopping basket and plastic tray.
3. Close the frame completely by pushing front and rear frame tubes together.

4.3. Sitting down, standing up and walking

CAUTION

Risk of falling

- Make sure that both brake levers are in parking position before sitting down, see §4.1.. Keep the brake levers in this position until you start walking again. Keep your feet between the rear wheels.
 - Do not propel the rollator while seated.
1. Push both brake levers down to parking position.
 2. While leaning on the handgrips, turn around and sit down on the seat.
 3. To stand up again, lean on the handgrips and turn around.
 4. Pull both brake levers up and use the running brakes.
 5. Release the brake levers and step forward with the rollator close to your body.

5. Maintenance

- i** Regular care ensures that your rollator is preserved in a perfectly functional condition. For the maintenance manual, refer to the Vermeiren website: www.vermeiren.com.

5.1. Cleaning

Use a soft, damp cloth and lukewarm water with a mild household soap to clean the rollator. Avoid soaking. Do not use abrasive cleaning agents to clean.

5.2. Disinfection

CAUTION

Risk of damage

- Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

Use a soft, damp cloth and lukewarm water with a mild household disinfectant to clean the rollator.

5.3. Inspection / Service / Repairs

The expected service life of your rollator is 5 years. The lifespan of the rollator is influenced by its use, storage, maintenance, service and cleaning.

Before you resume use of the rollator, it should be inspected on the following points:

- Completeness.
- Folding mechanism: functioning well.
- Bolts and nuts: tightly fastened.
- Brakes: functioning well.
- Wheels (stability, tracking, fixing screws).

- In case of regular contact with wires, hair or other residues, the wheels may get blocked. Check regularly that the wheels rotate freely and that they are in good condition.
- Frame (deformation, stability, connections).
- Handgrips (safe locking of height adjustment system, good support of handgrips).

Your rollator is constructed of aluminium and plastic components. We recommend to have your product inspected at least once a year, but the service frequency depends on the frequency and intensity of use. Consult your specialist dealer to make an agreement on service frequency.

Repairs and replacements may only be undertaken by trained persons, and only genuine replacement parts of Vermeiren should be used.

Do not hesitate to use the services offered by your specialist dealer. He/she will be glad to assist you regarding any service and repairs.

5.4. Reuse

Before each reuse, have the rollator disinfected, inspected and serviced according to the instructions in **§5.2.** and **§5.3.** .

5.5. Disposal at end of life

At end of life, you need to dispose your rollator according to the local environmental legislation. The best way to do so, is to disassemble the rollator to facilitate the transport of recyclable parts.

6. Technical specifications

The technical details below are only valid for this rollator, at standard settings and optimal ambient conditions. Take these details into account during use.

Brand	Vermeiren
Product group	Rollator
Type	276
Description	Dimensions
Maximum user weight	120 kg
Maximum basket load	5 kg
Overall length	750 mm
Folded length	440 mm
Overall width	630 mm
Folded width	630 mm
Overall height	1100 - 1230 mm
Folded height	1100 - 1230 mm
Handgrip height	1100 - 1230 mm
Recommended body height	1650 - 1900 mm
Distance between handgrips	445 mm
Seat height	570 mm
Seat width	440 mm
Seat depth	210 mm
Total mass - without basket	11 kg 9,5 kg
Wheel diameter	200 mm
Brakes	Running & parking brakes
Turning width	900 mm
Storage and use temperature	+5°C - +41°C
Storage and use humidity	30% - 70%
We reserve the right to introduce technical changes. Measurement tolerance +- 15 mm / 1,5 kg / 1,5°	

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**Service registration form**

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:

VERMEIREN GROUP

Vermeirenplein 1/15

2920 Kalmthout

Belgium

www.vermeiren.com

Version: D, 2025-01

Basic UDI-DI: 54151741206062765U

All rights reserved, including translation.

VERMEIREN
we care **for** you