
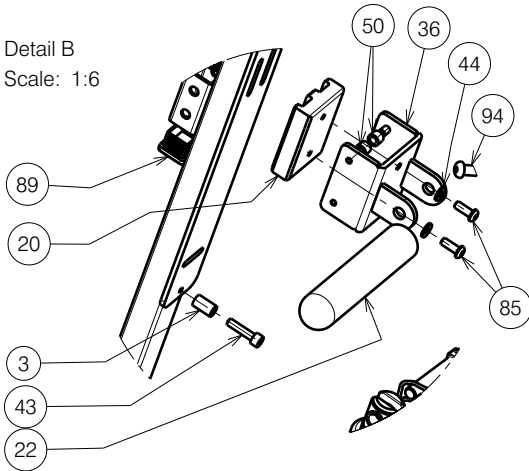
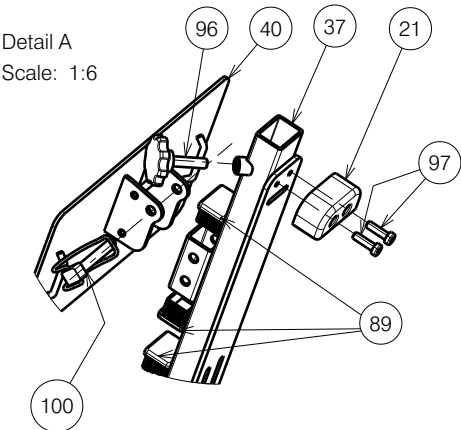


Tighten with torque wrench 25 Nm W600135: nr.46 and nr.92

Use glue WEICON 302-43 when tighten screws 73 and 81

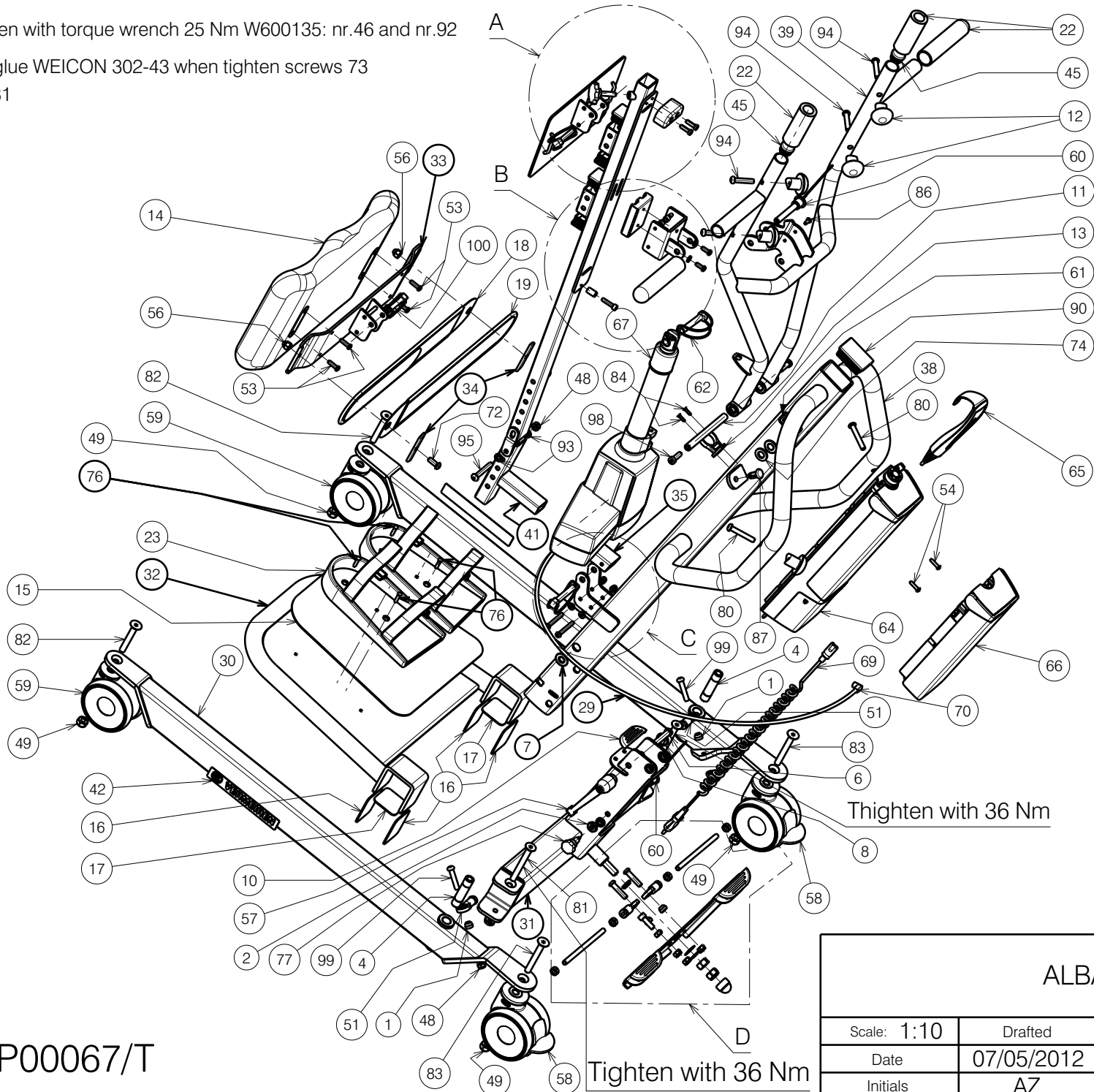


Tighten with 36 Nm

Tighten with 36 Nm


EP00067/T

ALBATROS 2				Plan No.
1/3				EP00067/ T
Scale: 1:10	Drafted	Verified	Confirmed	
Date	07/05/2012	13/02/2025	13/02/2025	
Initials	AZ	PM	MK	


8002108

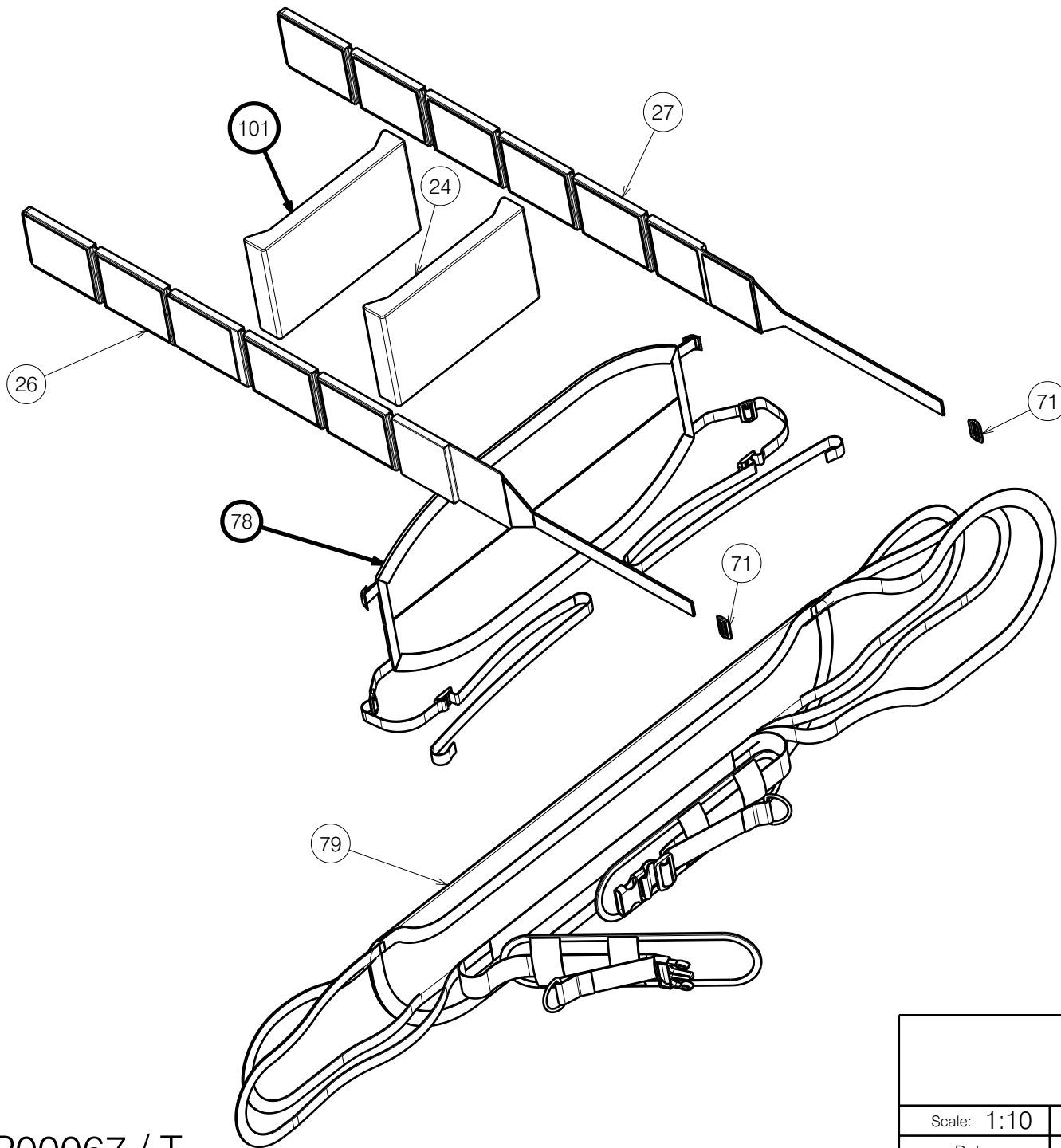

Detail C
Scale: 1:6

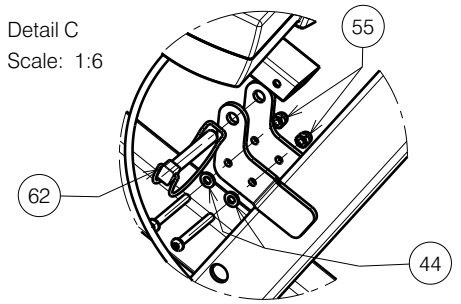

Detail D
Scale: 1:6

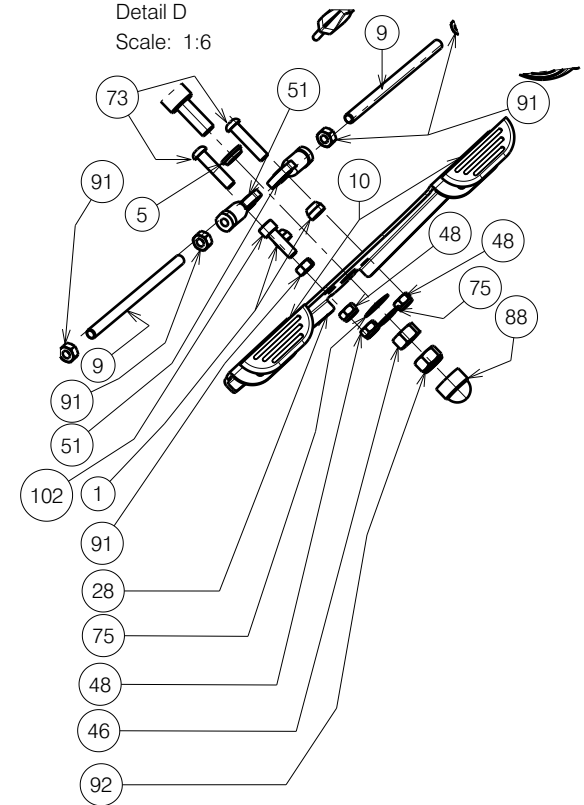

8002108

EP00067 / T

ALBATROS 2				Plan No.
2/3				EP00067 / T
Scale: 1:10	Drafted	Verified	Confirmed	
Date	07/05/2012	13/02/2025	13/02/2025	
Initials	AZ	PM	MK	

Material				Material				Material				8002108			
No.	Plan	Article	Quantity	No.	Plan	Article	Quantity	No.	Plan	Article	Quantity				
1	P00074772	2098400	4	36	YP03604	2832419	1	72	M8x20	1992349	2				
2	X001398	1009689	1	37	YP03610	2832519	1	73	M8x35	1993236	1				
3	X003802	1900373	1	38	P00040743	2082919	1	74	D13	1993362	2				
4	P00078139	G105000	2	39	YP03622	2833019	1	75	---	1993500	2				
5	X004288	1903587	1	40	YP03626	2832719	1	76	Rivet 5x20	1003897	4				
6	X004338	1708700	1	41	YP03646	2832819	1	77	---	1999935	2				
7	X004341	1708800	2	42	---	T001052	2	78	---	2991605	1				
8	X004353	1903588	1	43	M6x25	1001494	1	79	---	2991627	1				
9	P00075688	2098500	2	44	D6	1001564	4	80	M8x50	1902483	1				
10	X004580	1904798	3	45	---	1001608	2	81	M10x90 DIN7991	1921197	2				
11	X004603	1734700	1	46	M12	1001673	1	82	M10x70	1903406	2				
12	X004605	1905610	6	47	---	---	-	83	M10x80	1905617	2				
13	X004611	1904945	1	48	M8	1001682	6	84	Rivet4x14	1001584	2				
14	X004630	1905252	1	49	M10	1001683	4	85	M6x16	1008459	2				
15	X004647	1905718	1	50	---	1004550	4	86	4.8x12	1001585	2				
16	X004704	1905720	4	51	---	1006835	4	87	---	1009583	1				
17	X004705	1905721	2	52	M6x50	1006997	2	88	---	1003294	1				
18	X004717	1905722	1	53	M6x20	1006999	4	89	---	1902851	1				
19	X004718	1905723	1	54	M5x20	1998368	2	90	---	1905096	4				
20	XP00901	2992476	1	55	M6	1008492	2	91	M8	1001671	1				
21	XP00902	2833119	1	56	M8	1902024	2	92	M12	1001684	5				
22	---	1990766	4	57	M10x23	1903086	1	93	D8,7	1001556	1				
23	XP01288	2992331	2	58	---	1903402	2	94	M8x40	1008739	2				
24	XP00917	2991600	1	59	---	1903403	2	95	M8x55	1995854	4				
26	XP00919	2991603	1	60	---	1903578	2	96	M8x30	1008851	1				
27	XP00920	2991604	1	61	--	1904556	1	97	M6x20	1904811	1				
28	P00074771	2098354	1	62	D10x50	1998885	2	98	M8x25	1003944	2				
29	Y014535	1706719	1	63	M8x50	1905369	1	99	M8x45	1994473	2				
30	Y014536	1706819	1	64	CAL40	1918030	1	100	D8x50	1905609	2				
31	Y014537	1706919	1	65	HB30	1918031	1	101	---	2993737	1				
32	Y016061	1735219	1	66	BAL40	1918029	1	102	M8x25	1001499	1				
33	Y016137	1732619	1	67	7.5 kN	1918033	1	103	M10 DIN439B	1903216	1				
34	Y017221	1743419	2	68*	---	1918034	1	ALBATROS 2				3/3	EP00067/ T		
35	YP03600	2832319	1	69	---	1906233	1								
				70	---	1918032	1								
				71	---	1991236	2								
EP00067 / T				* Not depicted - optional charger				Scale: 1:10				Drafted	Verified	Confirmed	
								Date				07/05/2012	13/02/2025	13/02/2025	
								Initials				AZ	PM	MK	