

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 1/3

2992223


FOLDABLE QUICK RELEASE ARMREST

Plan No.

E000946/U

Scale:	1:4	Drafted	Verified	Confirmed
Date	27.02.2014	14.10.2025	14.10.2025	
Initials	SD	PM	MK	



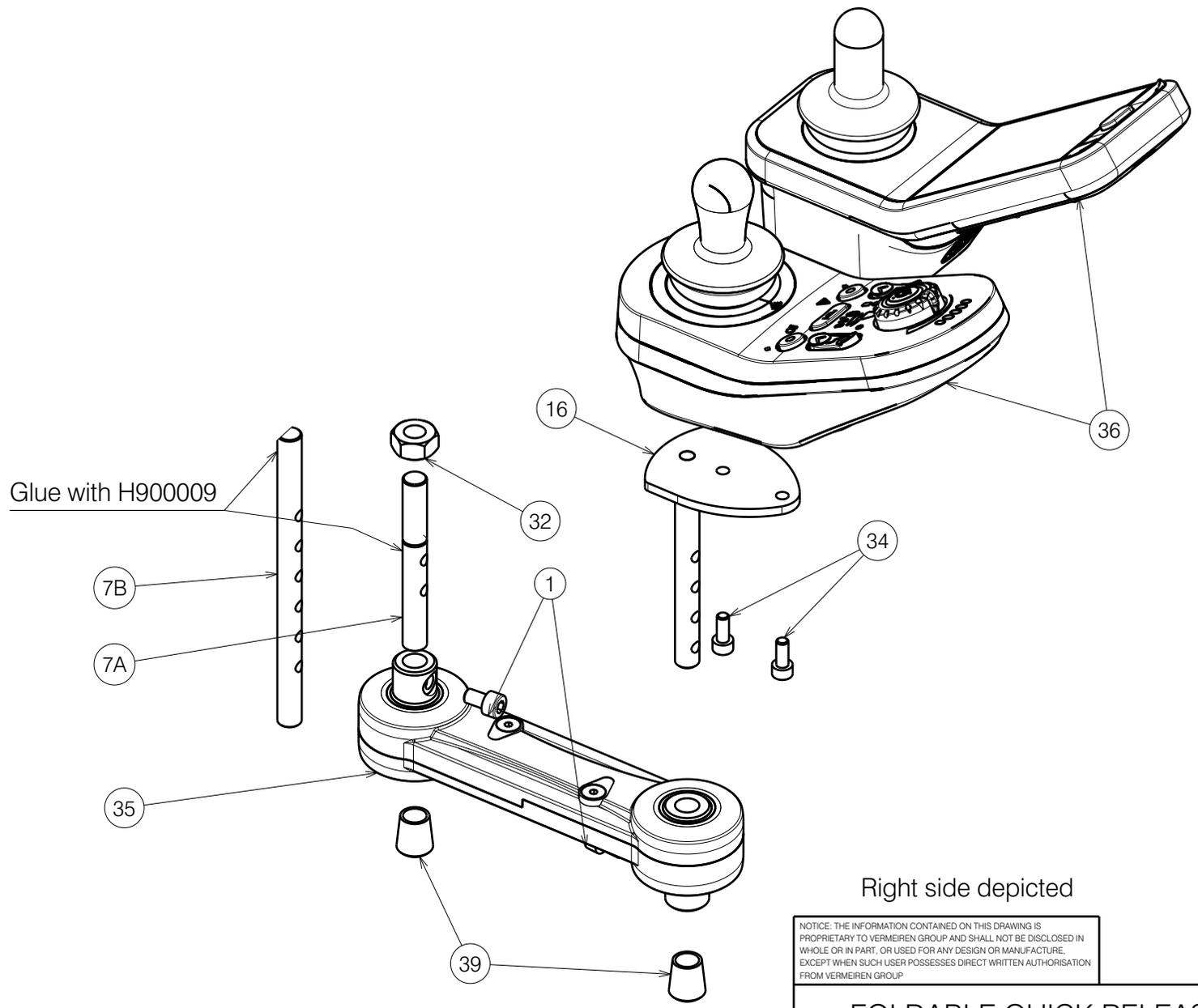

Right side depicted

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 2/3 2992223

FOLDABLE QUICK RELEASE ARMREST

Scale:	1:4	Drafted	Verified	Confirmed
Date	27.02.2014	14.10.2025	14.10.2025	
Initials	SD	PM	MK	

Plan No.
E000946/U

				Material							
No.	Plan	Article	Quantity	No.	Plan	Article	Quantity	No.	Plan	Article	Quantity
1	X002112	1001491	2	21A	Y020645	2003054 (R)	1	43	M6x6 ISO7380	1906051	2
2a	X003700	1998790 (R)	1		Y020644	2002954 (L)	1	44*	2,0 m	1912412	1
	X003699	1998789 (L)	1	21B	Y020719	2003854 (R)	1	45*	1,5 m	1912299	1
2b	Black soft	2998161 (R)	1		Y020718	2003754 (L)	1	46*	0.5 m	1912303	2
	Black soft	2998160 (L)	1	21C	Y023363	2055254 (R)	1	* Not depicted			
	Black skai	1913931 (R)	1		Y023362	2055154 (L)	1				
	Black skai	1913930 (L)	1	22	M8X35 DIN933	1001444	1				
	Soft grey	1911556 (R)	1	23	M6X16 DIN912	1001492	2				
	Soft grey	1911555 (L)	1	24	---	1001581	1				
3	X004423	1717300	1	25	---	1001616	2				
4	X004736	1905736	2	26	M6 DIN985	1001680	2				
5	X004997	1773200	1	27	M8 DIN125	1001556	1				
6	X005004	1773354	1	28	X005401	1910679	1				
7	X005100	G116500 (St.)	1	29	M6X20 ISO7380	1006999	2				
	X004992	G116300 (Option)	1	30	M6X12 ISO7380	1007000	4				
8	X005213	1910168	1	31	D3x10	1008674	1				
9	X005215	1910104 (R)	1	32	M10X1 DIN934	1901345	1				
	X005216	1910105 (L)	1	33	M6X16 DIN7991	1994752	1				
10	X005272	1798600	1	34	M5X12 DIN912	1001489	2				
11	X005273	1791000	1	35	P000027	1908428	1				
12	XP00190	2990049 (L)	1	36	SE39	1912301	1				
	XP00085	2990047 (R)	1		SE39-SE19	1912296	1				
13	Y018161	17694.. (R)	1		SE39-SE19-ACT	1912305	1				
	Y017339	17693.. (L)	1		LCD	1913523	1				
14	Y018173	1770254	1		Drive only	1910700	1				
15	Y018397	17696.. (R)	1	38	M4x4 DIN916	1910674	1				
	Y018394	17695.. (L)	1	39	---	1900072	2				
16	Y021992	2021854	1	40	M6x20 DIN912	1001493	1				
17	X005367	1910362	1	42	Y021389	2012054	1				
18	Y020143	2002854	2								
19	X004994	1773154	1								
20	Y020367	20026 (R)	1								
	Y020368	20027 (L)	1								
20A	X005214	G118254	1								
20B	X005270	G001183	1								
20C	X005282	G101900	1								

Notes	
21A - Regular models	
21B - SUD size 45	
21C - SUD size 40	
Configuration	Cables
SU (stand-up)	1x44 - 1x45
SE42 (lift)	1x44 - 1x45
SE09 / SU	1x44 - 1x45 - 2x46
SE09 / SE42	1x44 - 1x45 - 2x46

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 3/3 2992223

FOLDABLE QUICK RELEASE ARMREST				Plan No.
				E000946/U
Scale:	1:4	Drafted	Verified	Confirmed
Date	27.02.2014	14.10.2025	14.10.2025	
Initials	SD	PM	MK	

