

Glue N°8 into N°1  
 Loctite 495 instant  
 adhesive H900006.

Material			
No.	Plan	Article	Quantity
1	YP00376	18120...	1
2	---	2990076 (grey)	2
	---	2990079 (black)	
3	---	2990075 (grey)	1
	---	2990080 (black)	
4	---	2990077 (grey)	1
	---	2990081 (black)	
5	---	1998324 (grey)	1
	---	2990082 (black)	
6	XP00106	2990072	2
7	---	1996073	1
8	X003293	1001626	4
9	---	1003903	4
10	---	1007954	4
11	---	1005767	4
12	---	1001608	2
13	---	1001561	4
14	Sticker	T001051	1
15	Sticker	T001097	1
16*	XP03128	2996685	8
17*	Cardboard	PP00961	4
18*	Feet	2994744	2


Bucket set - article 1996073

No.	Plan	Article	Quantity
7a	XP00097	2990065	1
7b	XP00098	2990066	1
7c	XP00113	GP00900	1

\*use for packaging, quantities for whole package of 5 chairs


NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

E000120 Toilet Chair 9062				---
				Plan No. E000120/Q
Scale: 1:8	Drafted	Verified	Confirmed	
Date	11/02/2004	30-10-2025	30-10-2025	
Initials	BS	MP	SD	

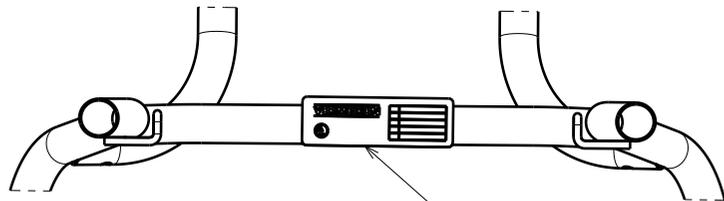

Central position

15

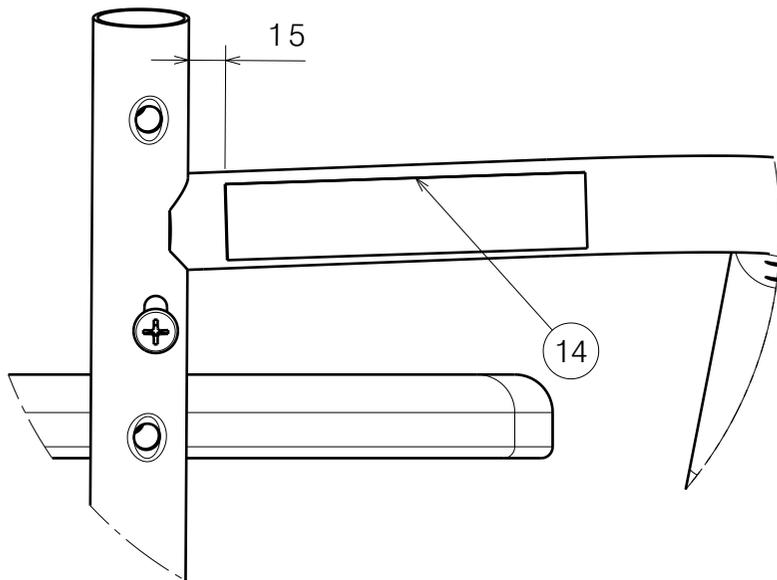

Material			
No.	Plan	Article	Quantity
1	YP00376	18120...	1
2	---	2990076 (grey)	2
	---	2990079 (black)	
3	---	2990075 (grey)	1
	---	2990080 (black)	
4	---	2990077 (grey)	1
	---	2990081 (black)	
5	---	1998324 (grey)	1
	---	2990082 (black)	
6	XP00106	2990072	2
7	---	1996073	1
8	X003293	1001626	4
9	---	1003903	4
10	---	1007954	4
11	---	1005767	4
12	---	1001608	2
13	---	1001561	4
14	Sticker	T001051	1
15	Sticker	T001097	1
16*	XP03128	2996685	8
17*	Cardboard	PP00961	4
18*	Feet	2994744	2

\*use for packaging, quantities for whole package of 5 chairs

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

---
Plan No.
E000120/Q

E000120 Toliet Chair 9062			
Scale: 1:1	Drafted	Verified	Confirmed
Date	11/02/2004	30-10-2025	30-10-2025
Initials	BS	MP	SD