


No.	Plan	Article	Qty
1	FRONT COVER	29035	1
2	3M DUAL LOCK 15mm x 25mm	2998521	2
3	RUBBER	3992256	4

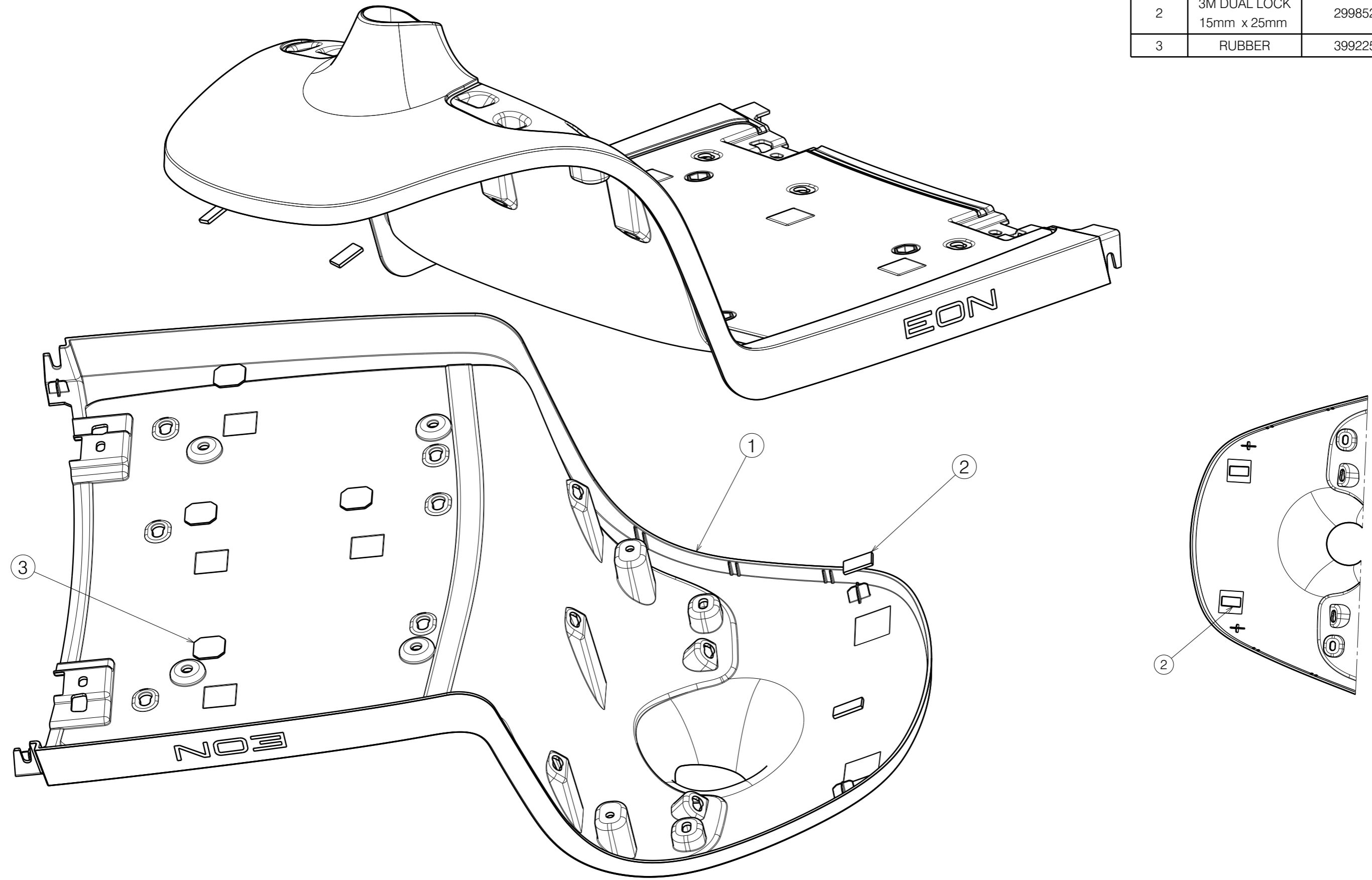

- Before stick 3M Dual Lock prepare surface on plastic part:
1. Roughen the surface by abrasive pad
 2. Clean surface with Isopropyl Alcohol (IPA), propan-2-OL.
 3. Press 3M Dual Lock after stick

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORIZATION FROM VERMEIREN GROUP.

SV EON FRONT COVER COMPL.			
Scale:	1:5	Drafted	Verified
Date	22/07/2025	03/10/2025	03/10/2025
Initials	BK	MK	TK

Plan No.
E00108383/A.2