


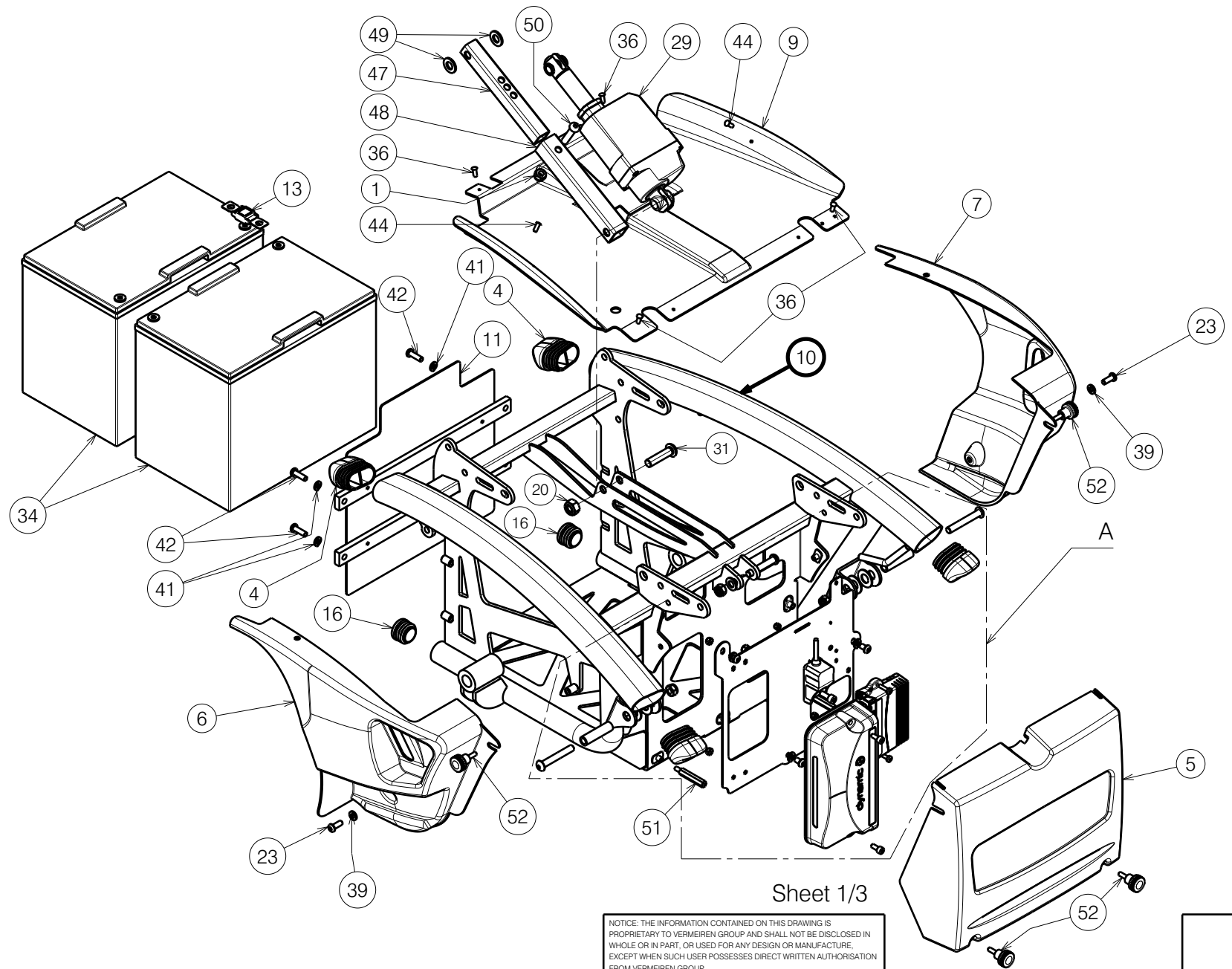

Sheet 1/3

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.


## FRONT LOWER FRAME TIMIX

|            |            |            |            |
|------------|------------|------------|------------|
| Scale: 1:7 | Drafted    | Verified   | Confirmed  |
| Date       | 03/07/2012 | 10/06/2025 | 10/06/2025 |
| Initials   | JG         | AZ         | MK         |

Plan No.

E000823/ N



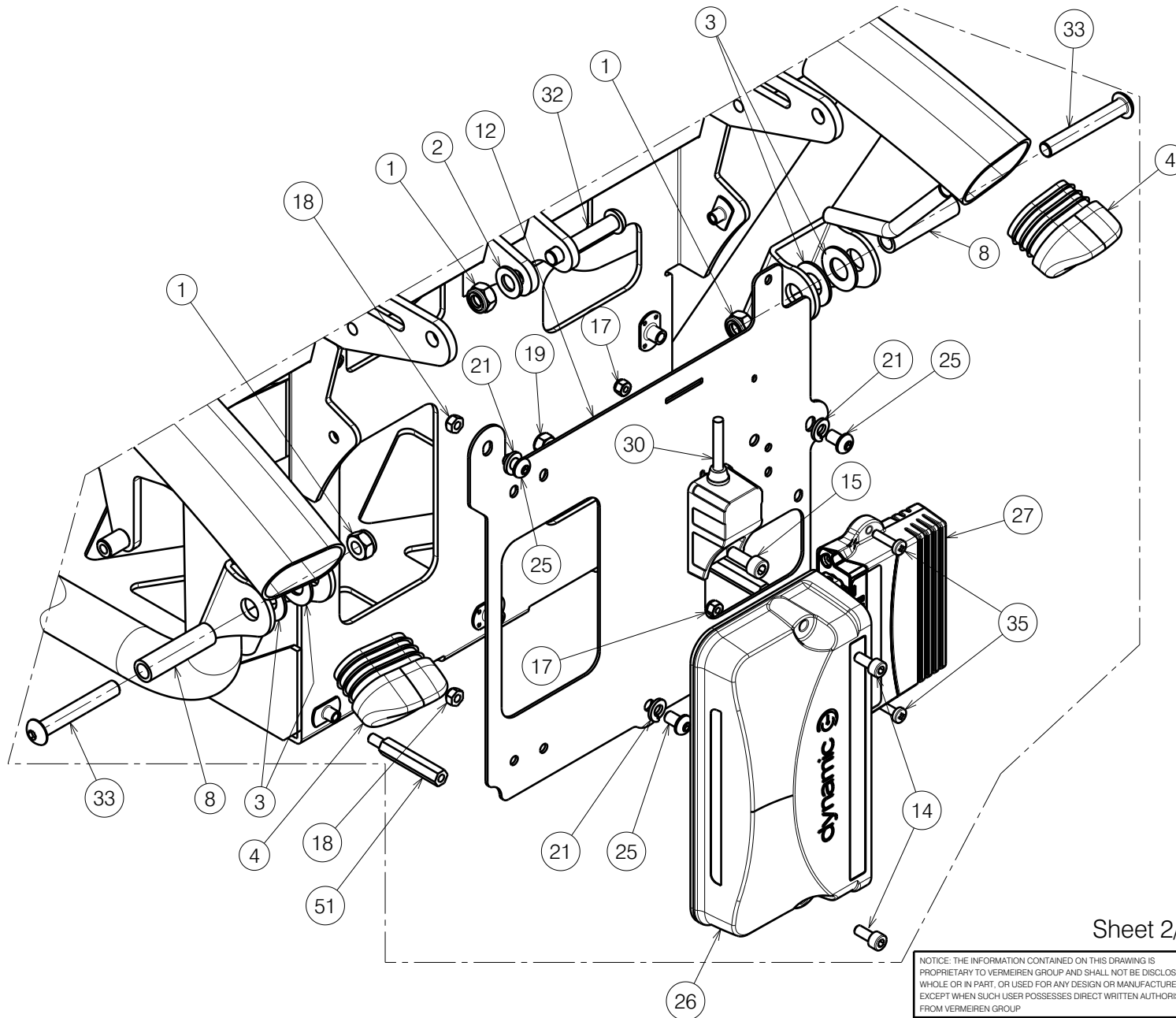

Sheet 2/3

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS  
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN  
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,  
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION  
 FROM VERMEIREN GROUP.

## FRONT LOWER FRAME TIMIX

|            |            |            |            |
|------------|------------|------------|------------|
| Scale: 1:7 | Drafted    | Verified   | Confirmed  |
| Date       | 03/07/2012 | 10/06/2025 | 10/06/2025 |
| Initials   | JG         | AZ         | MK         |

Plan No.

E000823/ N



| Bill of Material |                  |         |          |
|------------------|------------------|---------|----------|
| No.              | Plan             | Article | Quantity |
| 1                | X001016          | 1001682 | 4        |
| 2                | X001119          | 1001554 | 1        |
| 3                | ---              | 1908512 | 4        |
| 4                | X004778          | 1908255 | 4        |
| 5                | X004953          | 1908123 | 1        |
| 6                | X004965          | 1908256 | 1        |
| 7                | X004966          | 1908260 | 1        |
| 8                | X004968          | 1772700 | 2        |
| 9                | X004970          | 1908261 | 1        |
| 10               | Y017910 DP       | 17689.. | 1        |
| 11               | Y018191          | 1770454 | 1        |
| 12               | Y018965          | 1778254 | 1        |
| 13               | 150 A            | 1910488 | 1        |
| 14               | M5x12 DIN912     | 1001489 | 2        |
| 15               | M6x16 DIN912     | 1001492 | 1        |
| 16               | ---              | 1001606 | 2        |
| 17               | M4 DIN985        | 1001678 | 2        |
| 18               | M5 DIN985        | 1001679 | 2        |
| 19               | M6 DIN985        | 1001680 | 1        |
| 20               | M10 DIN985       | 1001683 | 1        |
| 21               | DIN127-B         | 1004414 | 3        |
| 22               | M5x10 ISO7380    | 1007063 | 4        |
| 23               | M6x16 ISO7380    | 1008459 | 4        |
| 25               | M6x10 ISO7380    | 1900641 | 3        |
| 26               | 60A (6 / 10 kph) | 1912294 | 1        |
|                  | 75A (14 kph)     | 1912302 | 1        |

| Bill of Material |                |         |          |
|------------------|----------------|---------|----------|
| No.              | Plan           | Article | Quantity |
| 27               | 2 ACT          | 1912304 | 1        |
|                  | 4 ACT          | 1912307 | 1        |
| 29*              | LA23           | 1906790 | 1        |
| 30               | ---            | 1912309 | 1        |
| 31               | M10x40 ISO7380 | 1992592 | 1        |
| 32               | M8x45 ISO7380  | 1994473 | 1        |
| 33               | M8x55 ISO7380  | 1995854 | 1        |
| 34               | 80 Ah          | 1909646 | 2        |
| 35               | M4x16 ISO7380  | 2990185 | 2        |
| 36               | ---            | 1990752 | 4        |
| 37*              | ---            | 1906791 | 1        |
| 38*              | ---            | 1906792 | 1        |
| 39               | M6 DIN125A     | 1000973 | 2        |
| 41               | M6 DIN127      | 1004414 | 4        |
| 42               | M6x20 ISO7380  | 1006999 | 4        |
| 44               | M6X50 DIN963   | 1001483 | 2        |
| 47**             | Y019503        | 1785354 | 1        |
| 48**             | Y019507        | 1785554 | 1        |
| 49**             | M10 DIN125     | 1001558 | 2        |
| 50**             | M8x35 ISO7380  | 1993236 | 1        |
| 51               | M5x40          | 1909530 | 2        |
| 52               | M5x15          | 1909529 | 4        |
| 53*              | not depicted   | 1906791 | 1        |
| 54*              | not depicted   | 1906792 | 1        |

\* For electric seat inclination

\*\* For manual seat angle

Sheet 3/3

|                                                                                                                                                                                                                                                               |            |            |            |                                                                                       |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------|------------|------------|---------------------------------------------------------------------------------------|
| NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP. |            |            |            | -                                                                                     |
| FRONT LOWER FRAME TIMIX                                                                                                                                                                                                                                       |            |            |            | Plan No.                                                                              |
|                                                                                                                                                                                                                                                               |            |            |            | E000823/ N                                                                            |
| Scale: 1:7                                                                                                                                                                                                                                                    | Drafted    | Verified   | Confirmed  |  |
| Date                                                                                                                                                                                                                                                          | 03/07/2012 | 10/06/2025 | 10/06/2025 |                                                                                       |
| Initials                                                                                                                                                                                                                                                      | JG         | AZ         | MK         |                                                                                       |