


Material			
No.	Plan	Article	Quantity
1	Y015294	1714801	2
2	X001471 (Black)	1001143	2
3	XP03211	2996925	1
4	XP03212	2996926	1
5	XC00423	3991837	2
6	XP00176	1824100	2
7	XP00180	1824500	2
8	XP00192	1826100	2
9	XP00208	2990139	2
10	XP00297	2990125	2
11	Cap D22	1990648	4
12	M4 DIN 985	1001678	2
13	M6x30 DIN 966	1007954	6
14	M4x12 DIN 7985H	1900952	2

Armrest Right

Armrest Left

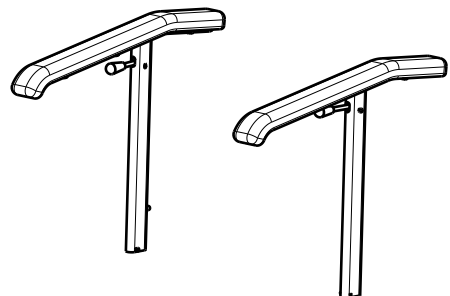

EP00534 / B

ARMREST / NORMANDIE 2020 / LANGUEDOC 2020 / PROVENCE 2020				Plan No. EP00534 / B
Scale: 1:5	Drafted	Verified	Confirmed	
Date	13/07/2020	16/04/2021	16/04/2021	
Initials	AZ	AZ	TK	