


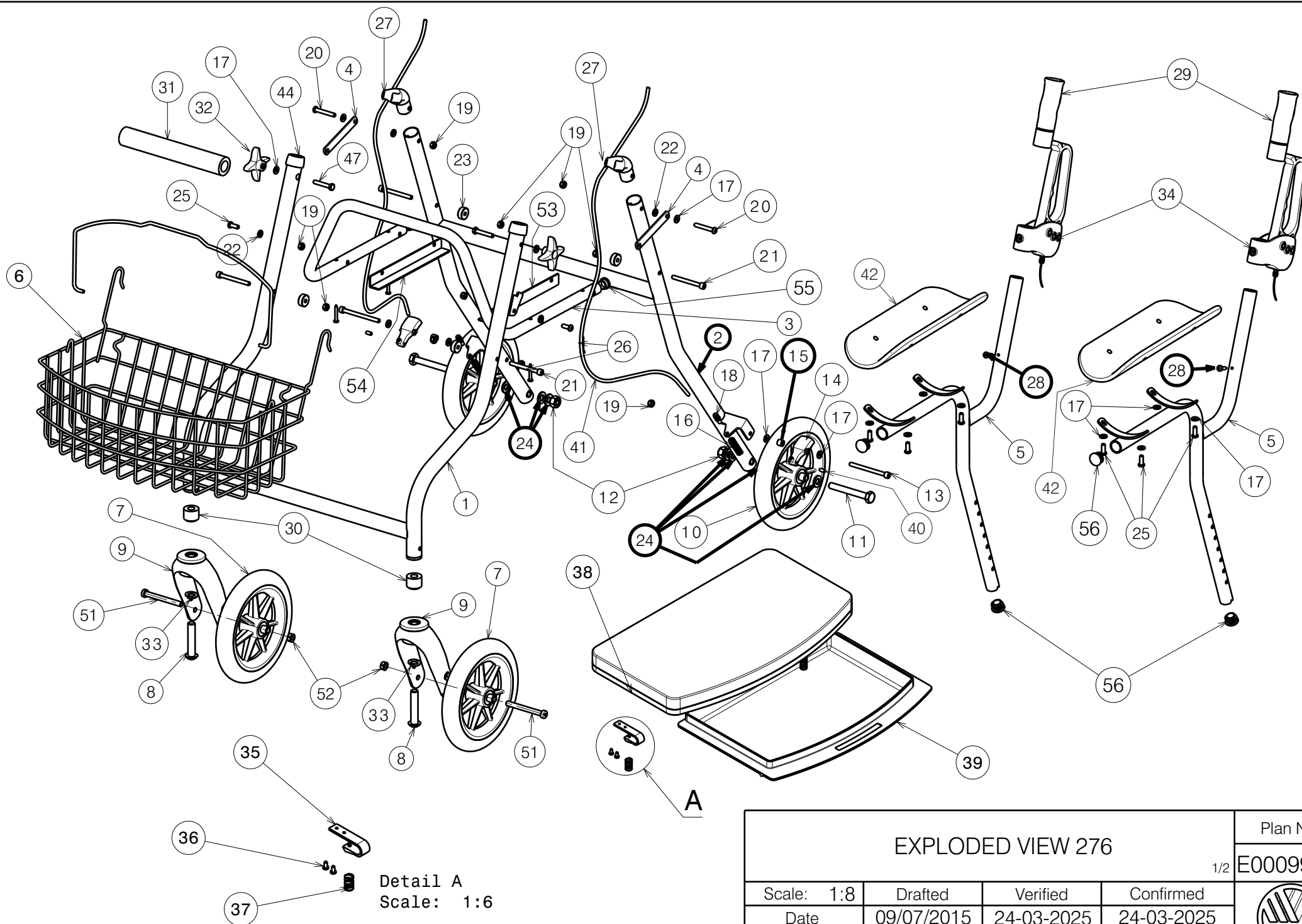

EXPLODED VIEW 276

Plan No.

1/2 E000994/M

Scale: 1:8

Drafted

Verified

Confirmed

Date

09/07/2015

24-03-2025

24-03-2025

Init.

AR

AZ

TK



Material				Material			
No.	Plan	Article	Quantity	No.	Plan	Article	Quantity
1	YP09307DP	26435...	1	32	Sterknop M6	2995334	2
2	YP00372	18117...	1	33	M12 DIN 127 A	1001572	2
3	Y004067	15072...	1	34	A00081572	1909113	1
4	Y004072	15073...	2	35	XP00071	2990040	2
5	YP09308DP	2643604	2	36	Screw D4x12	1990816	4
6	P00087176	1611704	1	37	X004865	1003901	2
7	Front wheels	1916560	2	38	P00087222	1994587	1
8	M12x60	1993419	2	39	X002335	1993411	1
9	Forks	1991447	2	40	M5x10 DIN 916/45H	1001485	2
10	Rear wheels	1914606	2	41	L=980mm	1003279	2
11	M10x80 DIN931	1001457	2		/	/	/
12	M10 DIN985	1001683	2	42	---	1996192	2
13	M6x75 DIN 912/8.8	1993413	2	43	---	---	---
14	X002306	1502304	2	44	XP00028	2990012	2
15	X003698	1004550	2	* 45	---	---	---
16	X002448	1517200	2	* 46	LOGO VERMEIREN	T001002	1
17	D6,4 DIN 125 A	1001564	16	47	M6x35 DIN931	2995333	2
18	---	1003558	2	* 48	---	W620150	2
19	M6 DIN985	1001680	10	* 49	---	W620151	2
20	M6x40	1006111	2	* 50	---	1003558	2
21	M6x60 DIN 912/8.8	1005957	4	51	M8x76 DIN912	1998993	2
22	M6 DIN125A	1000973	4	52	M8	1998994	2
23	XP00099	2990067	4	53	Y004199	1511304	1
24	D10,5 DIN 125 A	1001558	8	54	Y004200	1511404	1
25	M6x16	1008459	10	55	Cap D19	1001602	2
26	D4x30	1001545	4	56	Cap D22	1990648	4
27	X002305	1993120	2	*Not depicted			
28	D4.8X22 DIN 7981	2990070	2				
29	X005513	1992600	2				
30	XP00108	GP00604	2				
31	---	1993417	1				

EXPLODED VIEW 276

2/2

Scale:	Drafted	Verified	Confirmed
Date	09/07/2015	24-03-2025	24-03-2025
Init.	AR	AZ	TK

Plan No.
E000994/M