


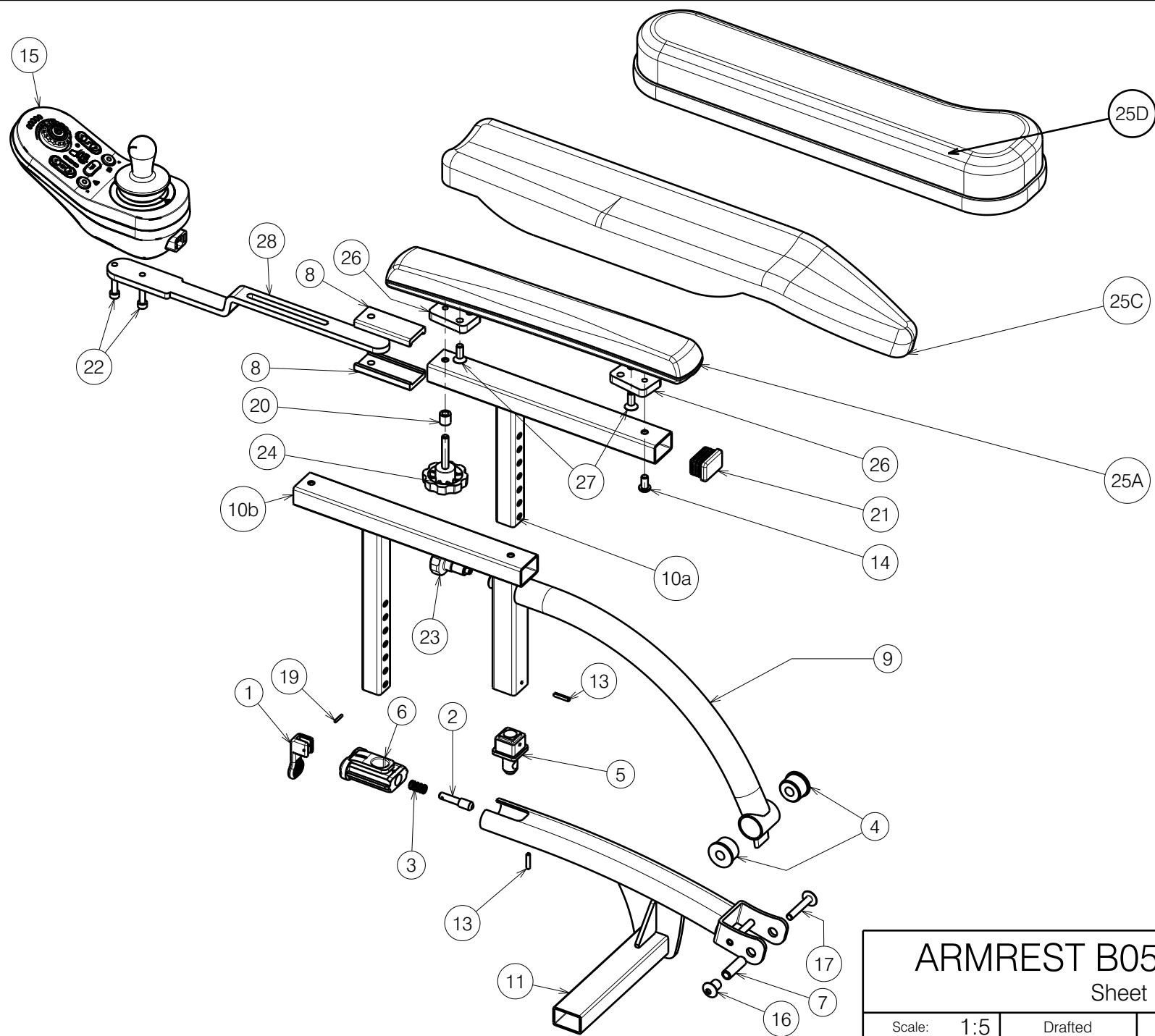

1906998

ARMREST B05 TYPE NAVIX

Sheet 1/2

Scale:	1:5	Drafted	Verified	Confirmed
Date	28/03/2011	03/12/2024	03/12/2024	
Initials	SD	AZ	MK	

Plan No.

E000703 / N



E000703 / N

Material				
No.	Plan	Article	Quantity	Remarks
1	X002132	1992423	2	
2	X002163	1488100	2	
3	X002169	1992791	2	
4	X004863	1906881	4	
5	X004869	1906883	2	
6	X004870	1906882	2	
7	X004890	1755000	2	
8	X004905	1907234	2	
9	Y017316	17493..	2	
10a	Y017318	1750454	2	For standard seat
10b	Y017995	1764654	2	For L15/L35
11	Y017336	17492.. (R)	1	
	Y017312	17490.. (L)	1	
13	Ø3x16 DIN1481	1003791	4	
14	M6x12 ISO7380	1007000	3	
15	---	1909020	1	LE drive only
	---	1910700	1	DLX drive only
	---	1912301	1	DLX drive + SE19 + 2 ACT
		1912296	1	DLX drive + SE19 + 5 ACT
		1912305	1	SE19 + 5 ACT
		1913523	1	LCD + joystick
16	M6 RT	1905500	2	
17	M6x35 KT	1906984	2	
18**	1,5 m	1912299	1	Not depicted
19	Ø2x12 DIN1475	1992597	2	
20	---	1996716	1	
21	---	1997922	3	
22	M5x16 DIN912	1007198	2	
23	M10x1	1999935	2	
24	M6x35	2990215	1	
25A	X004460	1904569 (L+R)	2	Standard right / left
25C	X003700	1998790 (R)	1	L30 PU right
	X003699	1998789 (L)	1	L30 PU left
25D	---	1914649 (R)	1	L31 Soft right
	---	1914648 (L)	1	L31 Soft left
26*	Y018001	1762554	4	Use with 1904569
27*	M6x16 DIN7991	1994752	4	Use with 1904569
28	Y019532	1785754	1	
29**	1 m	1909022	1	Not depicted

Notes: Only right side depicted,
quantities are for complete set (L&R)
* Only with 1904569
** Not depicted

1906998

ARMREST B05 TYPE NAVIX				Plan No.
Sheet 2/2				E000703/ N
Scale: 1:5	Drafted	Verified	Confirmed	
Date	28/03/2011	03/12/2024	03/12/2024	
Initials	SD	AZ	MK	