


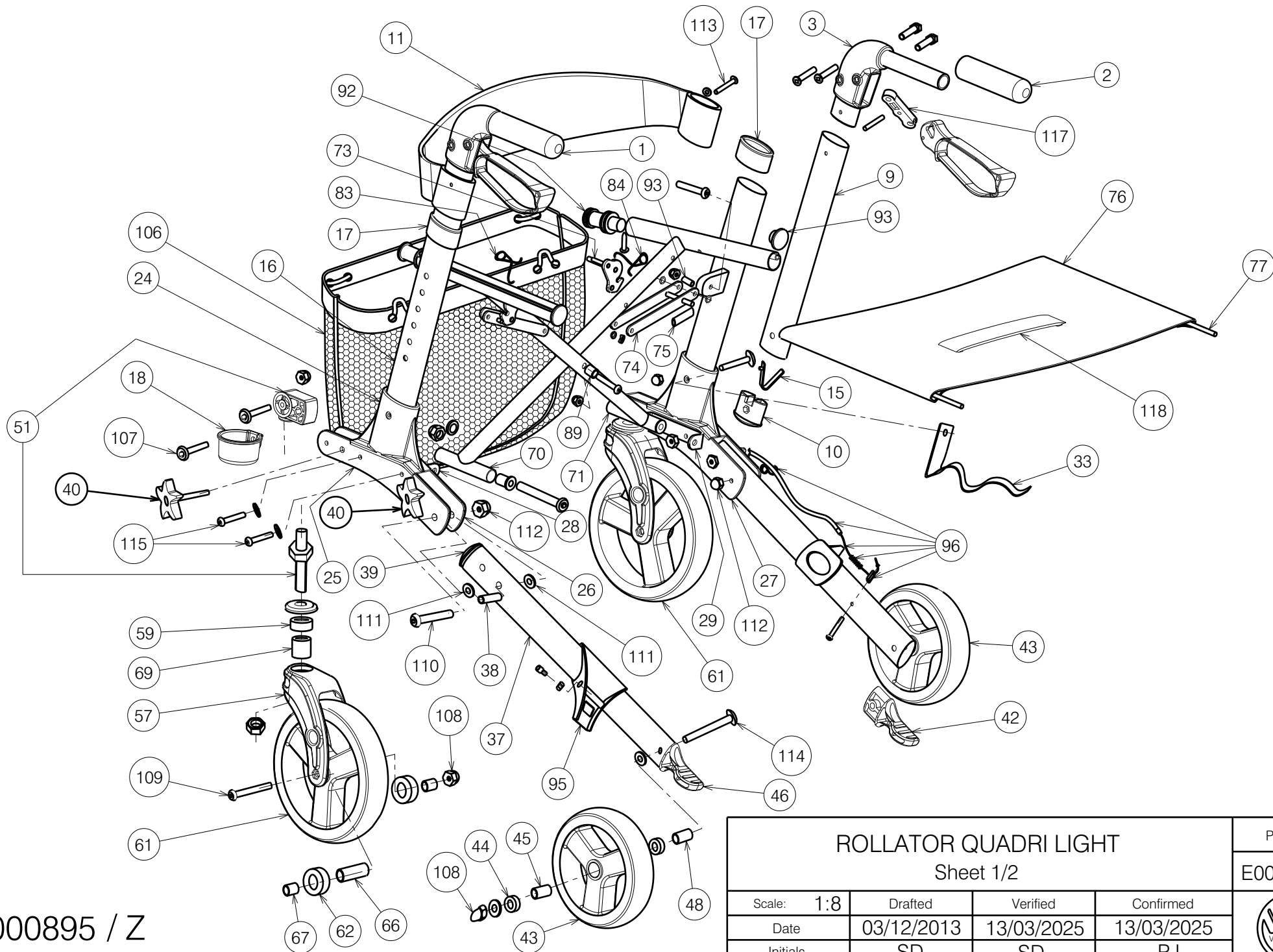

E000895 / Z

8002320

ROLLATOR QUADRI LIGHT Sheet 1/2

Plan No.

E000895 / Z

Scale: 1:8

Drafted

Verified

Confirmed

Date

03/12/2013

13/03/2025

13/03/2025

Initials

SD

SD

PJ



Bill of Material				Bill of Material				Bill of Material			
No.	Plan	Article	Quantity	No.	Plan	Article	Quantity	No.	Plan	Article	Quantity
1	Left	1904436	1	43a	---	1909858	2	73	---	1909882	1
2	Right	1904435	1	43b	---	1910687	2	74	---	1909883	2
3	---	1909823	2	43c	---	1915971	2	75	---	1909892	4
9	---	1910437	2	44	608Z	1909871	4	76	---	1909893	1
10	---	1909824	2	45	---	1909862	2	77	---	1909894	2
11a	---	1909825	1	46	---	1917786 (L)	1	83	Left	1909945	1
11b	---	1912732	1	48	---	1909860	2	84	Right	1909946	1
15	---	1909826	2	51	---	1919790	2	89	---	1909895	1
16	---	1910438	2	57	---	1909874	2	92	---	1909896	2
17	---	1916432	2	58	---	1909876	2	93	---	1909897	2
18	---	1915016	1	59	---	1992559	4	95	---	1909898	2
24	---	1909829	2	61a	---	1909859	2	96	---	1909899	2
25	---	1909846	2	61b	---	1910688	2	106	---	1909901	1
26	---	1909847	1	61c	---	1915971	2	107	M5x50 ISO7380	1910277	1
27	---	1909848	1	62	608RS	1909872	4	108	M8 DIN986	1902024	4
28	---	1909849	1	66	---	1909862	2	109	M8x64	1911162	2
29	---	1909853	1	67	---	1909863	4	110	M6x40 ISO7380	1006111	2
33	---	1909854	2	69	---	1909878	2	111	M6 DIN125	1000973	4
37	---	1910439	2	70	---	1909879	1	112	M6 DIN986	1008492	4
38	---	1909855	2	71	---	1909880	1	113	M5x32	1902835	2
39	---	1909824	2	72	---	1909881	1	114	M8x88	1911161	2
40	M6	1909856	4					115	M6x42	1912333	4
42	---	1917787 (R)	1					117	---	1916122	2
								118	---	1910787	1

E000895 / Z

Notes

11a = Old
11b = New (from 31/05/2016)

43a = Old
43b = from 11/2014
43c = from SN D1917469

61a = Old
61b = from 11/2014
61c = from SN D1917469

42 = from H1930256
46 = from H1930256

ROLLATOR QUADRI LIGHT

Sheet 2/2

Scale: 1:8

Drafted

Verified

Confirmed

Date

03/12/2013

13/03/2025

13/03/2025

Initials

SD

SD

PJ

Plan No.

E000895 / Z



8002320