



rev 17.01.2025
EV ONE LU

11.05.2026

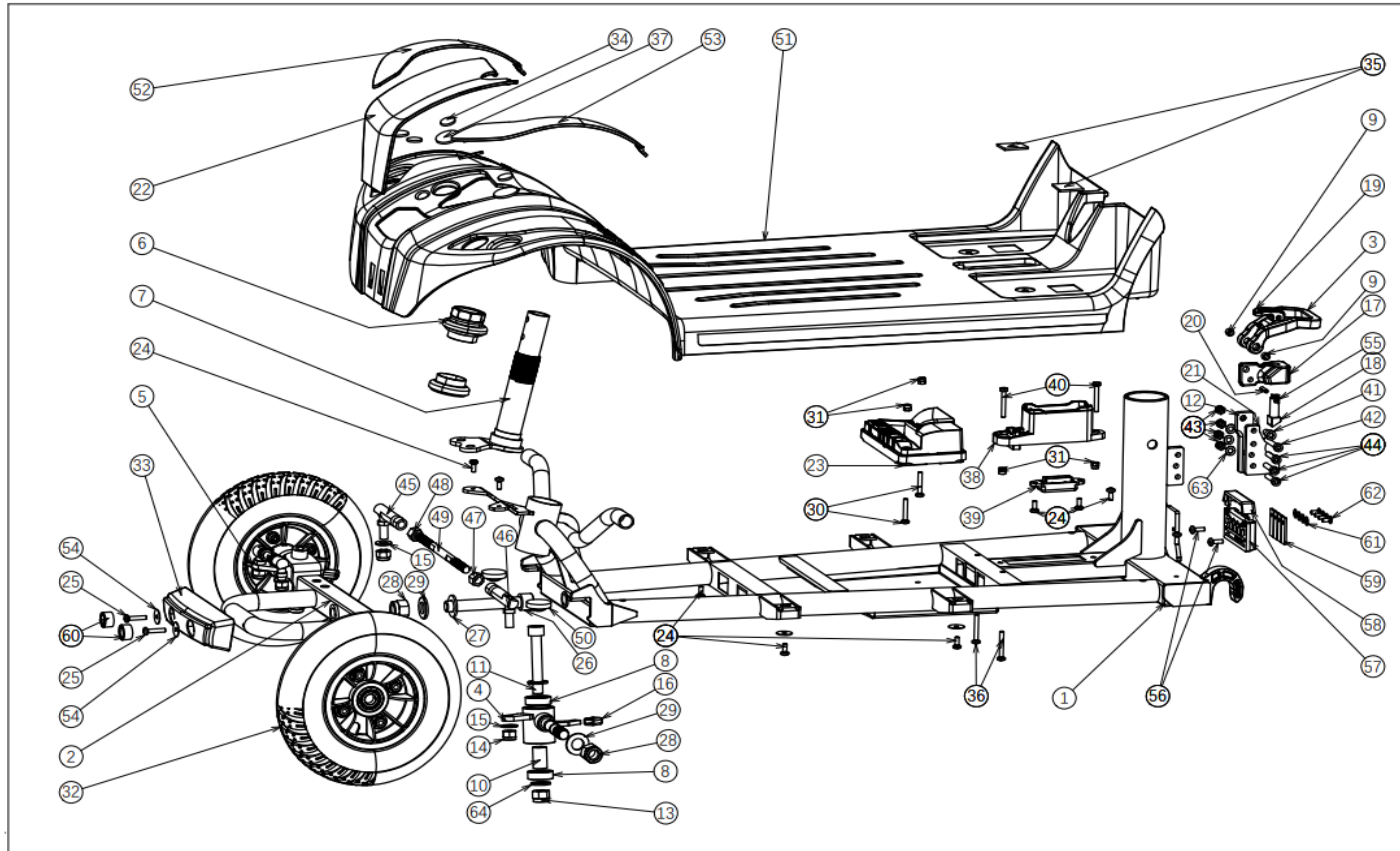


no	part nu	part	QT	no	part nu	part	QT
1	1920656	MAIN FRAME	1	36			
2	1918862	FRONT FORK	1	37			
3				38	1920657	BATTERY CONNECTOR HOLDER 1	1
4	1920340	FRONT AXLE L		39	1920658	BATTERY CONNECTOR HOLDER	1
5	1920341	FRONT AXLE R			1922814	BATTERY CONNECTOR +CABLE	1
6				40			
7	1918856	STEERING AXLE ONE	1	41			
7				42			
7				43			
7				44	1918857	BALL JOINT LE	
8				46	1918858	BALL JOINT RIGHT	
8				45			
8				47			
8				48	1918859	CONNECTION ROD	
9			1	49			
10				50	1918851	FOOT PLATE	1
11				51	1920443	SET COLOR PANEL FRONT green	
				52	1920444	SET COLOR PANEL FRONT orange	
				52	1920445	SET COLOR PANEL FRONT Blue	
				52	1920446	SET COLOR PANEL FRONT pink	
			1	52			
13				53	1920634	WASHER M4 NF E25-513	
14				54			
15			2	55			
16				56			
17				57	1918861	FRONT CONTACT POINT HOLDER	1
18			2	58			
19				59	1918860	CONTACT POINT FRONT	4
20				60			
21				61			
22	1918852	FRONT COVER	1	62			
23	1919090	POWER MODULE	1	59	1920635	BOLT CAP	6


24			1				
25	1920633	BOLT M4X20 DIN 7985	13				
26							
27							
28							
29							
30							
31							
32	1922939	FRONTWHEEL ONE AIR	2				
33	1920632	FRONT BUMPER	1				
34							
35							

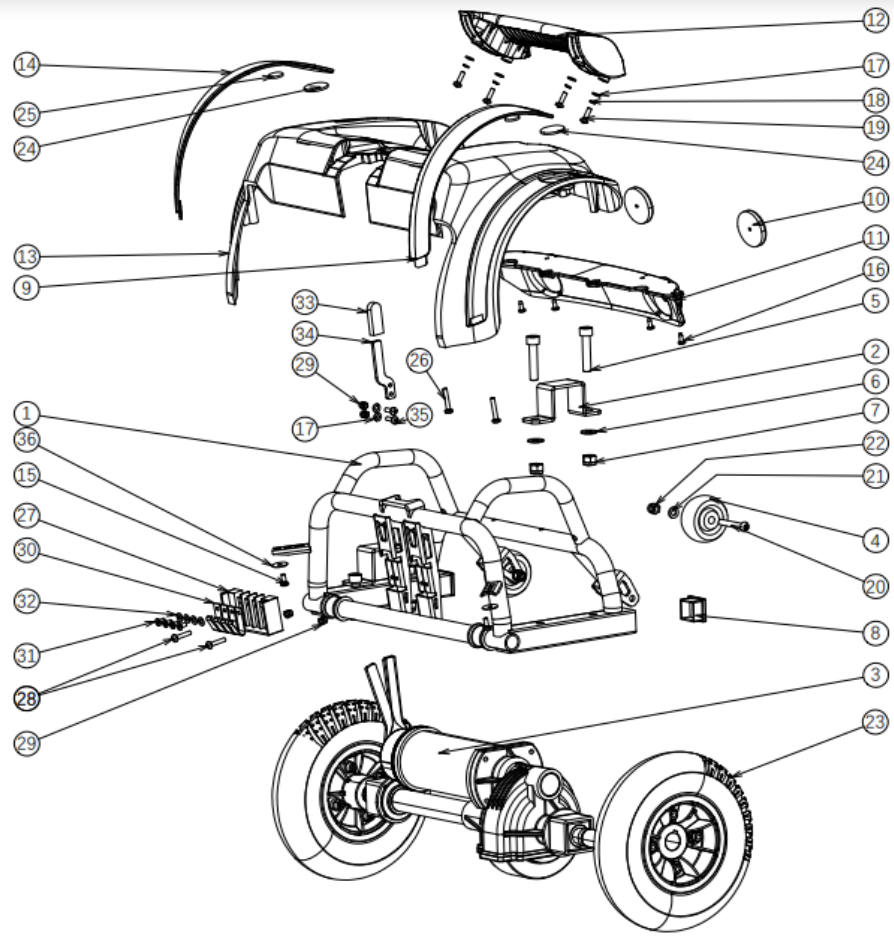


no	part nu	part	QT		part nu	part	QT	
0	1920394	TIRE	4					
0	1001263	INNER TIRE	4		1922206	SCREW ST3,5x9,5	4	
1					16	1922402	WASHER M4	4
2					17	1922403	SPRING WASHER	4
3					18	1922404	SCREW M4	4
3	1918887	DIFF.+MOTOR+REM	1		19	1922549	BOLT ANTI TIP	2
3	1920395	DIFF	1		20			
not depicted	1922014	BUSH TRANSMISION .	1					
3	1920396	MOTOR	1					
3	1920397	MOTORBRAKE	1					
4	1922523	WHEEL ANTI TIPPING	2		1922550	SPRING WASHER	2	
4					21	1922551	LOCK NUT	2
4					22			
4					23	1922938	REAR WHEEL LU	2
5					24			
6					25			
7					26		1	
8					27	1918889	REAR CONTACT POINT HOLDER	1
9					28			
10	1921454	REAR RED REFLECTOR	2		30	1918890	CONTACT POINT REAR	4
11	1918882	REAR COVER	1					
12	1918886	HANDLE REAR	1		31			
13	1918853	COVER MIDDLE	1		32			
14	1920439	COLOR PANEL SET L/R green			33			
14	1920440	COLOR PANEL SET L/R orange			34			
14	1920441	COLOR PANEL SET L/R Pink						
14	1920442	COLOR PANEL SET L/R blue						
15								
16					35			
17	1922190	RED MOTORBRAKE LEVER COVER	1		36			
not depicted	1923143	AXLE KEY	1					

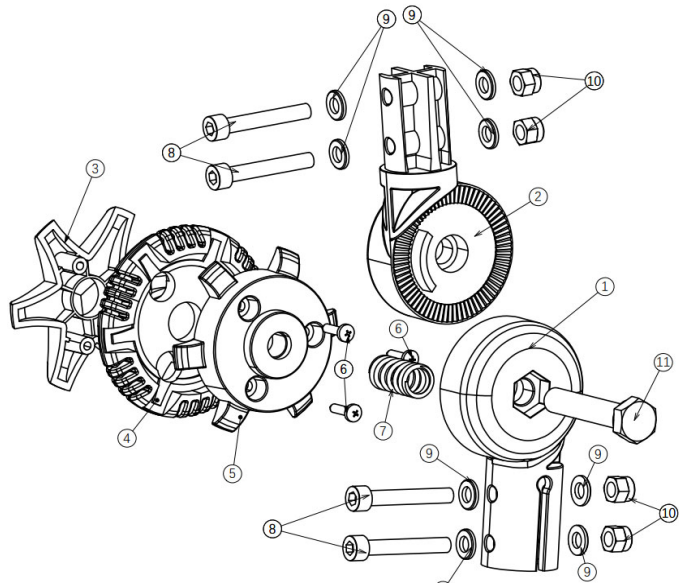

no	part nu	part	QT			
1	1921317	lower parrt angle adujstment				
2						
3	1921320	outer knob angle adjust	1			
4	1921319	middle knob angle adjust	1			
5	1921318	inner knob angle adjust	1			
6	1921322	bolt M5 X 16				
7	1921321	spring knob angle adjust	1			
8						
9						
10						
11	1922522	bolt angle adjust M10 X55	1			
not depicted	1922514	Set 1/12	1			

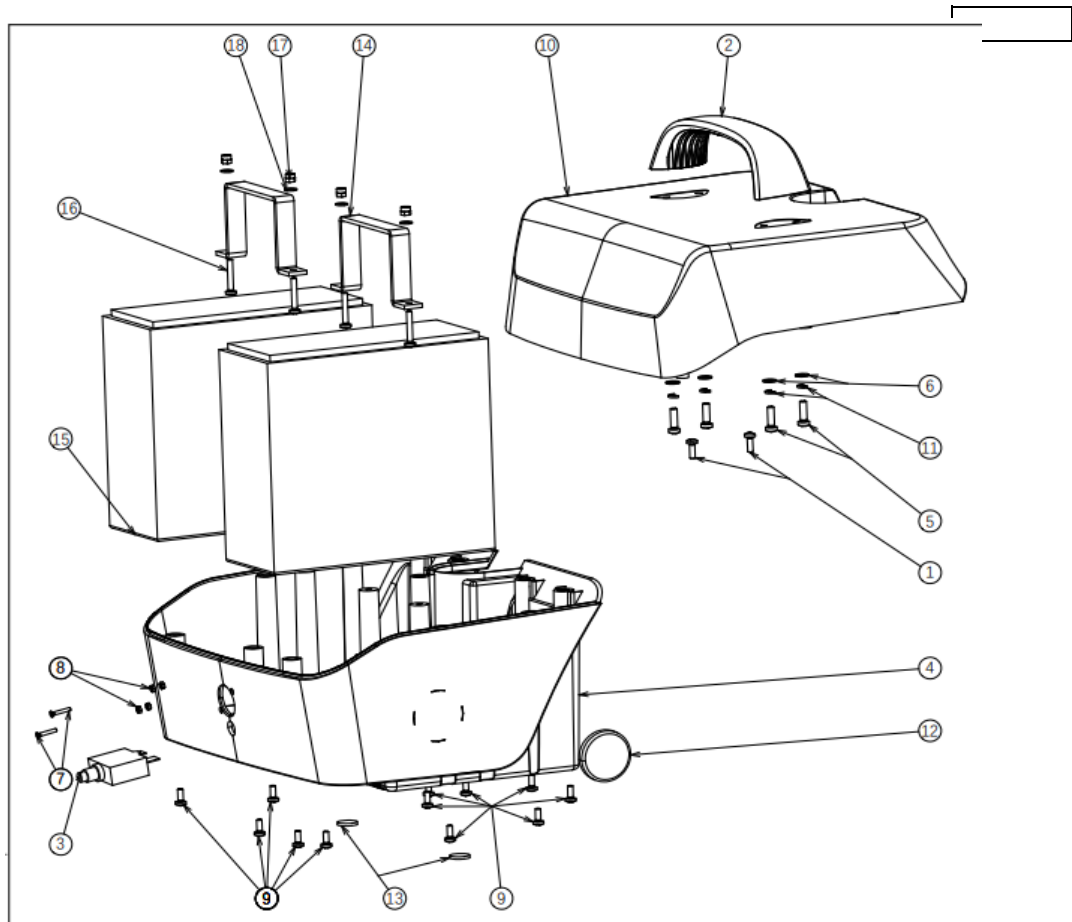


no	part nu	part	QT			
0	1920391	BATTERYBOX 25 AHCOMPLETE	1			
0	1920346	BATTERY BOX 25AH WITHOUT BATTERIES	1			
1						
2						
3	1920531	MAIN FUSE				
4						
5						
6						
7						
8						
9	1922784	Screw M4x10	5			
10						
11						
12						
13						
14						
15	1920301	BATTERY 25 A/H	2			
16						
17						
18						
	1922548	CHARGER 3 amp	optie			
0	1919798	CHARGER 2A	1			
not depicted	1922548	CHARGER 3 A (option)	0			

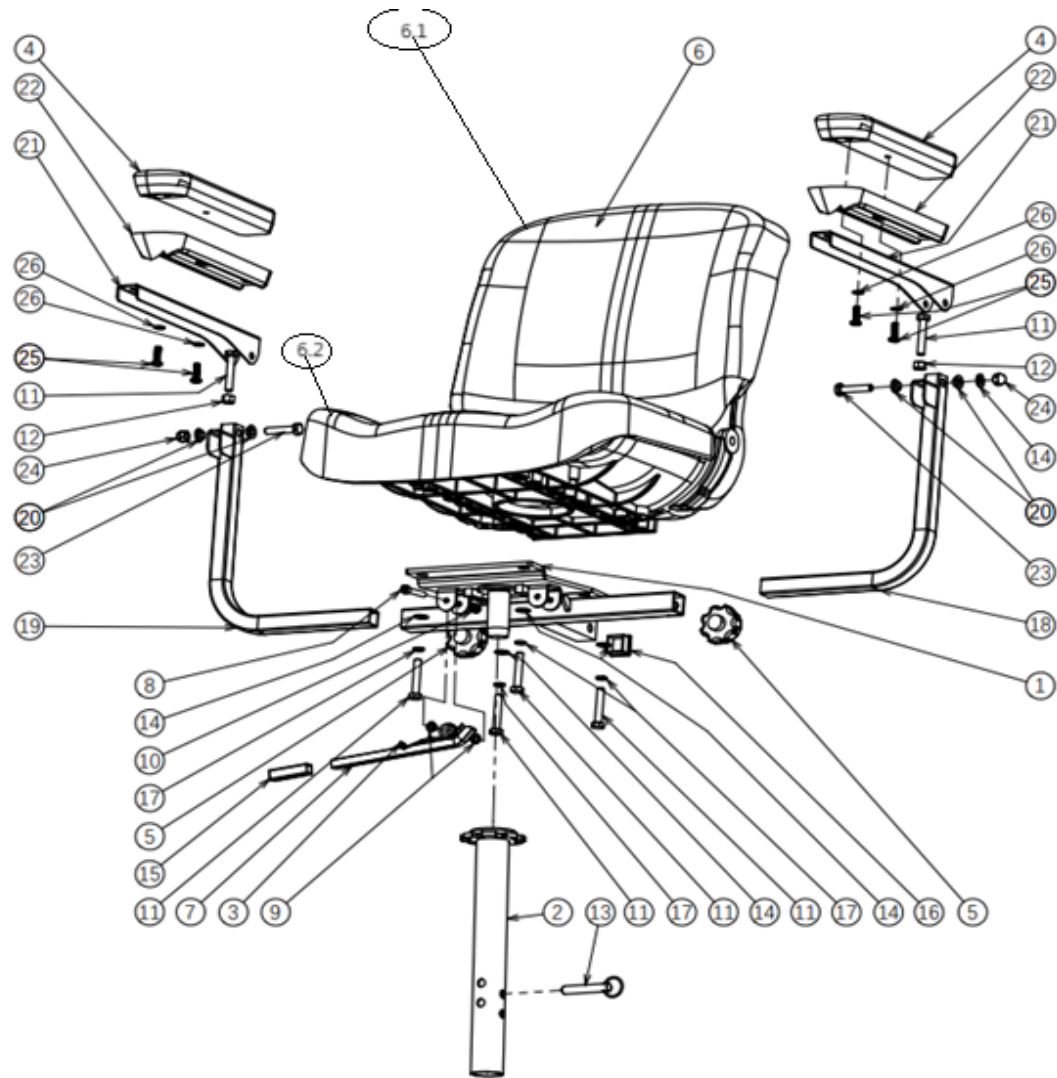


no	part nu	part	QT			
0						
1						
2	1918898	SEAT TUBE	1			
3						
4	1918895	ARM PAD	2			
5	1920266	BOLT ARMREST FIXATION	2			
6	1918899	CHAIR	1			
6.1	1920640	BACK	1			
6.2	1920641	SEAT				
7	1920645	SEAT HANDLE	1			
not depicted	1923200	SPRING SEAT HANDLE	1			
8						
8						
9						
10						
11	1924043	BOUT M8 X 45 DIN931	4			
12						
13						
14	1924044	RONDSEL M8 DIN126	4			
15						
16						
17	1924045	RONDSEL VEER M8DIN127B	4			
18	1918903	ARMREST LOWER LEFT	1			
19	1918907	ARMREST LOWER RIGHT	1			
20						
21	1918931	ARMRESR UPPER PART	2			
22						
23						
24						
	1920830	ARM REST COMPLETE L	1			
	1920831	ARM REST COMPLETE R	1			

1921438 SEAT PLATE COMPL W/A tube

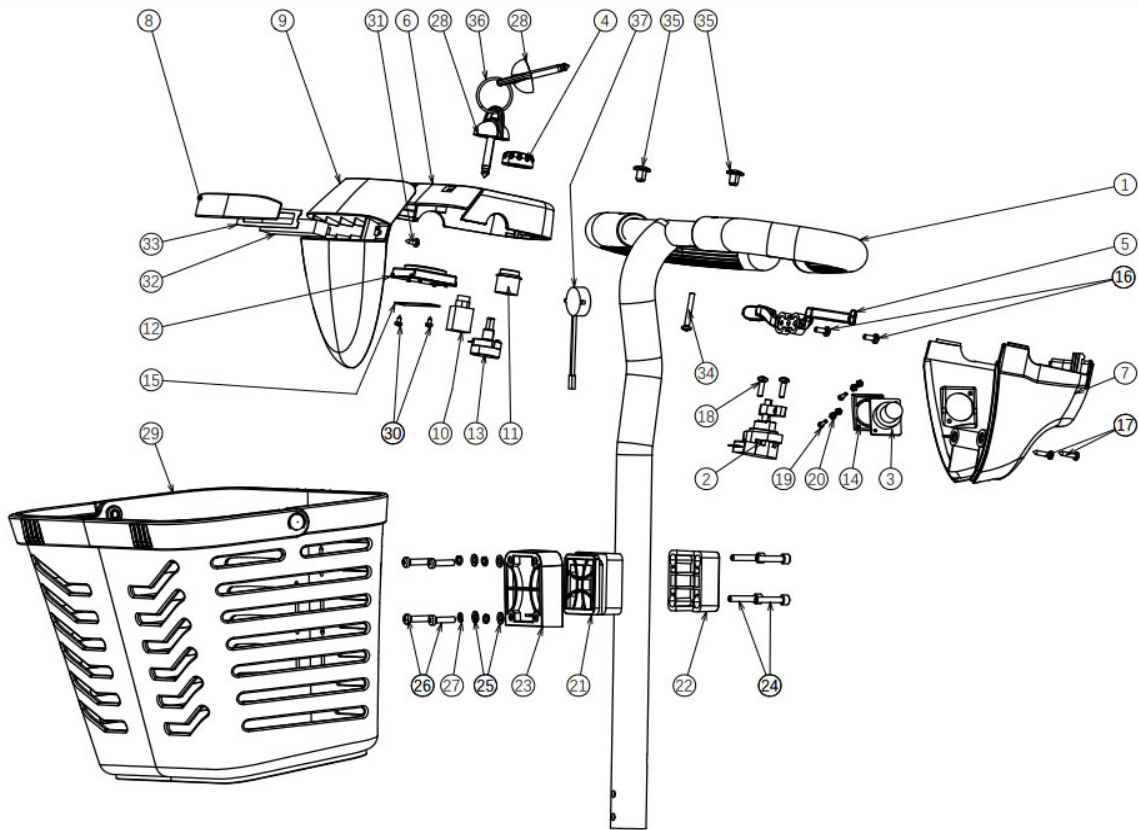

no	part nu	part	QT
1			
2	1918911	SPEED DIAL ASSEMBLY	1
3	1918918	CHARGER SOCKET	1
4			
5	1918910	SPEED LEVER	1
6	1923868	TOP COVER	1

7	1918913	STEERING COVER BACK	1
8	1921463	FRONT LIGHT COVER	1
9	1918914	STEERING COVER FRONT	1
10	1919437	CONTACT IGNITION	1
11	1920423	HORN SWITCH	1
12	1918912	BATTERY INDICATOR	1
13	1918909	POT SPEED	1
14			
15			
16			
17			
18			
19			
20			
21	1920502	ASSEMBLY PLATE BASKET MIDDLE	1
22	1920510	ASSEMBLY PLATE BASKET REAR	1
23	1920501	ASSEMBLY PLATE BASKET FRONT	1
	1920832	BASKET HOLDER COMPLETE	
24	1920511	BOLT M5X25	2
25	1920503	WASHER	4
26	1920505	BOLT M5X20	4
27			
28	1918908	IGNITION KEY	1
29	1920509	BASKET	1
30			
31			
32	1920629	FRONT LIGHT	1
33			
34			
35			
	3990651	MIRROR LEFT	1
	3990652	MIRROR RICHT	1
37	1920628	HORN	1

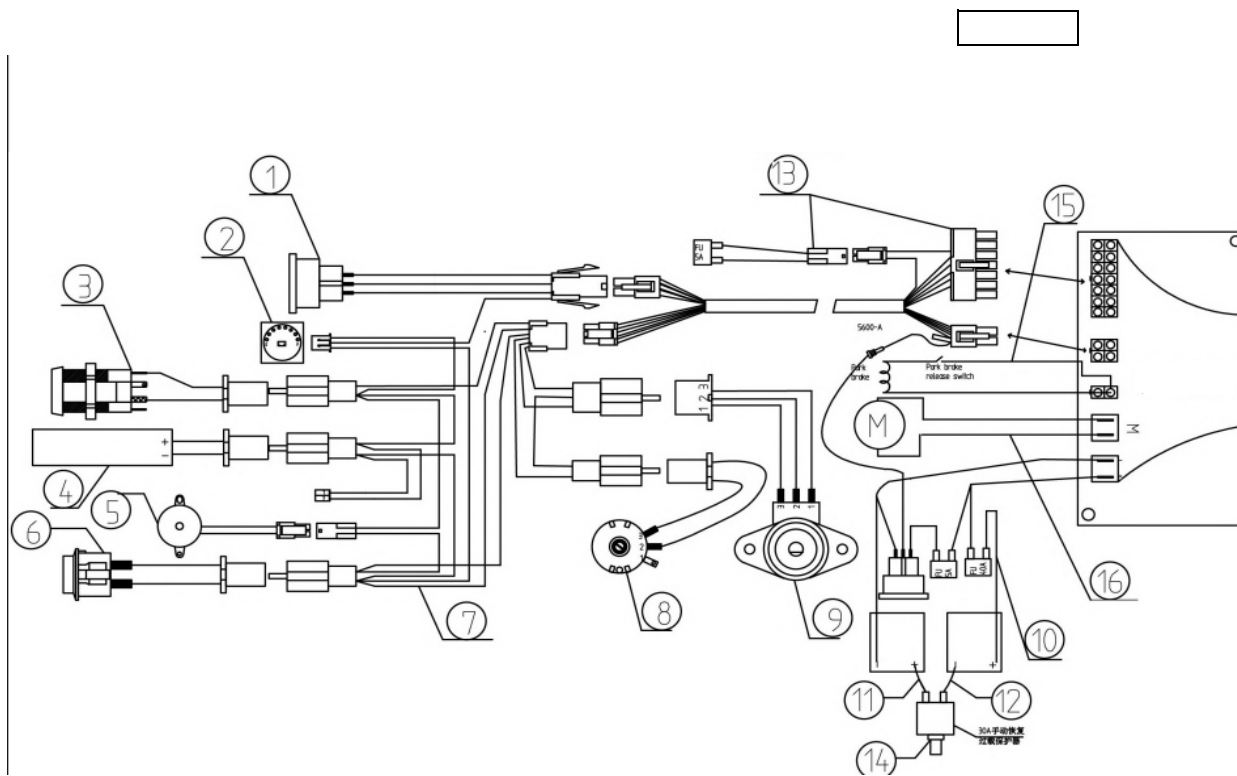

no	part nu	part	QT
1	1918918	CHARGER SOCKET	1
2			
3			
4			
5			
6			
7	1918976	SHORT CABLE TILLER	1
8			
9	1918909	POT. SPEED	1

10	1921636	CABLE+ FUSE	1	
11				
12				
13	1922401	MAIN CABLE	1	
14				
15				
16				