


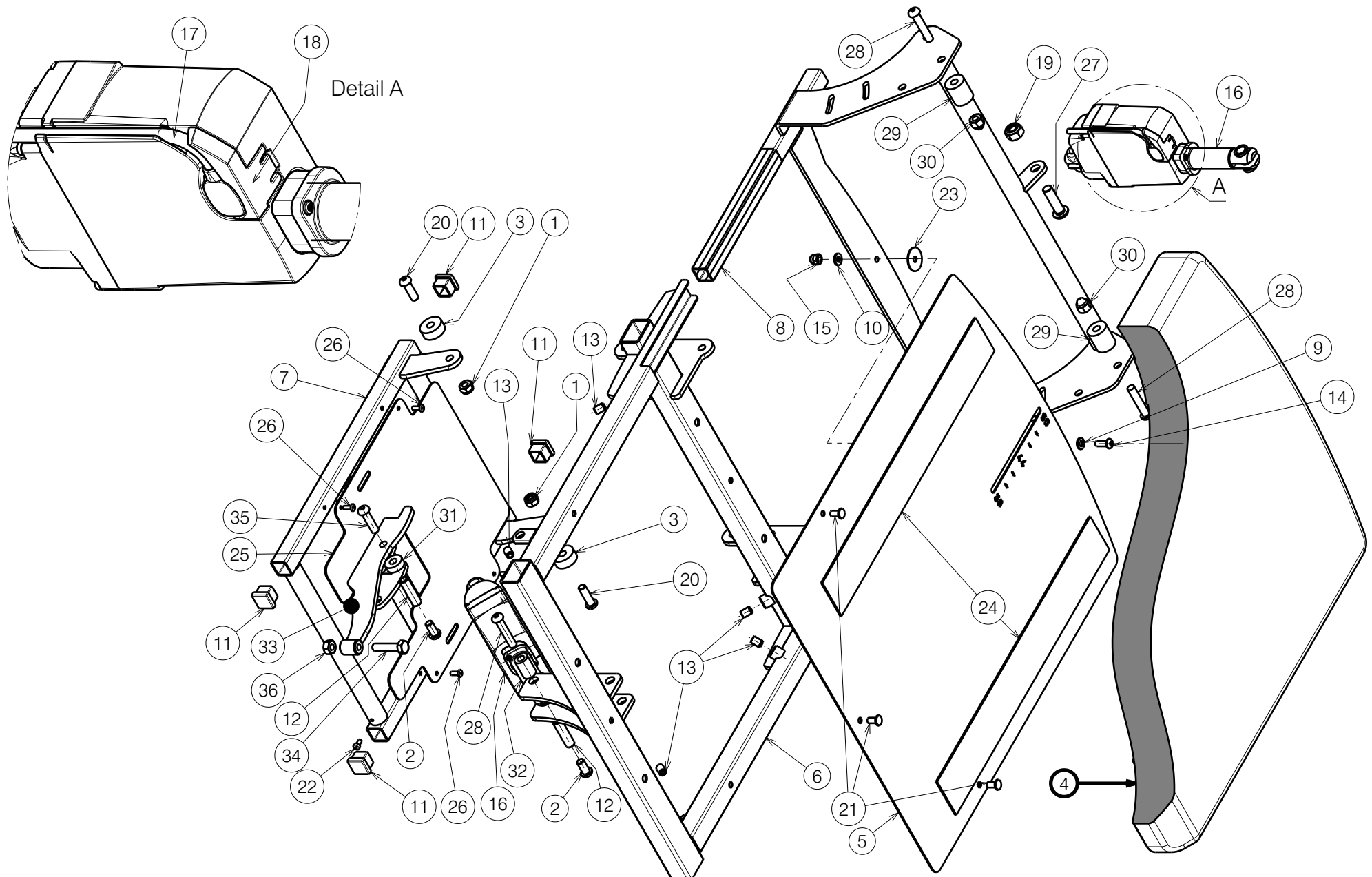

NAVIX SEAT FRAME B23 ELECTRIC

Sheet 1/2

Plan No.

E000697 / I

Scale: 9:50

Drafted

Verified

Confirmed

Date

03/03/2011

28/11/2024

28/11/2024

Initials

JG

AZ

MK



E000697 / I

Material							
No.	Plan	Article	Quantity	No.	Plan	Article	Quantity
1	X001016	1001682	2	16	---	1906790	2
2	M8x16 ISO7380	1991858	2	17	---	1906791	2
3	X004048	1902685	2	18	---	1906792	2
4	X004860	1906817 (40)	1	19	M10 DIN985	1001683	1
	X004855	1906741 (45)	1	20	M8x25 ISO7380	1990501	2
	X004887	1906818 (50)	1	21	---	1992835	3
5	Y017269	1751654 (40)	1	22	M4X08 DIN7984	1996530	1
	Y017280	1748854 (45)	1	23	X003036	1997486	1
	Y017375	1751554 (50)	1	24	---	2990413	(2X) 0.4m
6	Y017305	1751954 (40)	1	25	Y017643	1754754	1
	Y017285	1748754 (45)	1	26	---	1001583	4
	Y017282	1751354 (50)	1	27	M10x35 ISO7380	1009442	1
7	Y017331	1748954	1	28	M8x40 ISO7380	1008739	3
8	Y016954	1752354 (40)	1	29	X003111	1995853	2
	Y017347	1750154 (45)	1	30	M8	1902024	2
	Y016960	1752454 (50)	1	31	X005383	1911710	1
9	M6 DIN125 A	1000973	1	32	X005384	2014400	1
10	M6 DIN125 A	1001564	1	33	Y021834	2017254	1
11	---	1001612	4	34	M8x35 DIN933	1001444	1
12	X004804	1754800	2	35	M8x30 ISO7380	1992717	1
13	M8x12 DIN916	1004885	6	36	M8 DIN 934	1001671	1
14	M6x16 ISO7380	1008459	1				
15	M6	1008492	1				

E000697 / I

NAVIX SEAT FRAME B23 ELECTRIC				Plan No.
Sheet 2/2				E000697 / I
Scale: 9:50	Drafted	Verified	Confirmed	
Date	03/03/2011	28/11/2024	28/11/2024	
Initials	JG	AZ	MK	