


EP00488 / SE53 ARMREST

No.	Plan	Article	Qty
1	P00059612	2677254	1
2	M5x16 DIN912	1007198	2
3	M4x10 ISO7380	1998758	2
4	SE64	1913686	1

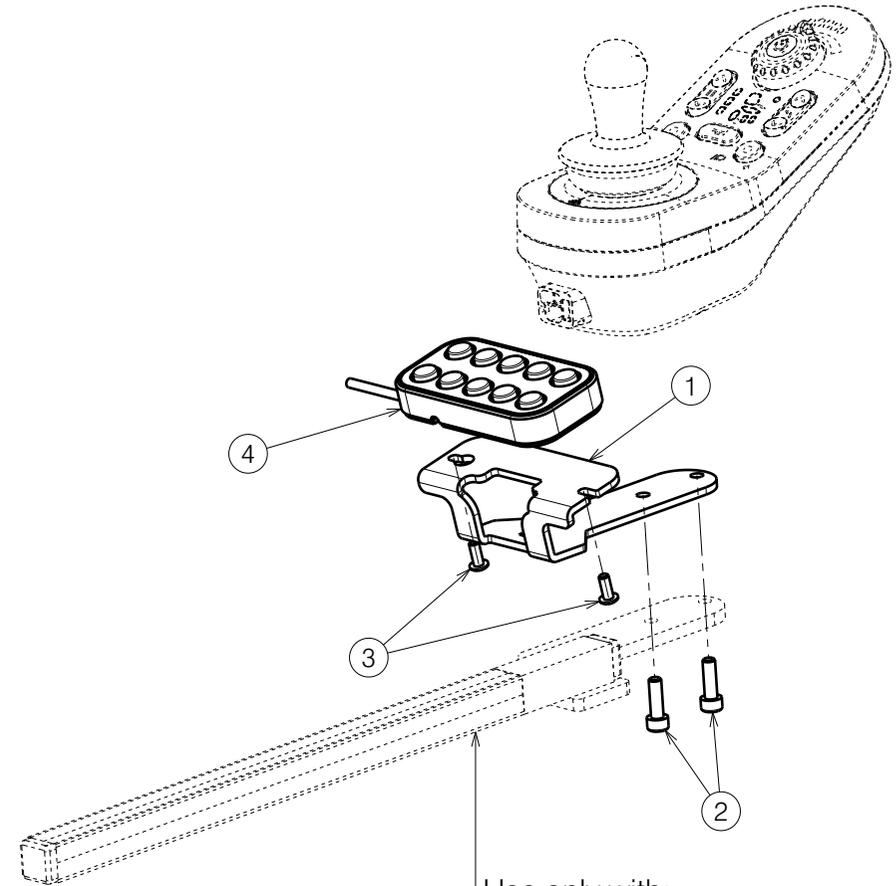


Use only with:  
 A) YP08058/26054 (R)  
 B) YP08059/26055 (L)

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.


\* - only right side depicted

SE64 OPTION / TURIOS				---
				Plan No. E00061546/A.2
Scale: 1:3	Drafted	Verified	Confirmed	
Date	16.06.2023	04/07/2023	04/07/2023	
Initials	AZ	MVG	MK	

No.	Plan	Article	Qty
1	P00059612	2677254	1
2	M5x16 DIN912	1007198	2
3	M4x10 ISO7380	1998758	2
4	SE64	1913686	1

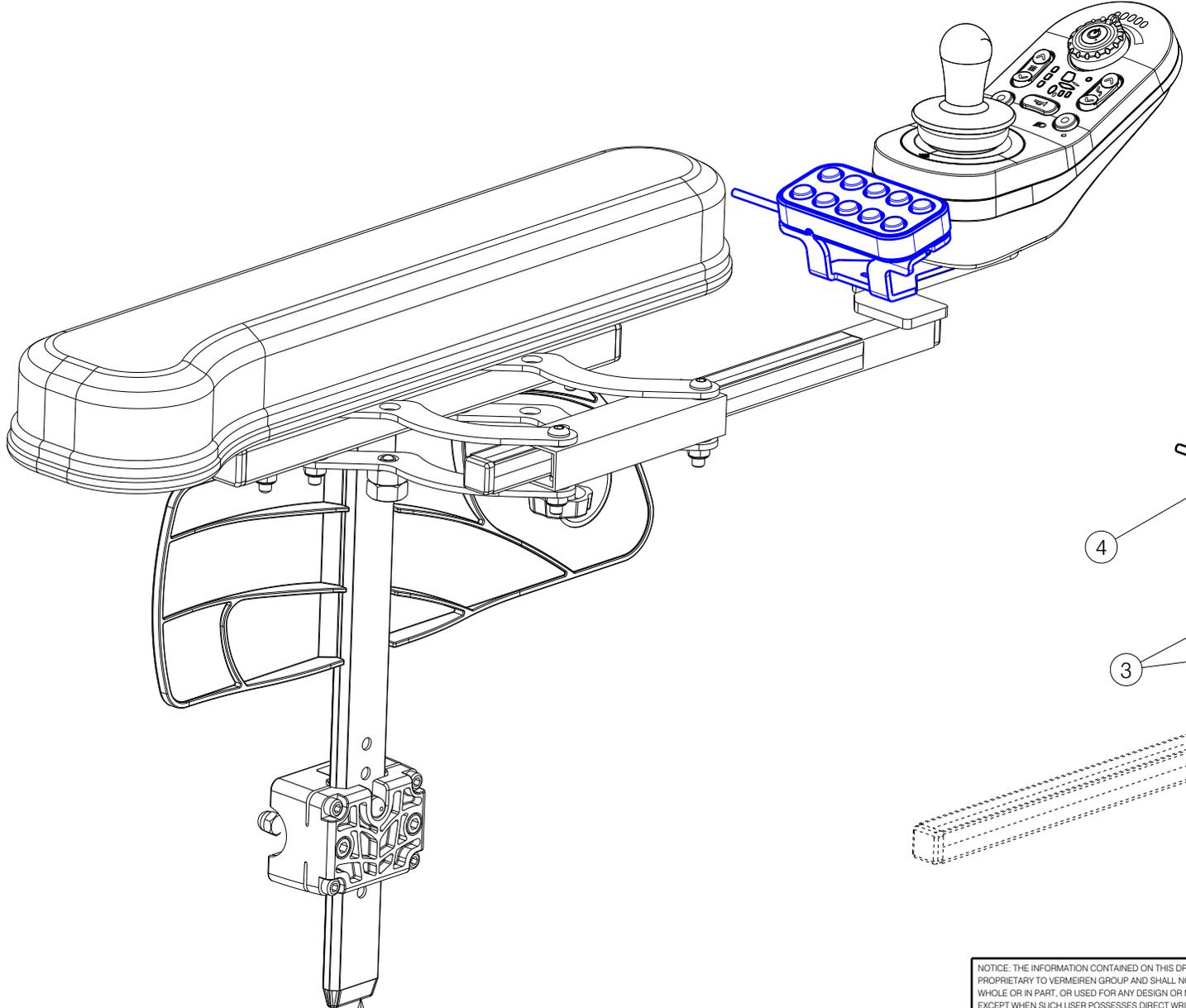

EP00489 / SE53 WIDE ARMREST

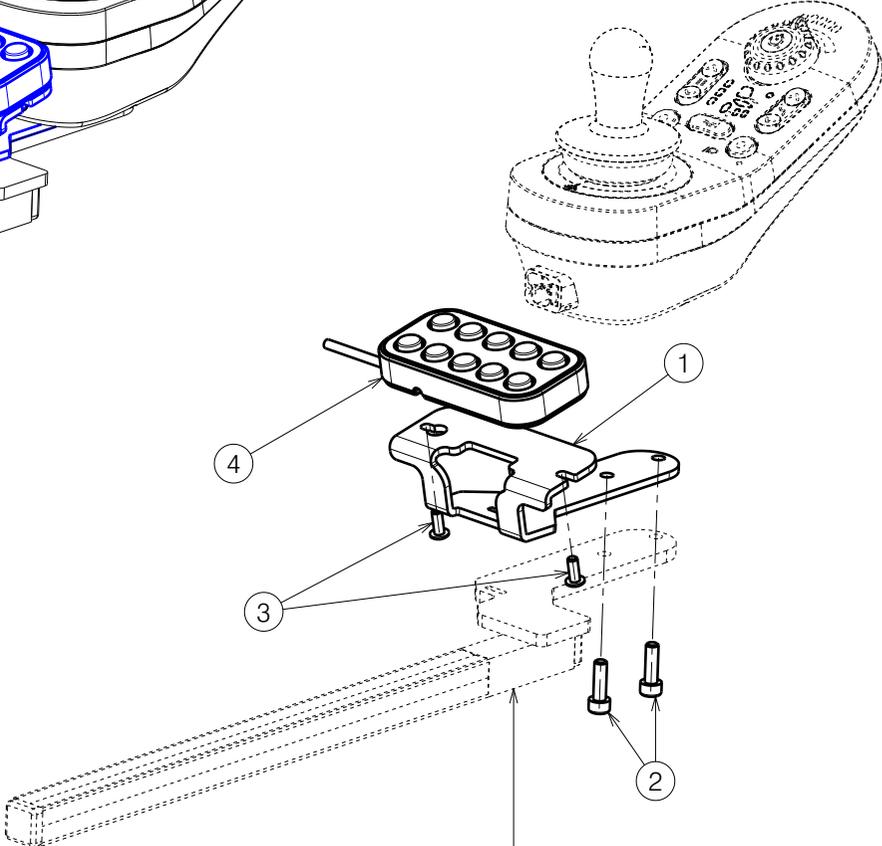

Use only with:  
 A) YP08058/26054 (L)  
 B) YP08059/26055 (R)


NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

<b>SE64 OPTION / TURIOS</b>				---
				Plan No.
				E00061546/A.2
Scale:	1:3	Drafted	Verified	Confirmed
Date	16.06.2023	04/07/2023	04/07/2023	
Initials	AZ	MVG	MK	

---

2/3

\* - only right side depicted

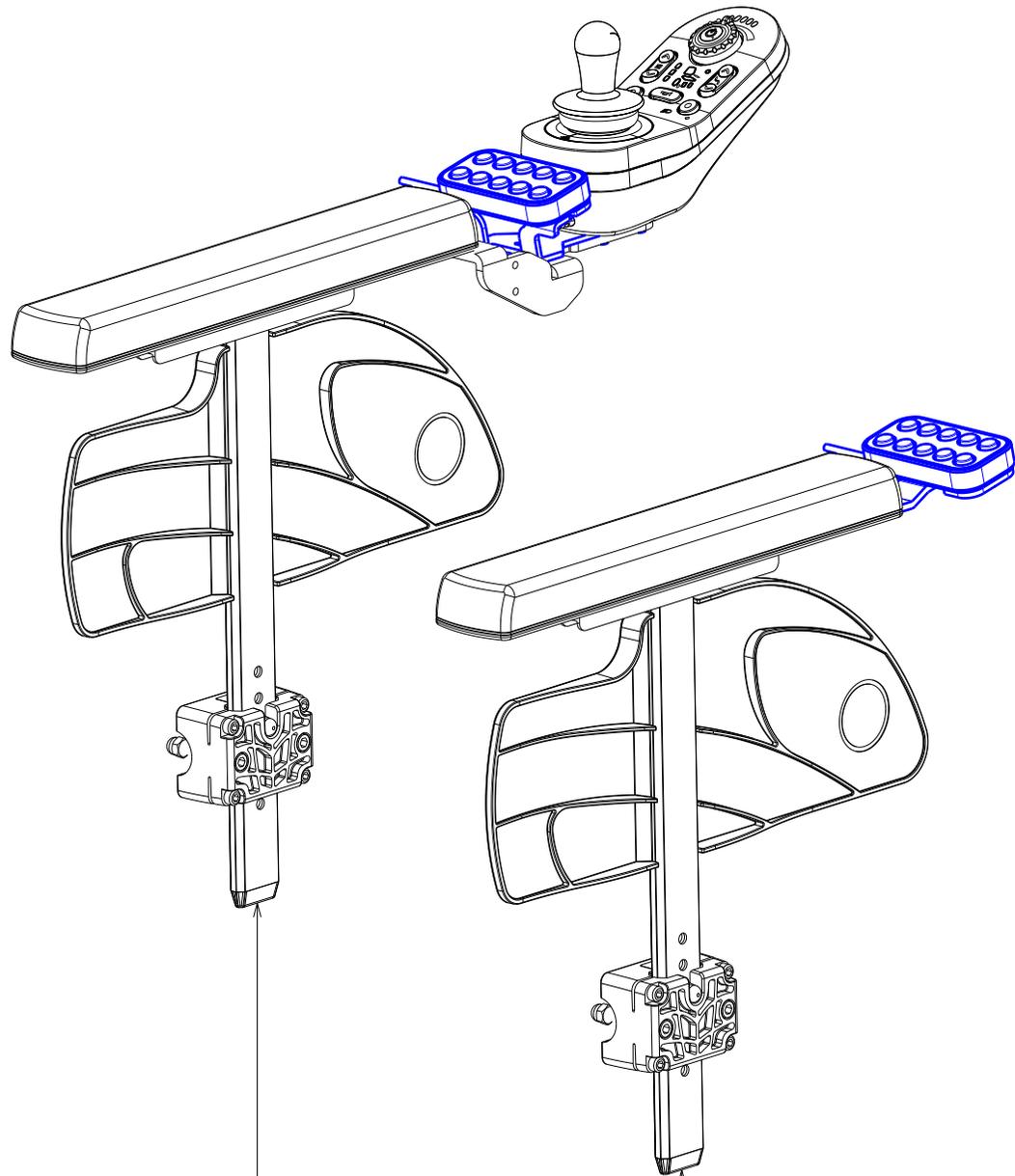


EP00450 / BI5 TYPE SQUOD /  
WITH REMOTE CONTROLE

EP00450 / BI5 TYPE SQUOD /  
WITHOUT REMOTE CONTROLE

\* - only right side depicted

No.	Plan	Article	Qty
1	P00059612	2677254	1
2	P00061683	2677354	1
3	M5x16 DIN912	1007198	2
4	M4x10 ISO7380	1998758	2
5	SE64	1913686	1

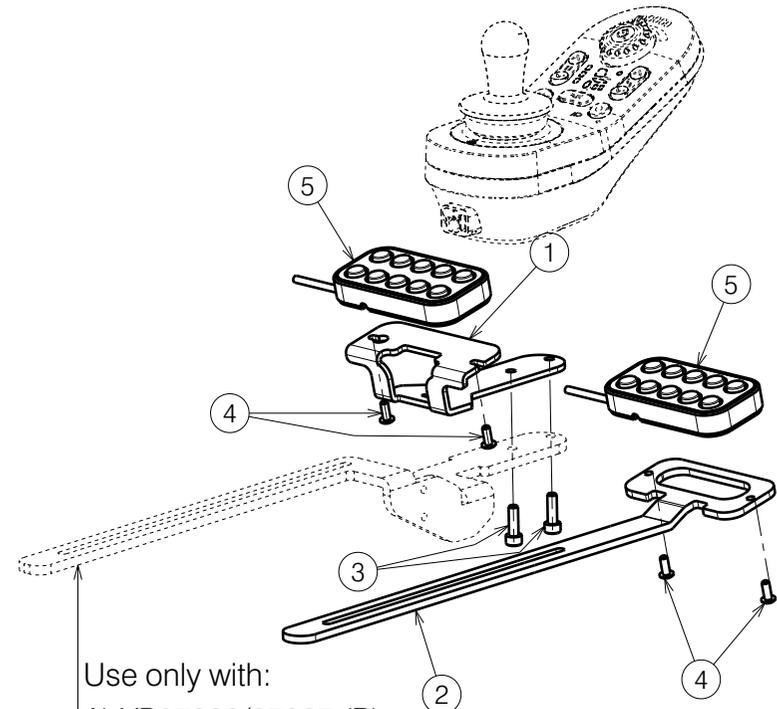

Use only with:  
A) YP07833/27827 (R)  
B) YP07834/27828 (L)

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS  
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN  
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,  
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION  
FROM VERMEIREN GROUP.

SE64 OPTION / TURIOS				---
				Plan No.
				E00061546/A.2
Scale: 1:4	Drafted	Verified	Confirmed	
Date	16.06.2023	04/07/2023	04/07/2023	
Initials	AZ	MVG	MK	

