


| Material | | | |
|----------|----------------------|---------|----------|
| No. | Plan | Article | Quantity |
| 1 | YP04681(Aqua/ Happy) | 2888506 | 1 |
| | YP04681(Sporty) | 2888565 | 1 |
| | YP04681(Safari) | 2888587 | 1 |
| 2 | YP04682(Aqua/Happy) | 2888406 | 1 |
| | YP04682(Sporty) | 2888465 | 1 |
| | YP04682(Safari) | 2888487 | 1 |
| 3 | --- | 1000894 | 1 [set] |
| 4 | --- | 2993196 | 1,12[m] |
| 5 | --- | 2993196 | 0,45[m] |
| 6 | M5x10 DIN 912/8.8 | 1001488 | 1 |
| 7 | M6 DIN 985 | 1001680 | 5 |
| 8 | X003698 | 1004550 | 1 |
| 9 | M6x25 DIN 933/8.8 | 1001431 | 1 |
| 10 | D6,5 | 1997328 | 1 |
| 11 | M6x40 | 1006111 | 4 |
| 12 | D6,4 DIN 125 A | 1000973 | 4 |
| 13 | D6,4 DIN 125 | 1001564 | 4 |
| 14 | --- | B000015 | 1 |
| 15 | M5 DIN 985 | 1001679 | 2 |
| 16 | D3,9x9,5 DIN 7981 | 1001530 | 2 |
| 17 | M5x16 | 1006112 | 2 |
| 18 | XP01255 | 2992206 | 1 |
| 19 | P00056781 | 1388900 | 1 |
| 20 | X001302 | 1387400 | 1 |
| 21 | X001016 | 1001682 | 2 |
| 22 | X003827 | 1009400 | 2 |
| 23 | M8 DIN 125 B | 1001556 | 2 |
| 24 | --- | 1990488 | 2 |

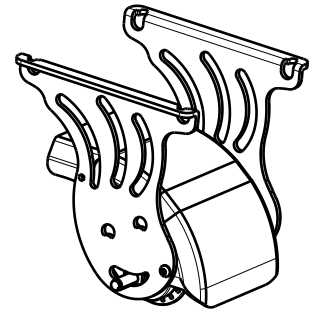

EP00155 / D

| | | | | |
|----------------------------|------------|------------|------------|------------------------|
| OPTION TORPEDO 2202/2215 | | | | Plan No. EP00155/ D |
| Scale: 1:4 | Drafted | Verified | Confirmed | |
| Date | 30/04/2014 | 22/05/2023 | 22/05/2023 | |
| Initials | AS | PM | TK | |