


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
E000981 / D

B08 TYPE D-SERIES			Sheet 1/2	Plan No. E000981 / D
Scale: 1:4	Drafted	Verified	Confirmed	
Date	27/02/2015	05/09/2019	05/09/2019	
Initials	TT	SD	PJ	

Material			
No.	Plan	Article	Quantity
1	Y003841	F002392	1
2	Y006293	1594154	1
3	Y012670	1672404	1
4	Y021362	2010454 (L)	1
	Y021353	2010554 (R)	1
5	Y021360	2010654 (L)	1
	Y021355	2010754 (R)	1
6	XC00129	3990491	1
7	XC00156	3991001 (38)	1
	X002102	1004824 (39-44)	1
	X001740	1004822 (46-52)	1
8	XC00300	3991091	1
9	XC00304	3991093	1
10	XC00305	3991094	1
11	X000106	G003218	1
12	X000586	1005742	2
13	X002137	1992415	1
14	X002169	1992791	1
15	X002238	1992702	1
16	X003036	1997486	2
17	X004414	1904130	1
18	X004418	1904207	1
19	X004660	1905195	1

Material			
No.	Plan	Article	Quantity
20	M6 DIN125A	1000973	1
21	M6X35 DIN912	1001496	1
22	Ø4.8X25 DIN7983	1001536	1
23	M6 DIN125A	1001564	3
24	Ø16	1001598	1
25	---	1001637	1
26	M6 DIN985	1001680	3
27	Small	1002163	1
	Large	1002158	1
28	M6X20 ISO7380	1006999	2
29	M6 DIN986	1008492	3
30	M6X30 ISO7380	1009588	1
31	M6X15	1009933	1
32	Ø3X24 DIN1475	1905640	1
33	M6X55 DIN912	1908726	1
34	M6X45 ISO7380	1991017	2
35	M6X30 DIN7984	1992721	1
36	420x40	1992802	1
37	Ø4X8	1992805	1

Only left side depicted



E000981 / D

B08 TYPE D-SERIES			Sheet 2/2	Plan No. E000981 / D
Scale: 1:4	Drafted	Verified	Confirmed	
Date	27/02/2015	05/09/2019	05/09/2019	
Initials	TT	SD	PJ	