


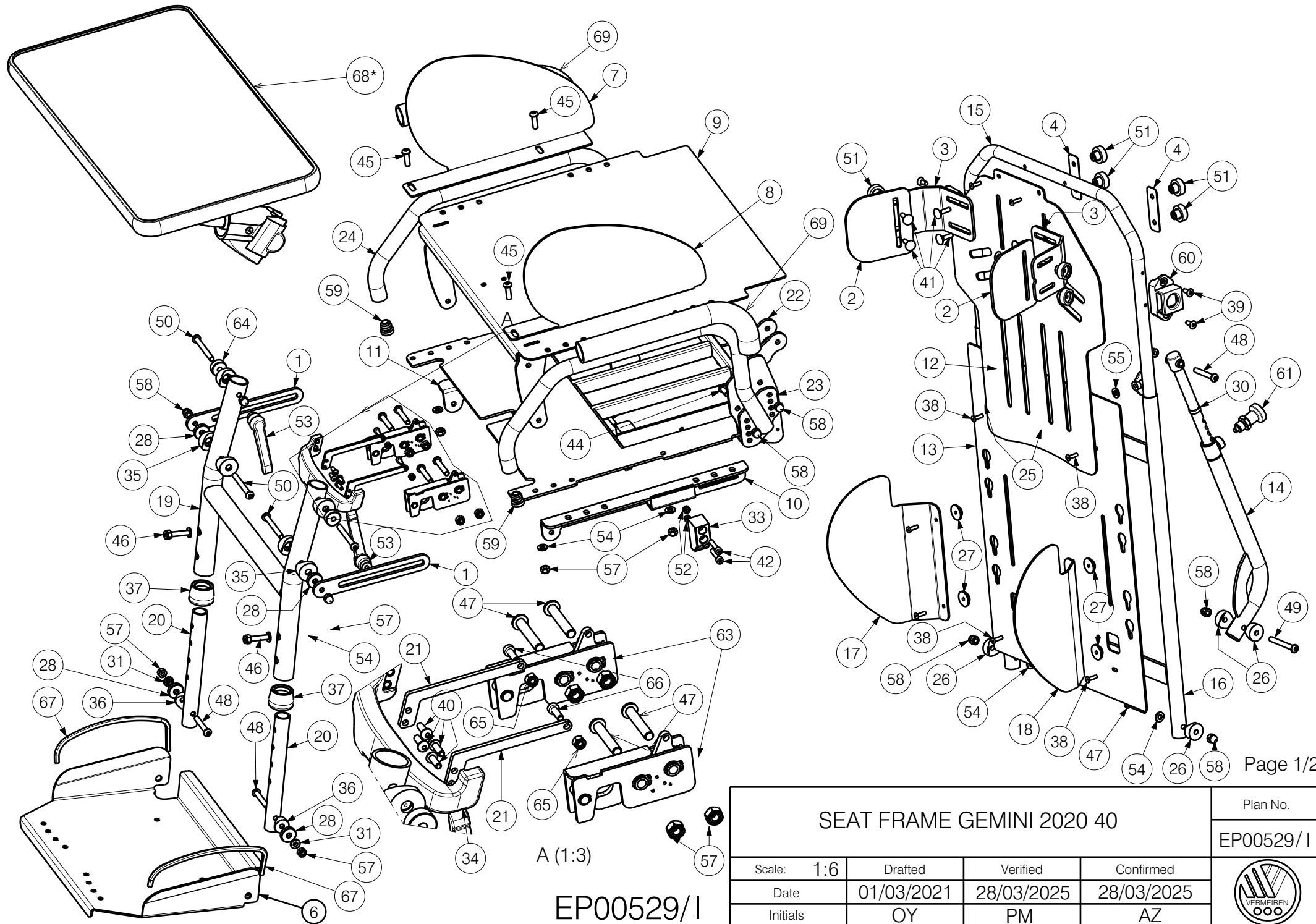

SEAT FRAME GEMINI 2020 40				Plan No.
				EP00529/I
Scale: 1:6	Drafted	Verified	Confirmed	
Date	01/03/2021	28/03/2025	28/03/2025	
Initials	OY	PM	AZ	

Bill of Material				Bill of Material				Bill of Material			
No.	Plan	Article	Quantity	No.	Plan	Article	Quantity	No.	Plan	Article	Quantity
1	YP00511	1815354	2	33	XP02997	2996618	2	64	XP00099	2990067	2
2	YP04259	2853154	2	34	XP03079	2996655	1	65	---	2994901	2
3	YP04295	2853054	2	35	X000106	G003218	6	66	---	1907582	2
4	YP04319	2852454	2	36	X000586	1005742	2	67	---	1003299	2
5	---	---	-	37	X002137	1992415	2	68*	U63	E000299	1
6	YP04502	2867854	1	38	Rivet D4x14	1001584	10	69	---	1991107	1
7	YP04554	2883754	1	39	Screw 4.8X16	2994905	4				
8	YP04555	2883854	1	40	M4x6 ISO 7380	2994903	4				
9	YP04561	2883254	1	41	M5x20 DIN 603	2991056	8				
10	YP04574	2867954	1	42	M5x25 DIN 912	2994906	4				
11	YP04575	2868054	1	43	---	---	-				
12	YP04577	2868354	1	44	M6x16 ISO 7380	2994067	4				
13	YP04578	2868454	1	45	M6x20 ISO 7380	2994910	4				
14	YP04579	2868554	1	46	M6x30 DIN 7984	1915416	2				
15	YP04583	2868754	1	47	M6x35 ISO 7380	2994131	6				
16	YP04584	2868854	1	48	M6x40 ISO 7380	2994066	3				
17	YP04585	2869154	1	49	M6x50 ISO 7380	2994965	1				
18	YP04586	2869254	1	50	M6x55 ISO 7380	2994376	4				
19	YP04617	2883654	1	51	Knob M5	2991057	8				
20	YP04618	2703254	2	52	M5 DIN 985	2994006	4				
21	YP08484	2617554	2	53	M6 Handle	1996843	2				
22	YP08564	2620754	1	54	M6 DIN 125	2994414	8				
23	YP08639	2624254	1	55	M6 DIN 125 PVC	2994415	6				
24	YP08640	2624354	1	56	M6 DIN 315	2994909	2				
25	XP00140	2990097	4	57	M6 DIN 985	2994063	13				
26	XP00200	2990129	4	58	M6 DIN 986	2994059	11				
27	XP00223	2990144	4	59	Cap D16	1001598	2				
28	XP00227	2990147	4	60	---	2990099	2				
29	---	---	-	61	Blocker Pin	2990150	1				
30	XP01099	2866000	1	62	---	---	-				
31	XP01119	2883900	2	63	Rotary Latch	EP00517	2				
32	---	---	-								
EP00529 / I							SEAT FRAME GEMINI 2020 40				Page 2/2
											Plan No.
											EP00529/ I
							Scale: 1:6	Drafted	Verified	Confirmed	
							Date	01/03/2021	28/03/2025	28/03/2025	
							Initials	OY	PM	AZ	