


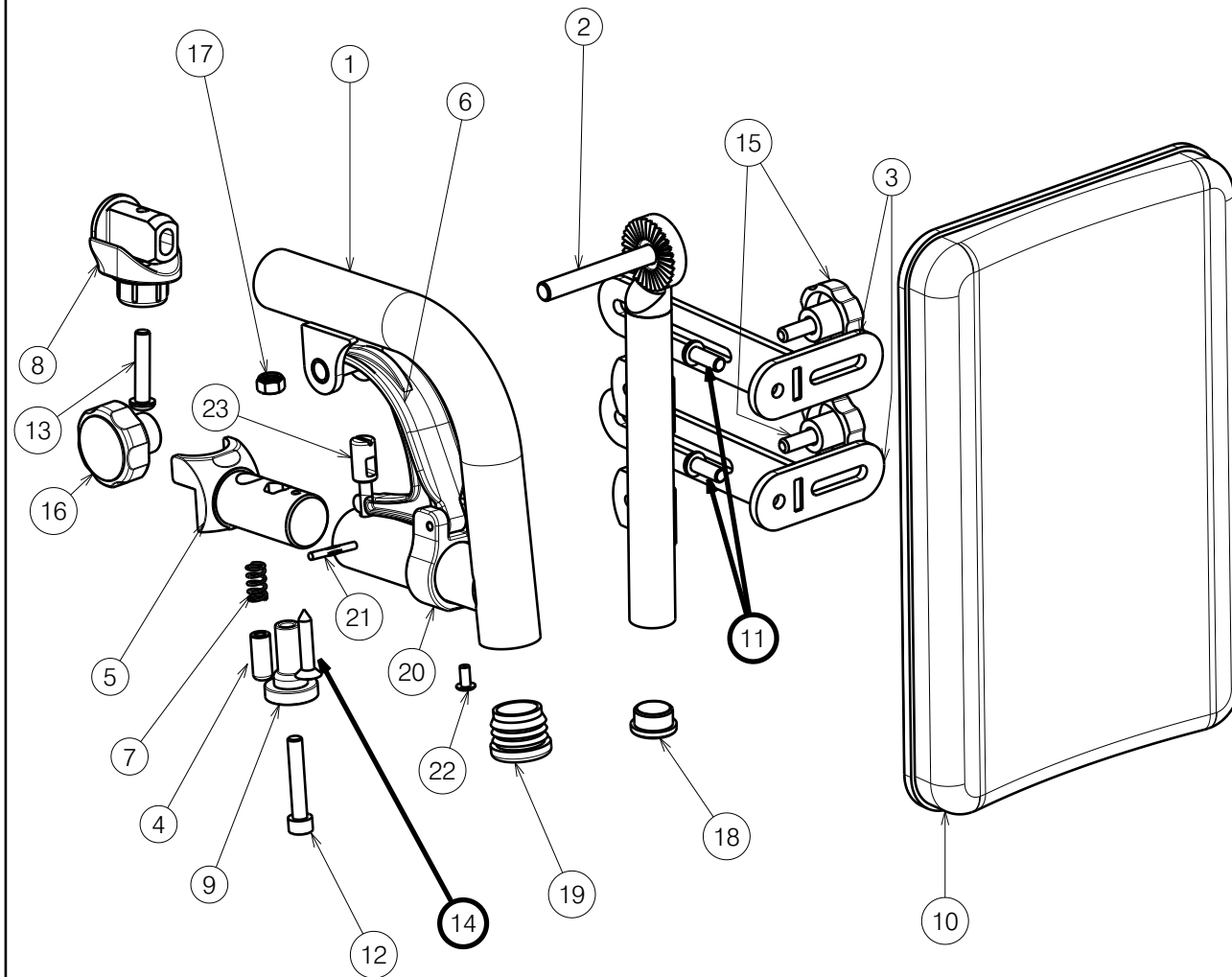

Material			
No.	Plan	Article	Quantity
1	YP08090	2607154 (L)	1
	YP08091	2607254 (R)	1
2	YP08093	2607354 (L)	1
	YP08094	2607454 (R)	1
3	YP08095	2607554	4
4	XC00304	3991093	2
5	XC00129	3990491	2
6	X004418	1904207	2
7	X002169	1992791	2
8	X004414	1904130	2
9	X004660	1905195	2
10	Plate small	1002163	2
	Plate large	1002158	2
11	M6x16	2994067	4
12	M6x35	2994634	2
13	M6x30	2994129	2
14	Screw 5x25	H016007	2
15	Star Knob M6x15	1001577	4
16	Plastic Head Ø35	1998550	2
17	Nut M6	1001680	2
18	Tube cap Ø19	1001602	2
19	Tube cap Ø25,4	1001608	2
20	CX00300	3991091	2
21	---	1905640	2
22	---	1992805	2
23	XC00305	3991094	2

BL7 Right side only depicted, quantities are for 1 pair

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

BK7				---
				Plan No.
				EP00499/D
Scale: 1:3	Drafted	Verified	Confirmed	
Date	27/02/2020	19/05/2025	19/05/2025	
Initials	PM	PM	AS	