

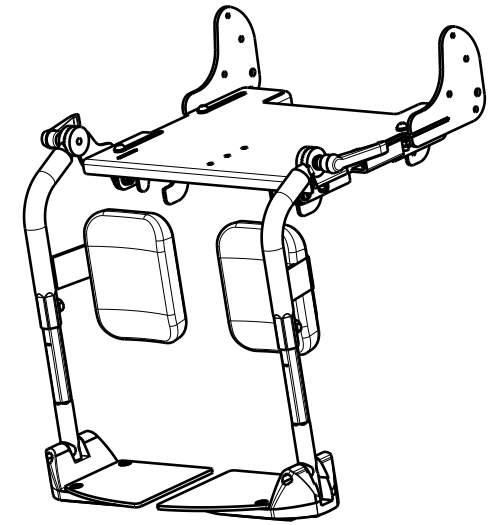


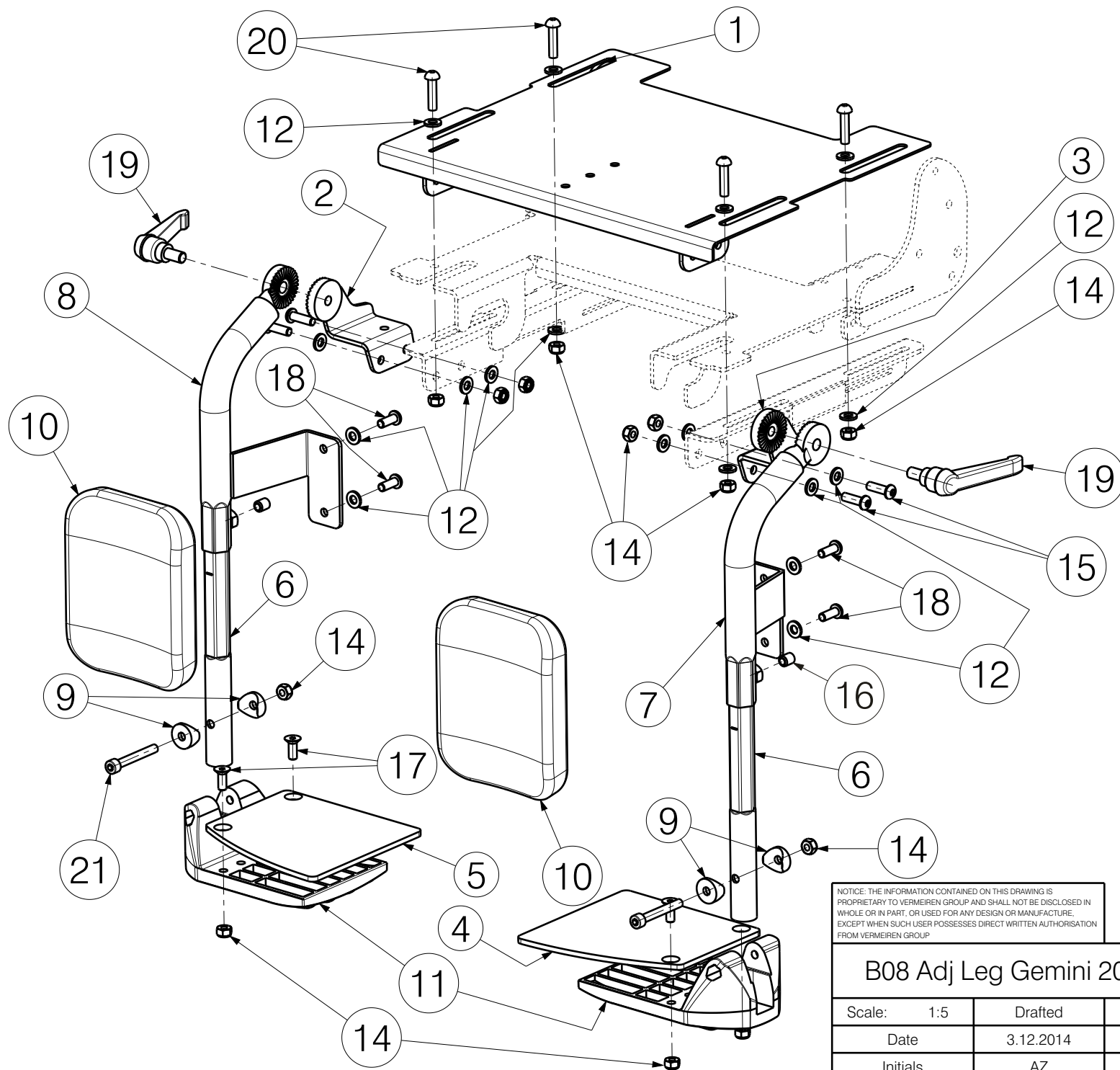

Isometric view
Scale: 1:8

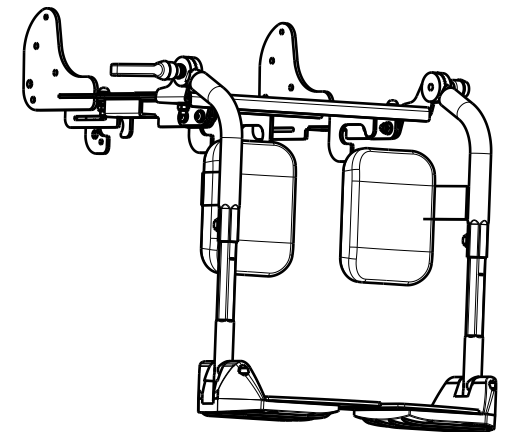

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

B08 Adj Leg Gemini 2020/32, Bayo 2020 32


Scale: 1:5	Drafted	Verified	Confirmed
Date	3.12.2014	26.06.2025	26.06.2025
Initials	AZ	PM	AZ

1/2

Plan No.

EP00196/C



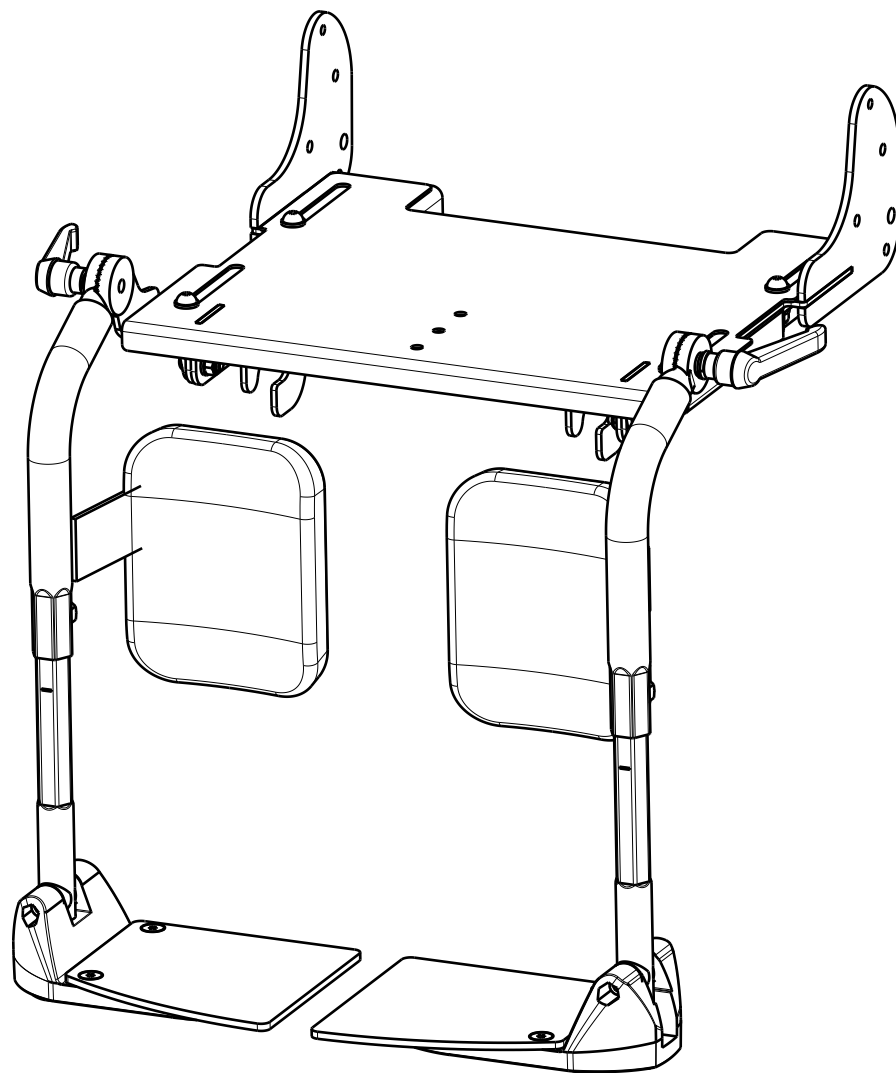

Material

No.	Plan	Article	Quantity
1	YP05014	28971...	1
2	YP05016	28973...	1
3	YP05017	28972...	1
4	YP05022	28975...	1
5	YP05023	28974...	1
6	YP05026	28976...	2
7	YP05028	28977...	1
8	YP05029	28978...	1
9	X000586	1005742	4
10	X003779	1900228	2
11	XP00007	2990000	2
12	D6,4 DIN 125 A	1001564	20
13	---	---	-
14	M6 DIN 985	1001680	14
15	M6x20	1006999	4
16	M8x10 DIN 916/45H	1008942	2
17	M6x16 DIN 7991/8.8	1994752	4
18	M6x16	1008459	4
19**	Lever M8x15	1006661	2
*	Star Knob M8x17	1902618	2
20	M6x25 DIN 912/8.8	1001494	4
21	M6x45 DIN 912/8.8	1003109	2

* - not depicted / nie ujeta

19* - Lever M8x15 use for BAYO 32

Star Knob M8x17 for GEMINI 2/32

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

2/2

B08 Adj Leg Gemini 2020/32, Bayo 2020 32

Plan No.

EP00196/C

Scale: 1:5	Drafted	Verified	Confirmed
Date	3.12.2014	26.06.2025	26.06.2025
Initials	AZ	PM	AZ

