


No.	Plan	Article	Qty
1	E00085879	-	1
2	E00081324	-	1
3	E00108383	-	1
4	E00108385	-	1
5	E00108120	-	1
6	E00108387	-	1
7	FRONT CARPET	2998627	1
8	CARPET	2998628	1
9	P00091705	27114...	4
10	SEAT	1922748	1
11	NUT M8 DIN985	3991743	5
12	SCREW M8x20 DIN933	1009400	4
13	NUT M6 DIN985	1001680	5
14	SCREW M6X35 ISO7380	2994131	1
15	XC00364	3991351	13
16	XC00373	3991368	13
17	SCREW M8X35 ISO 7380	1993236	1
18	SCREW M6X20 ISO7380 BLACK	2994010	4
20	SEAT TUBE	1922749	1
21	NUT M10	1001683	1
22	M10x55 DIN933	1005636	1

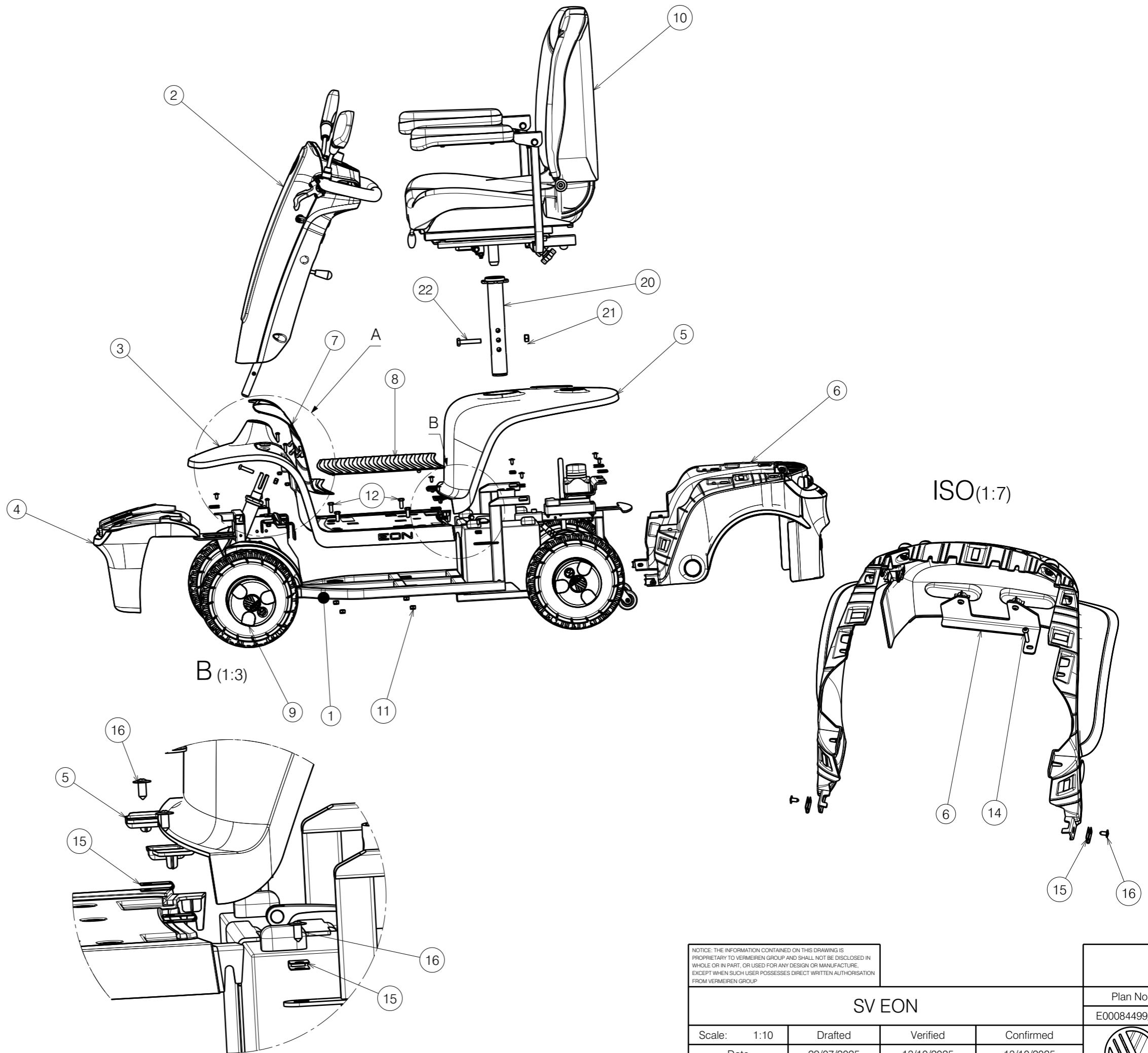

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

SV EON			
Scale: 1:10	Drafted	Verified	Confirmed
Date	29/07/2025	13/10/2025	13/10/2025
Initials	BK	BK	MK

Plan No.
E00084499/A.1