


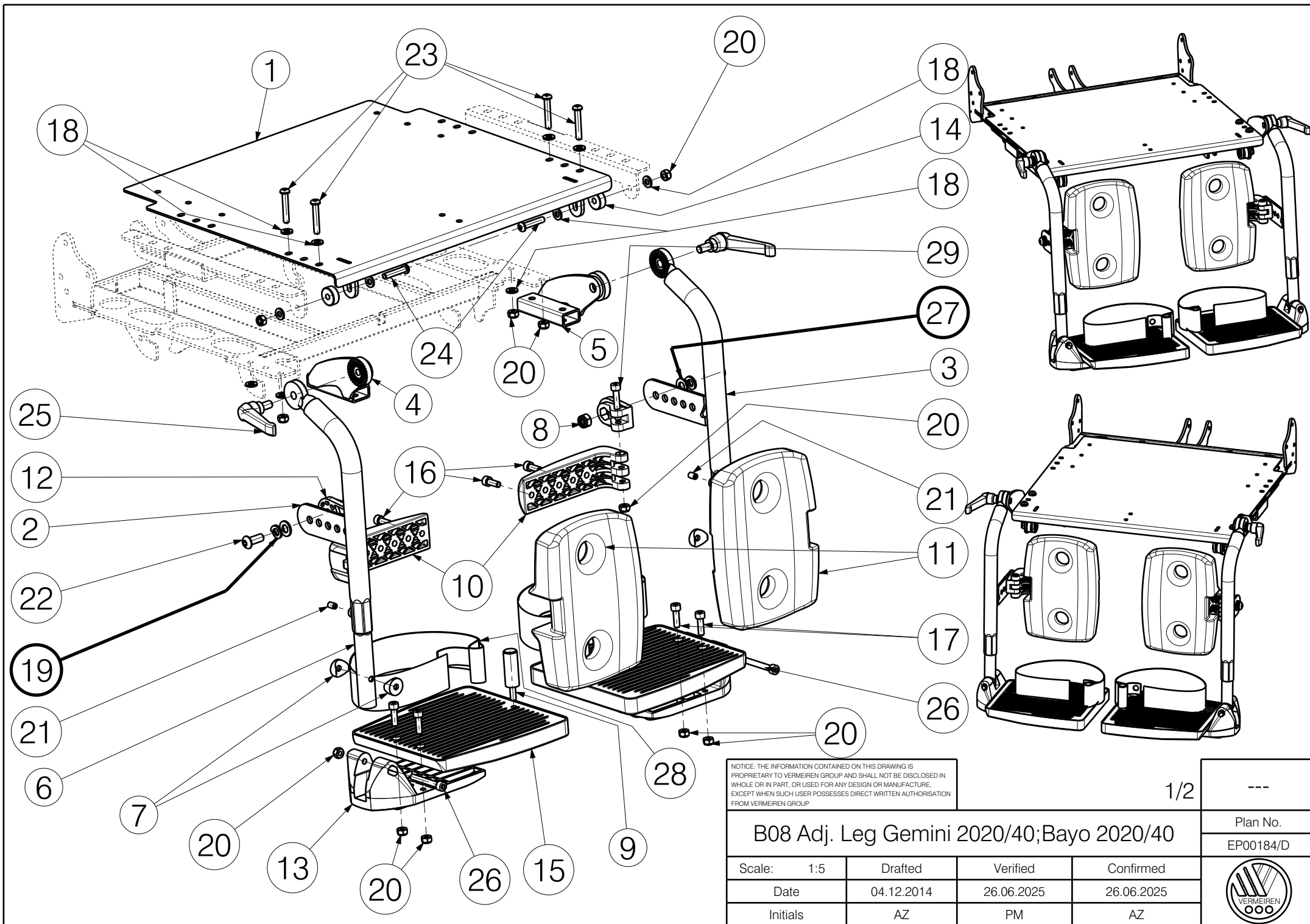

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

1/2

B08 Adj. Leg Gemini 2020/40; Bayo 2020/40


Scale:	1:5	Drafted	Verified	Confirmed
Date	04.12.2014	26.06.2025	26.06.2025	
Initials	AZ	PM	AZ	

Plan No.

EP00184/D



Material			
No.	Plan	Article	Quantity
1	YP04924	28927...	1
2	YP04925	28928...	1
3	YP04926	28930...	1
4	YP04927	28931...	1
5	YP04928	28932...	1
6	YP04945	28933...	2
7	X000586	1005742	4
8	X001016	1001682	2
9	X002238	1992702	2
10	X004784	1905950	2
11	X004785	1905949	2
12	X004787	1905947	2
13	XP00007	2990000	2
14	XP00099	2990067	2
15	XP00699	2990929	2
16	M6x16 DIN 912/8.8	1001492	4
17	M6x20 DIN 912/8.8	1001493	4
18	D6,4 DIN 125 A	1001564	12
19	M8 DIN 127 A	1001570	2
20	M6 DIN 985	1001680	14
21	M6x12 DIN 916/45H	1005949	2
22	M8x20	1992349	2
23	M6x35	1006057	4
24	M6x30	1009588	2
25	Lever M8x15	1006661	2
26	M6X45 DIN 912/8.8	1003109	2
27	D9 DIN 126	1001554	2
28	---	1992802	2
29	M6x35 DIN 912/8.8	1001496	2

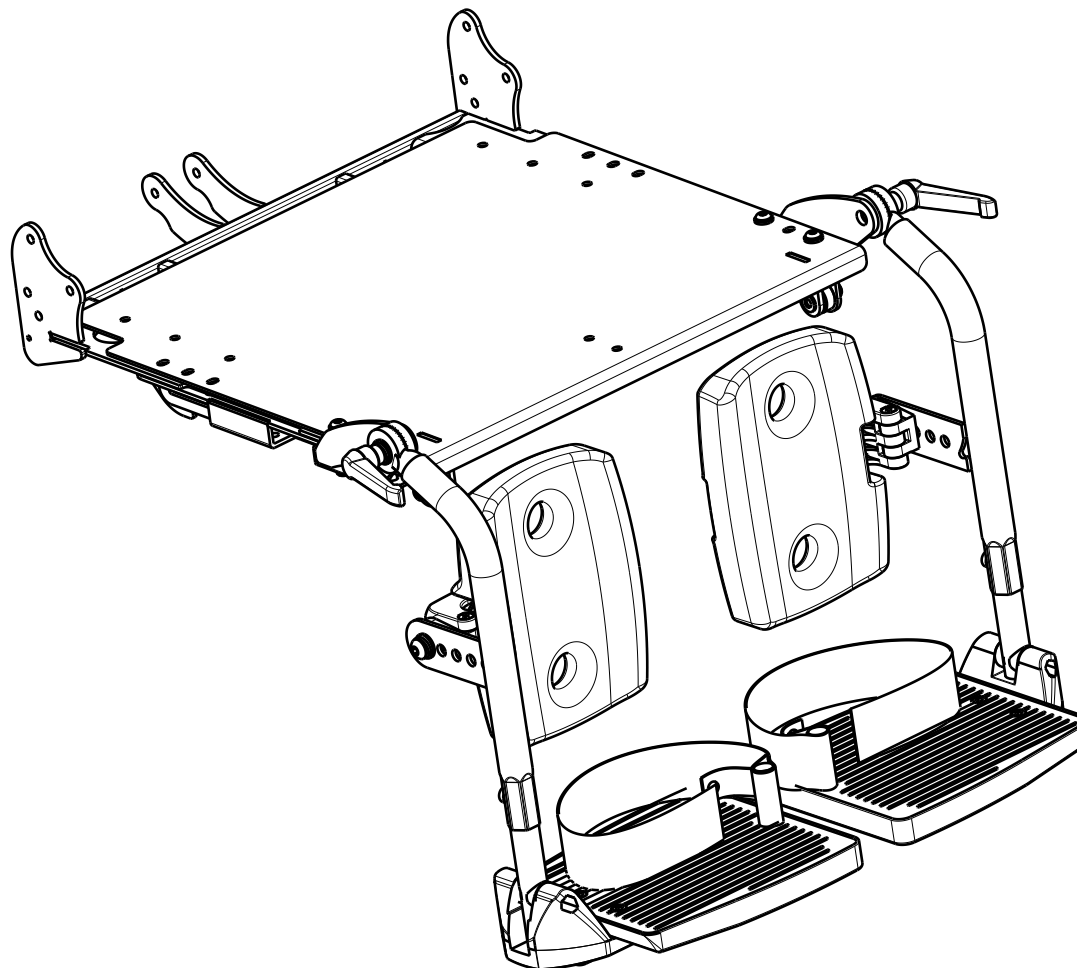

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
 PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
 WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
 EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
 FROM VERMEIREN GROUP.

2/2

B08 Adj. Leg Gemini 2020/40; Bayo 2020/40

Plan No.

EP00184/D

Scale: 1:5

Drafted

Verified

Confirmed

Date

04.12.2014

26.06.2025

26.06.2025

Initials

AZ

PM

AZ

