


Material

No.	Plan	Article	Quantity
1	X001016	1001682	1
2	X002133	1992424	4
3	X003692	1998763	4
4	X004452	1904058	4
5	Y018378	F006644(38)	2
	X004606	1905091(44/48)	2
	Y018379	F006645(40)	2
	X004607	1905092(42/46/50)	2
	X005002	1907928(52)	2
6	X004620	1905090	2
7	YC00645	1C24254 (38)	1
	YC00649	1C24454 (40)	1
	Y015113	1728954 (42)	1
	Y015099	1729154 (44)	1
	Y015116	1729354 (46)	1
	Y015119	1729554 (48)	1
	Y015122	1729754 (50)	1
	YC00654	1C24654 (52)	1
8	YC00644	1C24154 (38)	1
	YC00648	1C24354 (40)	1
	Y015114	1728854 (42)	1
	Y015100	1729054 (44)	1
	Y015117	1729254 (46)	1
	Y015120	1729454 (48)	1
	Y015123	1729654 (50)	1
	YC00653	1C24554 (52)	1
9	---	1001530	4
10	X001119	1001554	2
11	---	1001678	4
12	---	1001680	6
13	---	1006111	4
14	---	1006997	2
15	---	1991711	1
16	---	2990185	4
18A	---	H001126	0.84
18	---	1905646 (38/40/42/44)	1
	---	1905647 (46/48/50)	1
	---	3991018 (52)	1

Front side

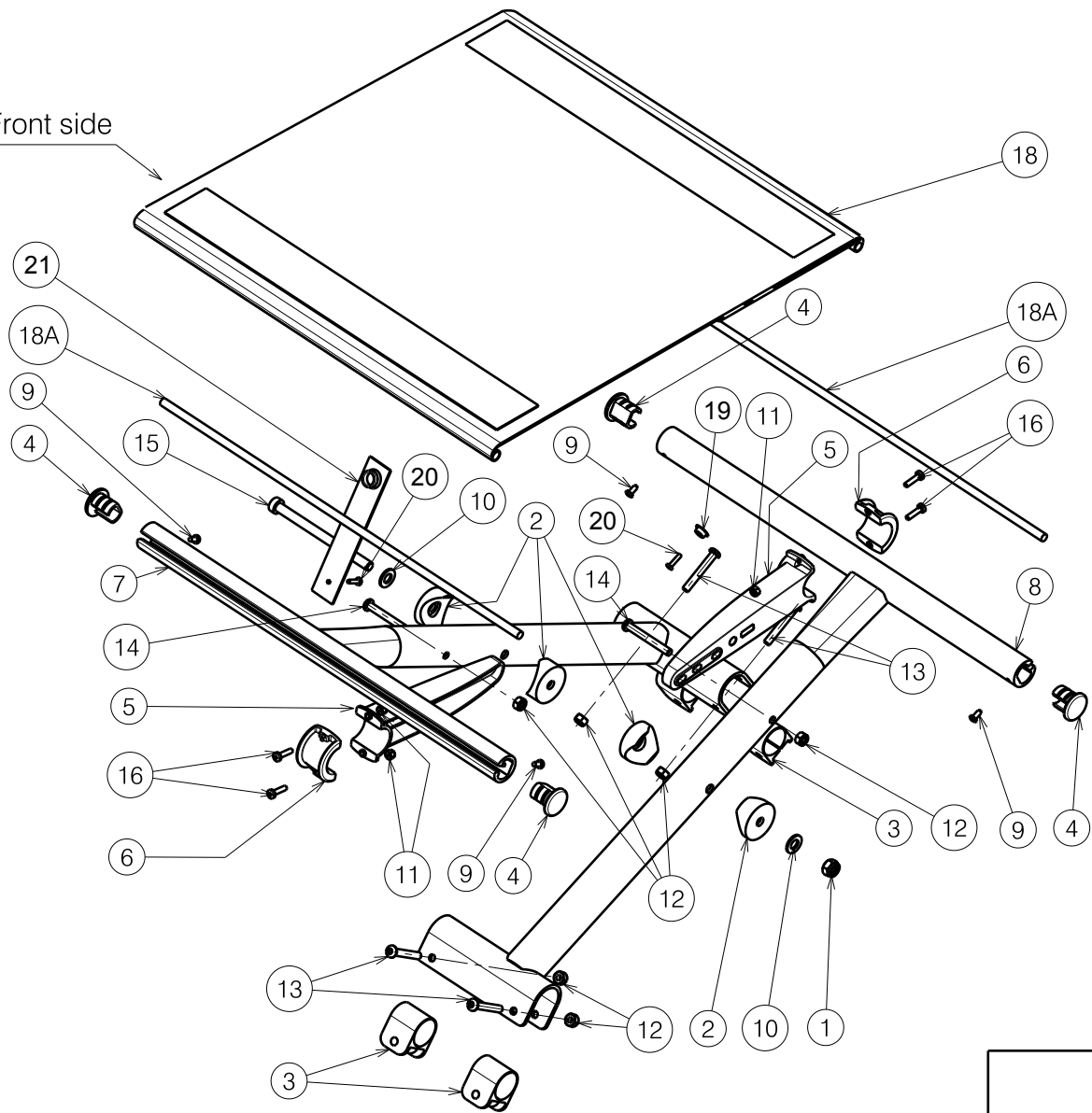

No.	Plan	Article	Quantity
19	---	1900773	1
20	---	1905708	2
21	---	1905711	1

CROSS D400

Plan No. / Plannr.

EC00098/B

Scale / Schaal: 1:5	Drafted / Getekend	Verified / Nagezien	Confirmed / Bevestigd
Date / Datum	03/09/2012	03/09/2012	03/09/2012
Init. / Init.	Lydia	AG	TG



EC00098/B