


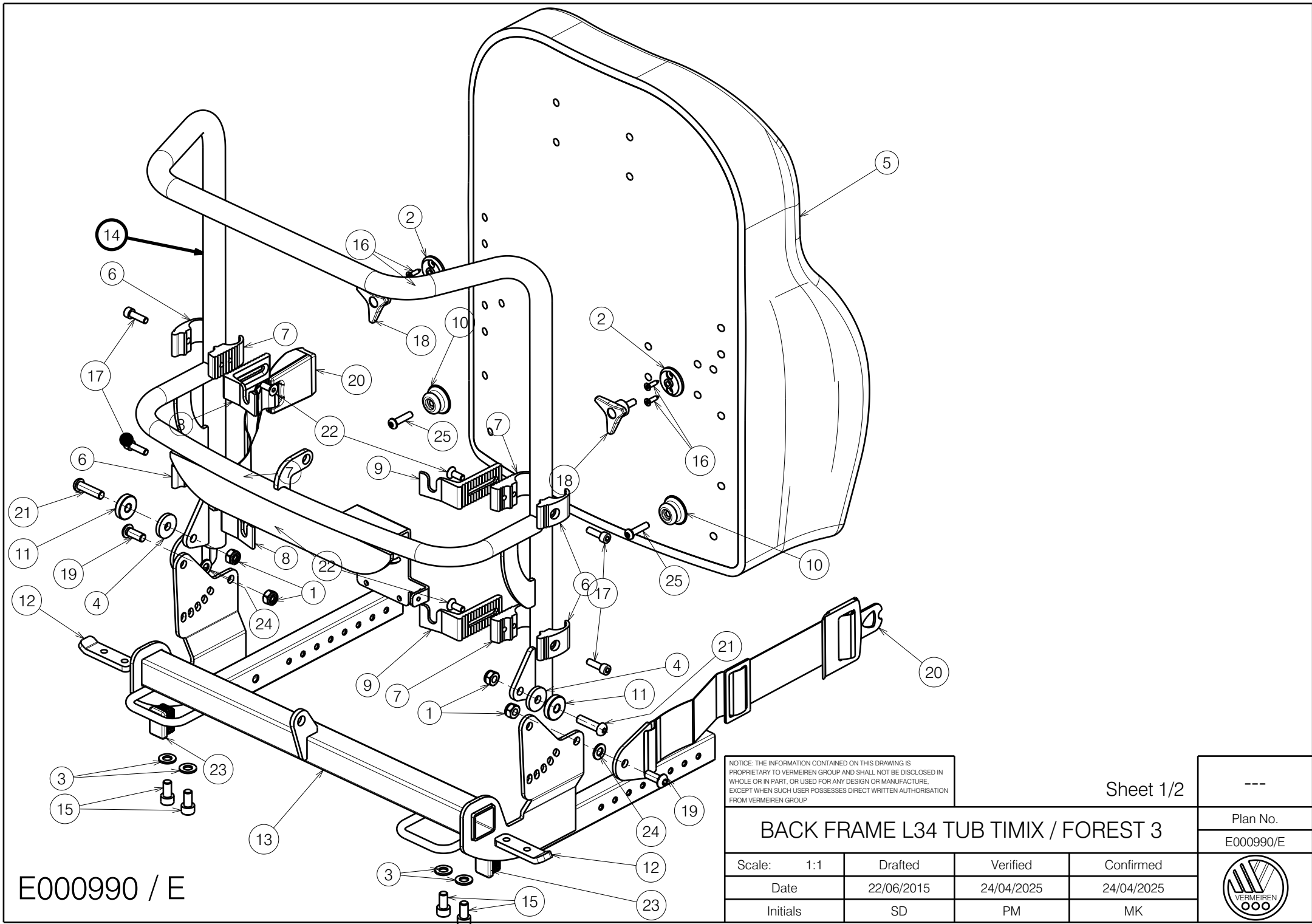

E000990 / E

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.

Sheet 1/2

BACK FRAME L34 TUB TIMIX / FOREST 3				Plan No.
				E000990/E
Scale: 1:1	Drafted	Verified	Confirmed	
Date	22/06/2015	24/04/2025	24/04/2025	
Initials	SD	PM	MK	

BACK FRAME L34 TUB TIMIX / FOREST 3

Material							
No.	Plan	Article	Quantity	No.	Plan	Article	Quantity
1	X001016	1001682	4	14	YP09205	2638854 (40)	1
2	X001052	G066600	2		YP09206	2638954 (45)	1
3	X001119	1001554	4		YP09207	2639054 (50)	1
4	X004048	1902685	2	15	M8x16 DIN912	1001498	4
5	---	1908883 (40A)	1	16	D3,9x16 DIN7982	1005767	4
	---	1908877 (40B)	1	17	M6x20 DIN912	1001493	4
	---	1908880 (40C)	1	18	M6x15	1009932	2
	---	1906938 (40D)	1	19	M8x20 ISO7380	1992349	2
	---	1908884 (45A)	1	20	---	2993352	1
	---	1908878 (45B)	1	21	M8x30 ISO7380	1992717	2
	---	1908881 (45C)	1	22	M6x16 DIN7991	1994752	4
	---	1906940 (45D)	1	23	30x20	1997922	2
	---	1908885 (50A)	1	24	M8 DIN125	1003199	2
	---	1908879 (50B)	1	25	M6x25 ISO7380	1006998	2
	---	1908882 (50C)	1				
	---	1906942 (50D)	1				
6	X004820	1744532	4				
7	X004821	1744632	4				
8	X004822	1744732	2				
9	X004823	1744832	2				
10	X004861	1906454	2				
11	XP00921	2991609	2				
12	Y018564	1774200	2				
13	Y019511	1784645 (40)	1				
	Y019510	1784754 (45)	1				
	Y019512	1784854 (50)	1				

A: Black skai - Blue lining
B: Black skai - Light green lining
C: Black skai - Red lining
D: Grey soft / Black skai

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

Sheet 2/2

BACK FRAME L34 TUB TIMIX / FOREST 3

Plan No.

E000990/E

Scale: 1:1	Drafted	Verified	Confirmed
Date	22/06/2015	24/04/2025	24/04/2025
Initials	SD	PM	MK



E000990 / E