



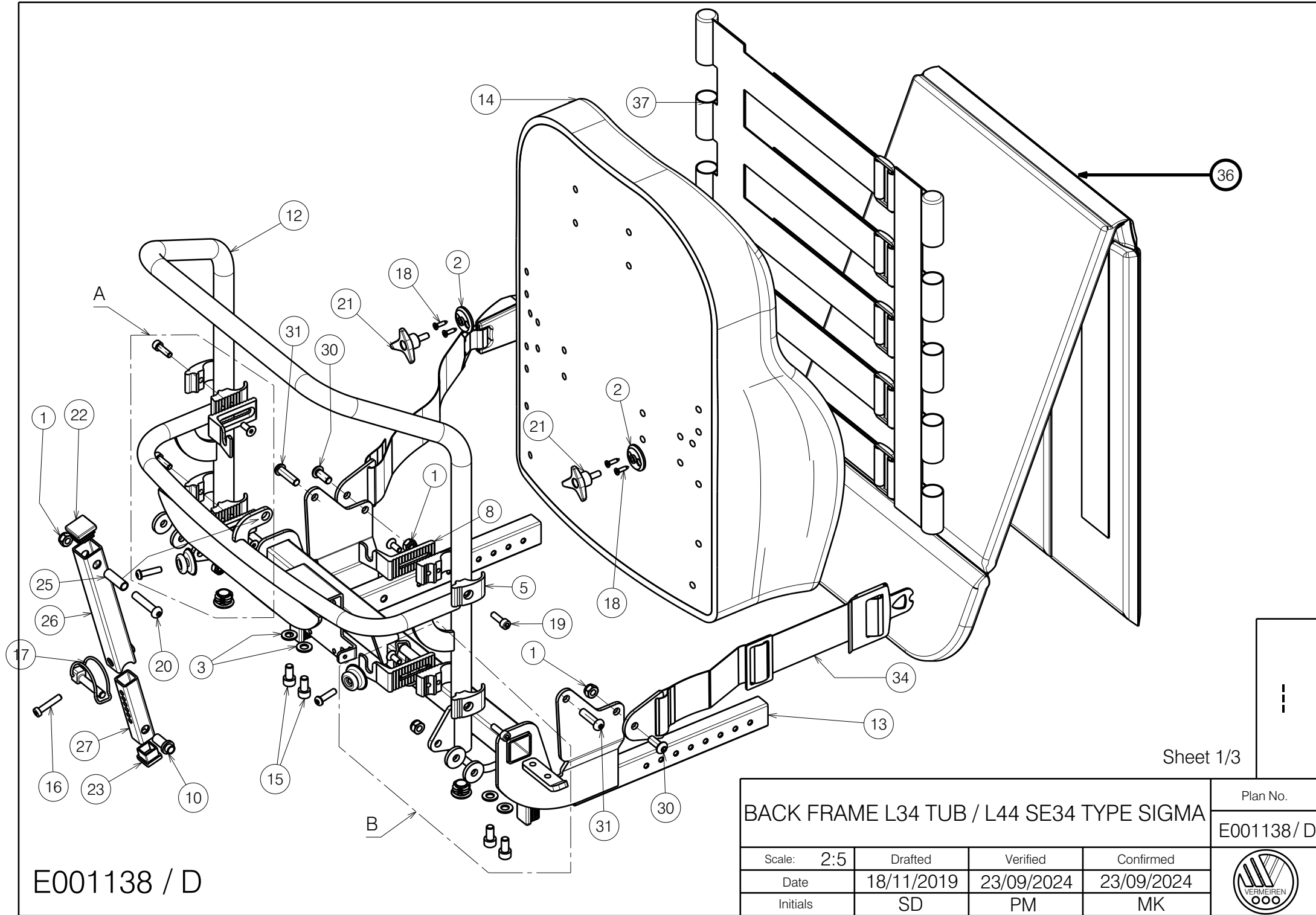


E001138 / D

Sheet 1/3

BACK FRAME L34 TUB / L44 SE34 TYPE SIGMA				Plan No.
				E001138/ D
Scale: 2:5	Drafted	Verified	Confirmed	
Date	18/11/2019	23/09/2024	23/09/2024	
Initials	SD	PM	MK	

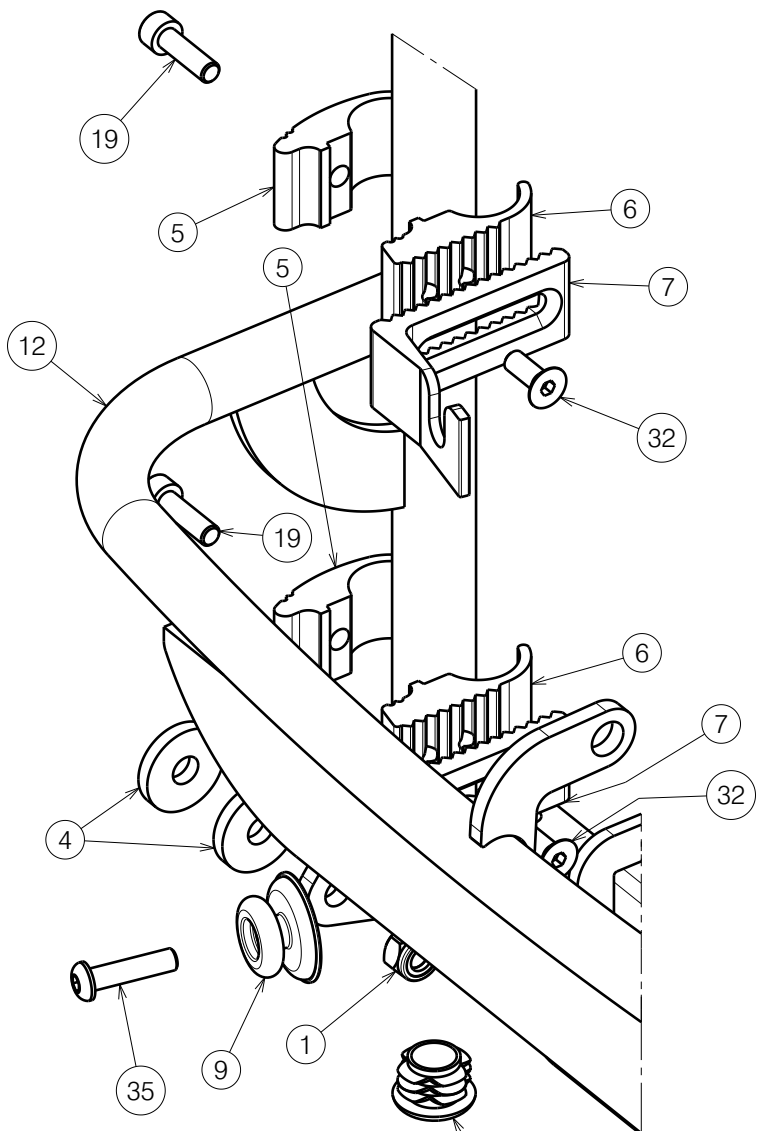

Detail A
Scale: 1:2

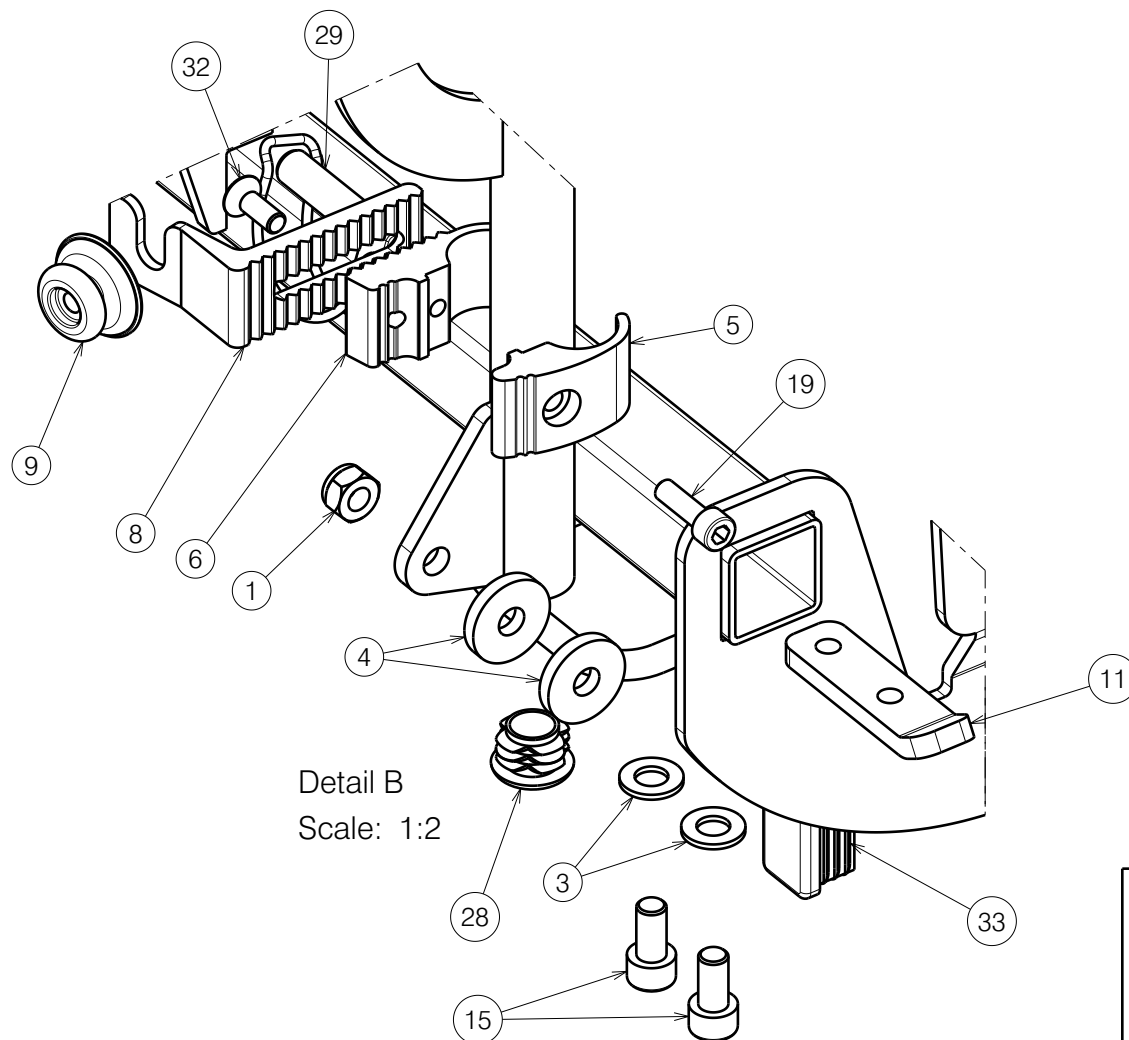

Detail B
Scale: 1:2

Sheet 2/3

E001138 / D

BACK FRAME L34 TUB / L44 SE34 TYPE SIGMA				Plan No.
				E001138 / D
Scale: 2:5	Drafted	Verified	Confirmed	
Date	18/11/2019	23/09/2024	23/09/2024	
Initials	SD	PM	MK	

Material			
No.	Plan	Article	Quantity
1	X001016	1001682	4
2	X001052	G066600	2
3	X001119	1001554	4
4	X004048	1902685	4
5	X004820	1744599	4
6	X004821	1744699	4
7	X004822	1744799	2
8	X004823	1744899	2
9	X004861	1906454	2
10	X005471	2011600	1
11	Y018564	1774200	2
12	YP09205	2638854 (40)	1
	YP09206	2638954 (45)	1
	YP09207	2639054 (50)	1
13	Y022312	2049854 (40)	1
	Y022980	2049754 (45)	1
	Y022313	2049954 (50)	1
14	Soft grey 40	1912001	1
	Soft grey 45	1911493	1
	Soft grey 50	1912063	1
	Soft black 40	2998244	1
	Soft black 45	2998245	1
	Soft black 50	2998246	1

Material			
No.	Plan	Article	Quantity
15	M8x16 DIN912	1001498	4
16	M6x30 DIN7984	1992721	1
17	Ø8x40	1992510	1
18	D3,9x16 DIN7982	1005767	4
19	M6x20 DIN912	1001493	4
20	M8x40 ISO7380	1008739	1
21	M6x15	1009932	2
22	25x25	1001617	1
23	20x20	1001612	1
25	Y021512	2011700	1
26	Y021526	2011554	1
27	Y021513	2011454	1
28	Ø22	1990648	2
29	Ø10x40	1991481	1
30	M8x20 ISO7380	1992349	2
31	M8x30 ISO7380	1992717	2
32	M6x16 DIN7991	1994752	4
33	30x20	1997922	2
34	---	2993352	1
35	M6x25 ISO7380	1006998	2
36	X004888	1906839 (40)	1
	X004857	1906740 (45)	1
	X004889	1906841 (50)	1
	---	1906843 (40)	1
37	---	1906844 (45)	1
	---	1906845 (50)	1

Sheet 3/3

E001138 / D

BACK FRAME L34 TUB / L44 SE34 TYPE SIGMA				Plan No.
				E001138/ D
Scale: 2:5	Drafted	Verified	Confirmed	
Date	18/11/2019	23/09/2024	23/09/2024	
Initials	SD	PM	MK	