



EP00463 / E

BRETAGNE WITH WHEELS				Plan No.
				1/3 EP00463/ E
Scale: 1:10	Drafted	Verified	Confirmed	
Date	22/10/2019	20/04/2022	20/04/2022	
Initials	PM	AZ	TK	

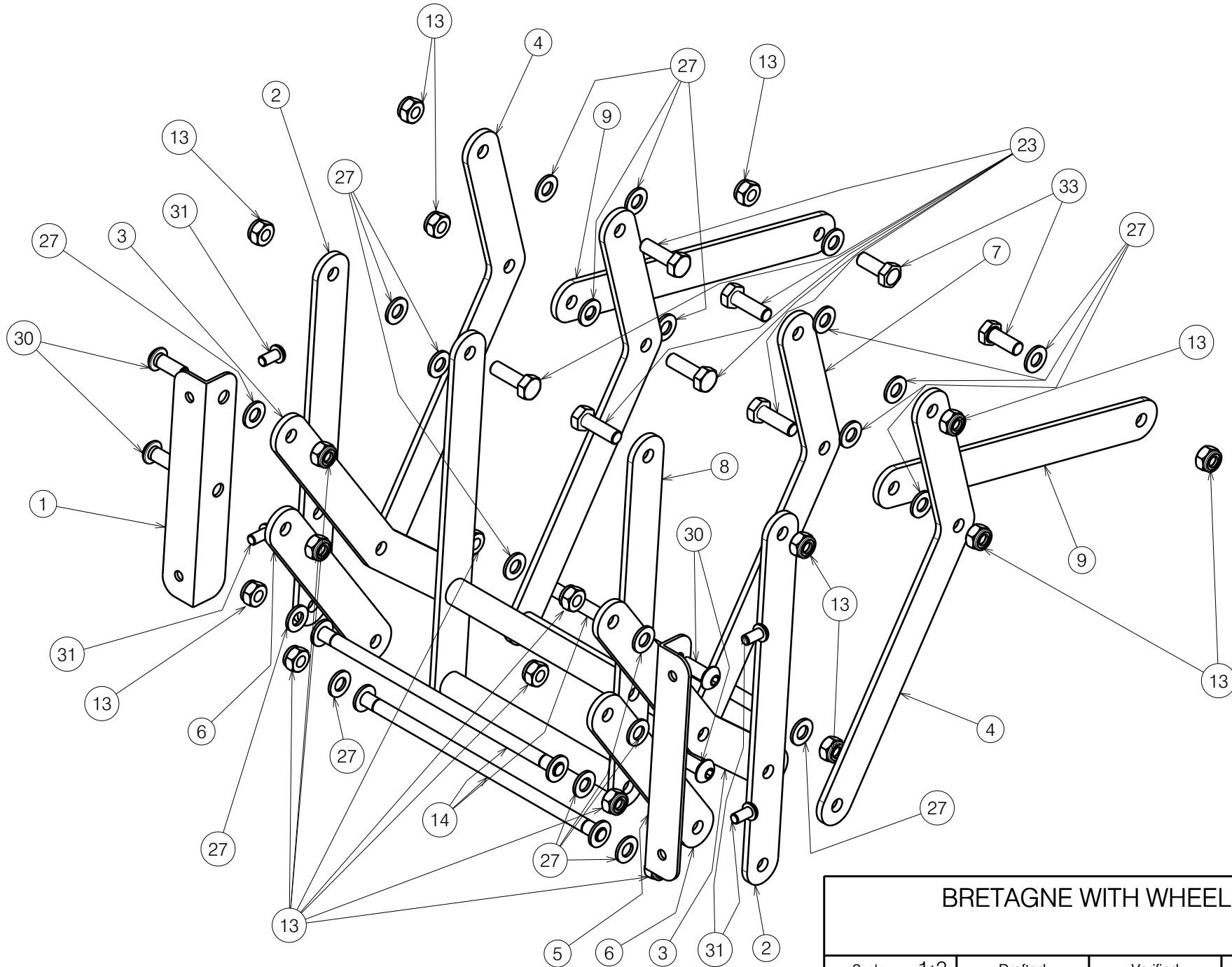

EP00463 / E

<b>BRETAGNE WITH WHEELS</b>				Plan No.
				2/3 EP00463/ E
Scale: 1:3	Drafted	Verified	Confirmed	
Date	22/10/2019	20/04/2022	20/04/2022	
Initials	PM	AZ	TK	

Material			
No.	Plan	Article	Quantity
1	YP00412	1822565	1
2	YP00483	1822465	2
3	YP00491	1822365	2
4	YP00568	1822265	2
5	YP00592	1822665	1
6	YP00635	1822865	2
7	YP00715	1824965	1
8	YP00716	1825065	1
9	YP05712	2722165	2
10a	YP07651	2776565	1
10b	YP08858 DP	2629865	1
11	YP07101	2773965	4
12	YP07547 DP	2774065	1
13	Nut M8	1001682	19
14	X001862	1447500	3
15	X005930	1916350	4
16a	XP01828	2993544	1
16b	XP03185	2996809	1
17	Armrest right	EP00465	1
18	Armrest left	EP00465	1
19	XP02824 (beech)	HP00620	1
	XP02826 (oxide)	HP00622	1
20	XP02825 (beech)	HP00621	1
	XP02827 (oxide)	HP00623	1
21a**	XP02628	HP00572	1
22a	XP02629	2994429	1
22b	XP03184	2996808	1

a - method 1

b - method 2

choose only one method and don't mix method 1 and 2 parts

\*not depicted on drawing

\*\*only for method 1 - not needed in method 2

\*\*\*only for method 2 - not needed in method 1

EP00463 / E

Material			
No.	Plan	Article	Quantity
23	M8x25 DIN933	1001442	6
24	M6 DIN934	1001670	8
25	M10 DIN985	1001683	4
26	M8x60 DIN912	1997781	2
27	D8,4 DIN125	1003199	30
28	Seat	EP00464	1
29	M8x45 DIN 933	1005901	1
30	M8x20	1992349	4
31	M6x12	1007000	16
32	Wheel D100 Black	1916101	4
33	M8x20 DIN933	1009400	2
34	M6x16 DIN7991 Black	2994064	32
35	M6x40	1006111	4
36	YP00915	1836965	1
37	X000989	1004539	4
38	YP06345	2735365	1
39	Cap 30x30	1003198	1
40*	Uph. footrest black	2994665	1
	Uph. footrest beige	2994667	1
	Uph. footrest brown	2994670	1
41*	Uph.backrest black	2994668	1
	Uph.backrest beige	2994669	1
	Uph.backrest brown	2994671	1
42	XP02829 (beech)	HP00624	1
	XP02830 (oxide)	HP00625	1
43	YP06347	2736065	1
44	YP06348	2736165	1
45	Washer M6	1001564	4
46	M6x20 DIN 7991	1903081	4
47b***	Y000462	F000050	2
48b***	X000273	G060600	2

BRETAGNE WITH WHEELS

Plan No.

3/3 EP00463/ E

Scale:	Drafted	Verified	Confirmed
Date	22/10/2019	20/04/2022	20/04/2022
Initials	PM	AZ	TK

