


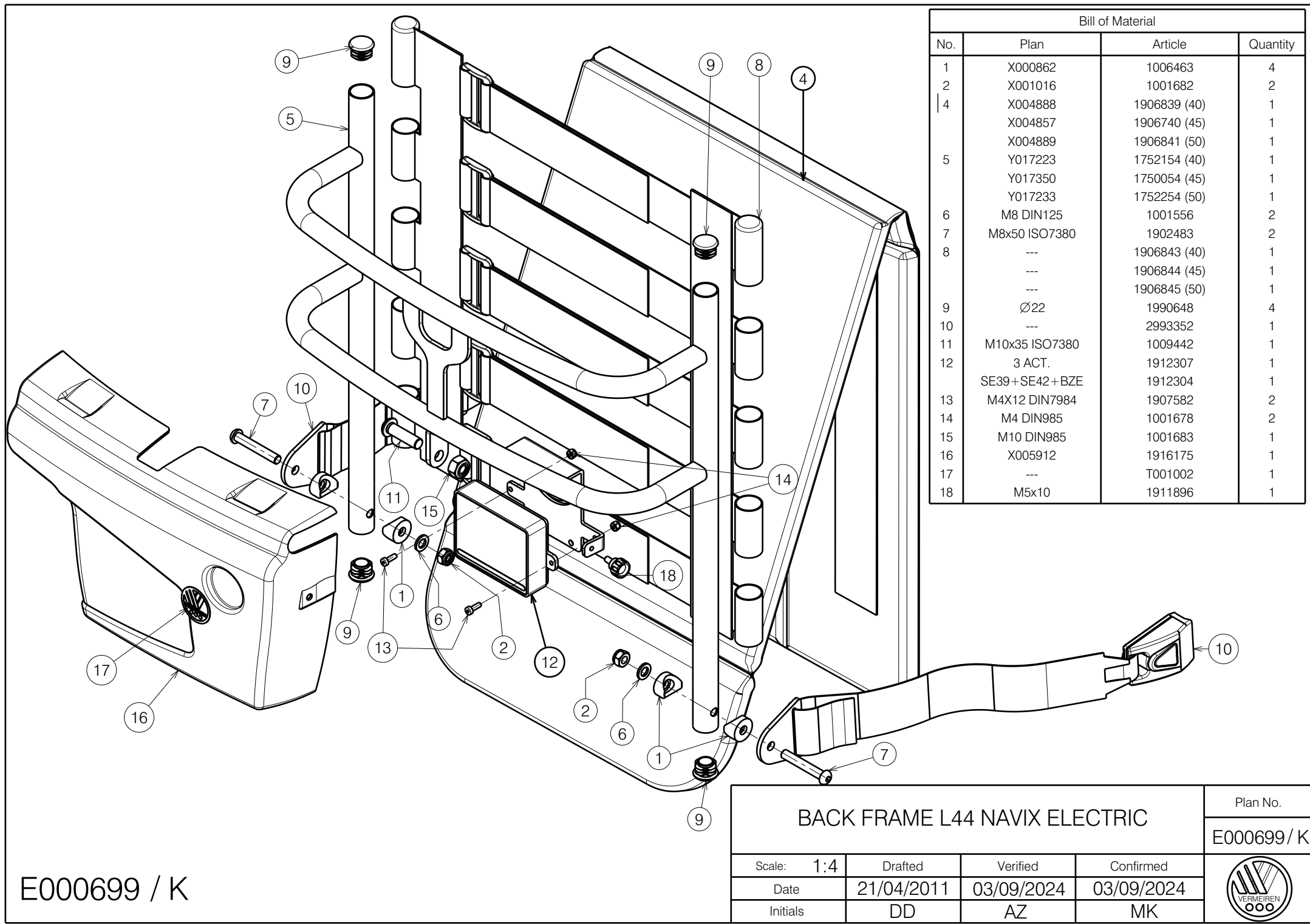

Bill of Material			
No.	Plan	Article	Quantity
1	X000862	1006463	4
2	X001016	1001682	2
4	X004888	1906839 (40)	1
	X004857	1906740 (45)	1
	X004889	1906841 (50)	1
	Y017223	1752154 (40)	1
5	Y017350	1750054 (45)	1
	Y017233	1752254 (50)	1
	M8 DIN125	1001556	2
6	M8x50 ISO7380	1902483	2
8	---	1906843 (40)	1
	---	1906844 (45)	1
	---	1906845 (50)	1
9	Ø22	1990648	4
10	---	2993352	1
11	M10x35 ISO7380	1009442	1
12	3 ACT.	1912307	1
	SE39+SE42+BZE	1912304	1
13	M4X12 DIN7984	1907582	2
14	M4 DIN985	1001678	2
15	M10 DIN985	1001683	1
16	X005912	1916175	1
17	---	T001002	1
18	M5x10	1911896	1

E000699 / K

BACK FRAME L44 NAVIX ELECTRIC				Plan No.
				E000699 / K
Scale: 1:4	Drafted	Verified	Confirmed	
Date	21/04/2011	03/09/2024	03/09/2024	
Initials	DD	AZ	MK	